


SENTRA HD Silver Vein

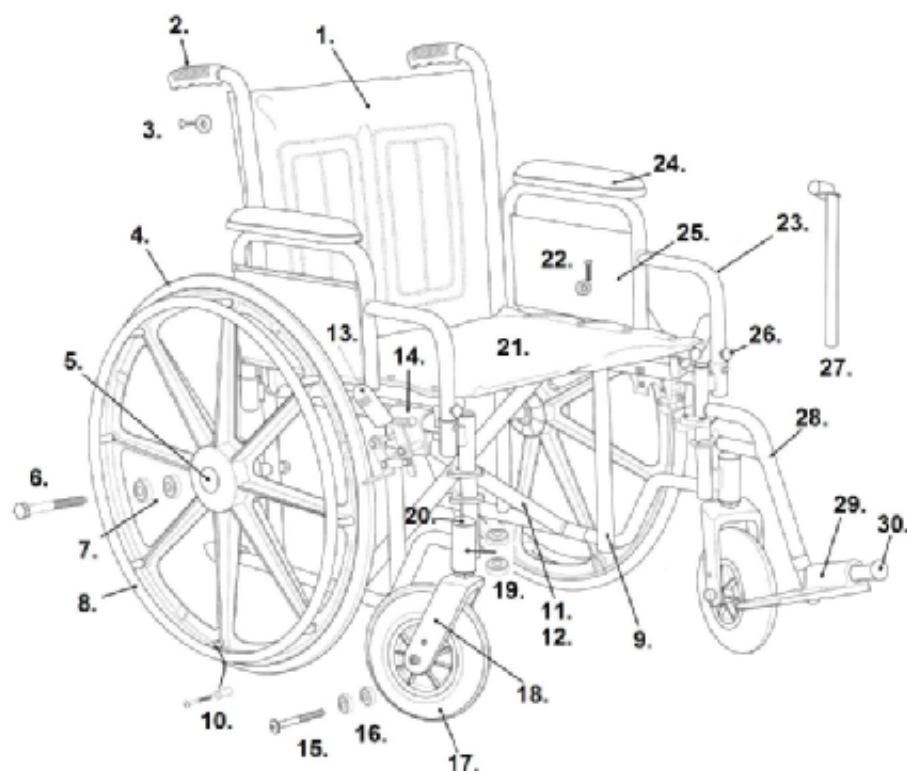

- | | |
|---|--|
| 1. Back Upholstery 20"- STDS2329BLK
Back Upholstery 22"- STDS2331BLK
Back Upholstery 24"- STDS2333BLK | 14. Wheel Lock Left- STDS4010L
Wheel Lock Right- STDS4010R |
| 2. Handgrip- STDS871 | 15. Front Axle Bolt- STDS1014 |
| 3. Back Upholstery Hardware (8 ea)- STDS5D5504 | 16. Caster Bearings- STDS1011 |
| 4. Rear Wheel Assy. s/n 1S & 4S- STDS1S1000HD
Rear Wheel Assy. s/n Y- STDS1000HD | 17. Caster- STDS1075 |
| 5. Hub Cap s/n 1S & 4S- STDS1055
Hub Cap s/n Y- STDS1000C | 18. Caster Fork- STDS3045-2 |
| 6. Rear Axle Bolt s/n 1S & 4S- STDS1S1053
Rear Axle Bolt s/n Y- STDS1052 | 19. Fork Stem Bearing- STDS1060 |
| 7. Rear Axle Bearing s/n 1S & 4S- STDS1S1012
Rear Axle Bearing s/n Y- STDS1012HD | 20. Fork Stem Dust Cap- STDS1051 |
| 8. Hand Rim s/n 1S & 4S- STDS004010
Hand Rim s/n Y- STDS006008 | 21. Seat Upholstery 20"- STDS2A2328BLK
Seat Upholstery 22"- STDS2A2330BLK
Seat Upholstery 24"- STDS2A2332BLK |
| 9. Side Frame s/n 1S or 4S Right- STDS4S521R
Side Frame s/n 1S or 4S Left- STDS4S521L | 22. Seat Upholstery Hardware (8 ea)- STDS001062 |
| 10. Hand Rim Hardware (8 ea)- STDS1016-8 | 23. Arm Assy. Adj. Desk Lt- STDSADDASPLSV
Arm Assy. Adj. Desk Rt- STDSADDASPR-SV |
| 11. Cross Frame 20"- [1]
Cross Frame 22"- STDS2A5222
Cross Frame 24"- [1] | 24. Arm Pad Desk- STDS9DSKVB |
| 12. Cross Brace Bolt- STDS001061 | 25. Arm Rest Panel Desk- STDS002044 |
| 13. Wheel Lock Tip- STDS8M4603 | 26. Arm Release Button- STDS8B |
| | 27. Slide Tube- STDS005200 |
| | 28. Footrest- STDSF-TF
Elevating Leg Rest- STDEL-R-TF |
| | 29. Footplate- STDS00004A |
| | 30. Rubber Bumper- STDS1026 |

[1] The part is no longer available.