

LINX FLASH CODES

Flash Code	Error	Solution
1 Flash	Joystick Error	Check cables and connections. Replace joystick.
2 Flash	Network or Configuration Error	Check cables and connections. Check Bluetooth pairing. Reconfigure the system. Recharge the batteries. Check charger. Replace controller.
3 Flash	Left Motor Error	Check cables and connections. Test and replace motor if bad. Replace controller.
4 Flash	Right Motor Error	Check cables and connections. Test and replace motor if bad. Replace controller.
5 Flash	Left Brake Error	Check cables and connections. Verify brake is released. Test and replace brake if bad.
6 Flash	Right Brake Error	Check cables and connections. Verify brake is released. Test and replace brake if bad.
7 Flash	Module Error	Check cables and connectors. Check controllers. Replace LINX Access Key. Replace controllers. Recharge batteries. If the chair stopped