

# BOBCAT X

## FLASH CODES

Flash Code	Description	Recommended Action
1	Low Battery Voltage	Charge battery
2	Motor Disconnect	Disconnect & Reconnect front and rear section
3	Motor Wiring Trip	Check motor connections
4	Freewheel Trip	Place the scooter in "drive" mode
5	N/A	N/A
6	Charger Inhibit	Disconnect the charger
7	Throttle Trip	Place throttle in neutral position
8	Controller Trip	Check connections at controller
9	Brake Trip	Place the scooter in "drive" mode
10	High Battery Voltage	Check battery connections