

CTE FLASH CODES

When the CTE controller senses a fault a number battery indicator LED lights will flash.

Flash Code	Meaning	Possible Reasons and Solutions
1	Low Battery	The battery voltage has dropped below a set value.
2	Low Battery	The battery voltage has dropped below a set value.
3	High Voltage	The batteries are overcharged. If driving downhill slow down and turn lights on, if so equipped. Test the charge to ensure that it cuts off when batteries are fully charged.
4	Motor Fault	The motor current has been at the maximum value too long. The motor may not be strong enough for the terrain (terrain too steep). The wheels may be rubbing on the frame. The motor may be faulty. Test the motor. Check the motor brushes. The controller overheats. Turn the scooter off and let controller cool down.
5	Brake Fault	Brake is disengaged, engage the brake, turn the scooter off for 5 seconds. Check the brake connections and wiring. Test the brake.
6	Throttle Fault	The throttle is misadjusted. Test and adjust the throttle. Replace the throttle if it cannot be adjusted.
7	Throttle Fault	Throttle was engaged when unit was turned on. Turn unit off for 5 seconds and turn on without engaging the throttle.
8	Motor Fault	Check the motor connections and wiring. Check the motor brushes. Test the motor.
9	Controller Fault	Check the controller connections and wiring. Replace the controller.