

COBRA FLASH CODES

The diagnostic flash codes for your scooter are designed to help you perform basic troubleshooting quickly and easily. The diagnostic flash code is displayed as one of seven flashing lights on the battery indicator. The farthest left light indicates flash code 1, while the farthest right light indicates flash code 7:



Please refer to the diagram below for flash codes, description of problem, and recommended actions.

Flash Code	Description	Recommended Action
1	Low Battery Voltage	Charge batteries. Check battery cables, connections, and fuses.
2	Thermal Fault	Power off the scooter, let rest for 15 minutes, power back on to resume driving.
3	Park Brake Fault	Place the freewheel lever in "drive" mode.
4	Throttle Fault	Make sure the throttle is in the neutral or rest position when powered on.
5	Throttle Fault	Defective throttle, contact your authorized Drive Medical Provider.
6	Motor Fault	Contact your authorized Drive Medical Provider.
7	Unknown Fault	Contact your authorized Drive Medical Provider.
All Lights	Over Voltage	Turn scooter off; contact your authorized Drive Medical Provider.

Once the recommended action is taken, turn the scooter off and then back on again to clear the flash code. If the problem was not corrected, or another problem exists the controller may display another flash code.