

# FALCON FLASH CODES

When the Falcon controller senses a fault, first there will be two short beeps to indicate that an error has occurred. Then the Falcon will beep one to six times to indicate the error.

Code	Meaning	Possible Reasons and Solutions
1	Low Battery	The battery voltage has dropped below a set value.
2, 3 or 4	Controller Fault	Check the controller connections and wiring. Replace the controller.
5	Throttle Fault	The throttle is misadjusted. Test and adjust the throttle. Replace the throttle if it cannot be adjusted.
6	Brake Fault	Brake is disengaged, engage the brake, turn the scooter off for 5 seconds. Check the brake connections and wiring. Test the brake.