

# MAVERICK FLASH CODES

Flash Code	Meaning	Possible Reasons and Solutions
1	Low Battery	Battery voltage has dropped below 23.3 volts. Charge the batteries as soon as possible.
2	Low Voltage	Battery voltage has dropped below 16.5 volts. Charge the batteries.
3	High Voltage	Unplug the charger and turn the scooter off, then on. Reduce speed going down a hill.
4	Current Limit Time Out	The scooter has drawn too much current for too long. Turn scooter off for a few minutes. If problem continues check wire harness and motor for shorts.
5	Brake Fault	Check that the freewheel lever is in the drive position. Check that the brake connection is plugged in.
6	Throttle Fault	Return throttle to neutral. Turn the scooter off then on. If problems continue recalibrate or replace throttle.
7	Throttle Fault	Check that the throttle is calibrated. Check for shorts in throttle wiring.
8	Motor Fault	Check for short in the motor and motor wiring.
9	Internal Controller Fault	Replace the controller.