

S DRIVE FLASH CODES

When the S Drive controller senses a fault a number battery indicator LED lights will flash.

Flash Code	Meaning	Possible Reasons and Solutions
1	Low Battery	The battery voltage has dropped below a set value.
2	Motor Fault	Check the motor connections and wiring. Check the motor brushes. Test the motor.
3	Wiring Fault	There is a short circuit between the motor and batteries. Check the connections and wiring between motor and batteries.
4	Brake Fault	Brake is disengaged, engage the brake, turn the scooter off for 5 seconds. Check the brake connections and wiring. Test the brake.
5	Not Used.	
6	Charger Fault	Disconnect the charger from the scooter.
7	Throttle Fault	The throttle is not in the neutral position. Make sure that the throttle is not engaged when turning on the scooter. Test and adjust the throttle.
8	Controller Fault	Check the controller connections and wiring. Replace the controller.
9	Brake Fault	Brake is disengaged, engage the brake, turn the scooter off for 5 seconds. Check the brake connections and wiring. Test the brake.
10	High Voltage	The batteries are overcharged. If driving downhill slow down and turn lights on, if so equipped. Test the charge to ensure that it cuts off when batteries are fully charged.