



10270KDRSV HOW TO ASSEMBLE

1. Verify that all parts are present.

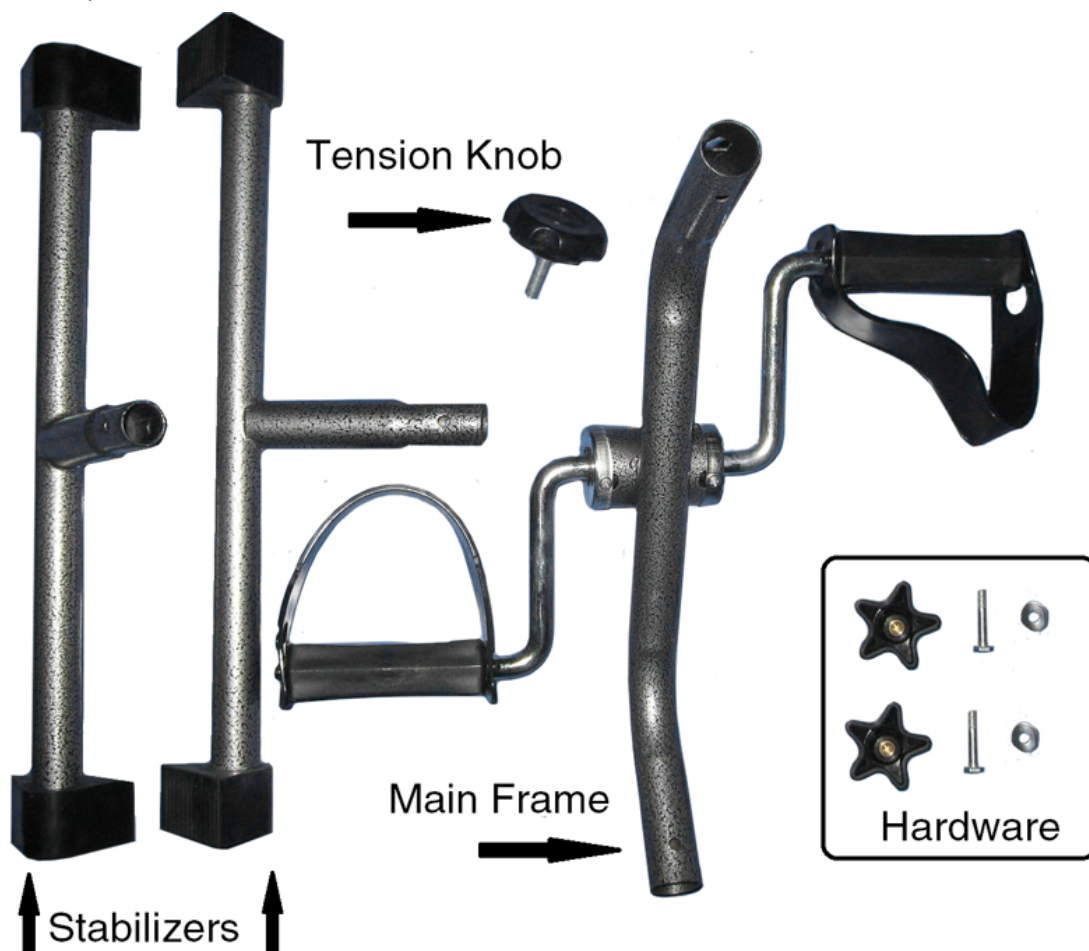

2. Insert one of the stabilizers into the main frame. Verify that the stabilizer is in the correct position.

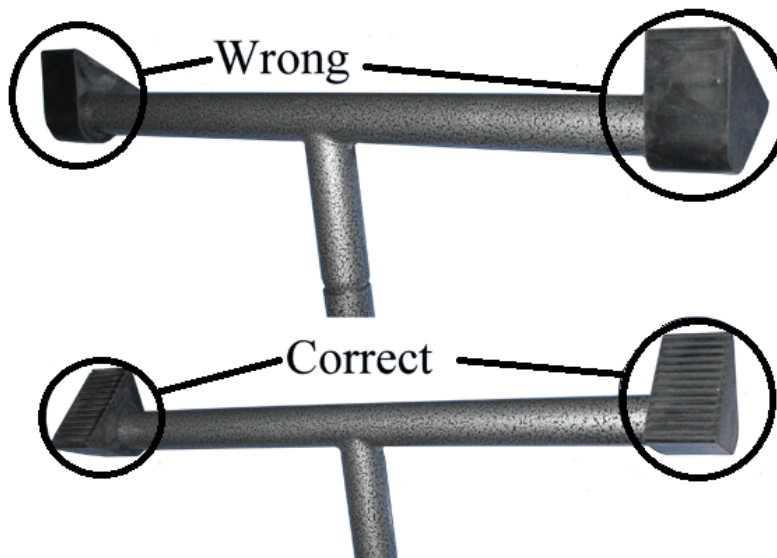




3. Insert one of the bolts through the main frame and stabilizer.




4. Make sure the bolt is pushed into the six sided cut out in the main frame.




5. Install the one of the cove washers on the bolt with the concave side facing the main frame.

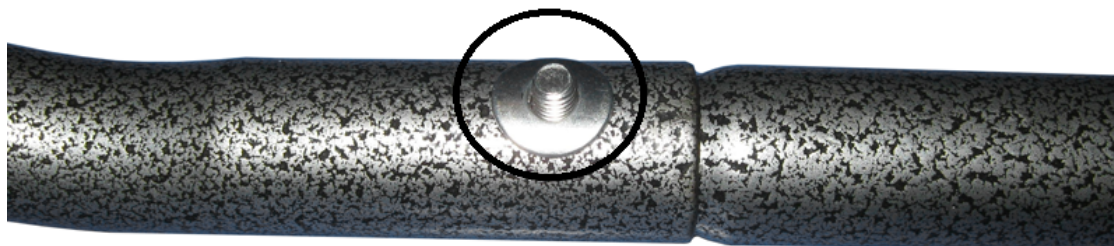

6. Install one of the small knobs on the bolt and tighten. Repeat steps 2 to 6 for the other stabilizer.

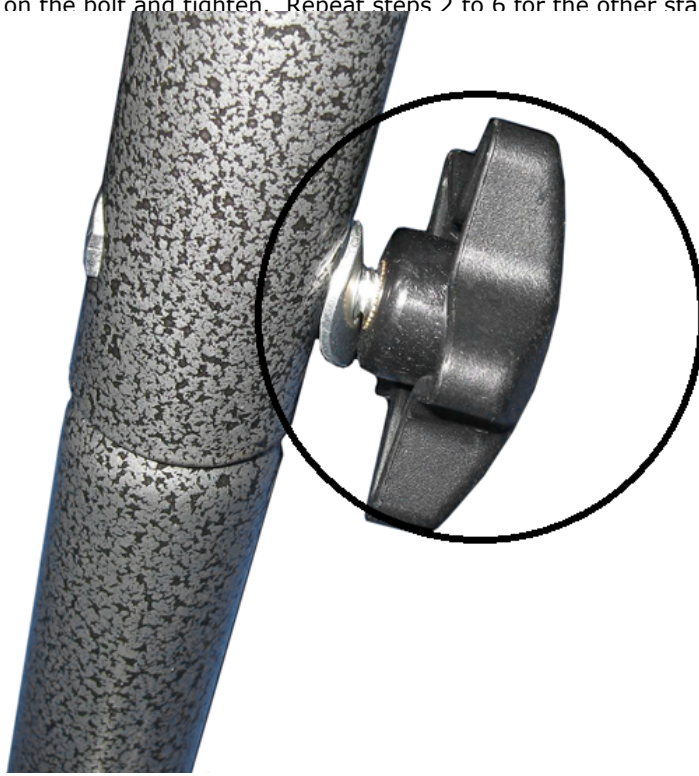

7. Attach the Tension Adjusting Knob on to the main frame

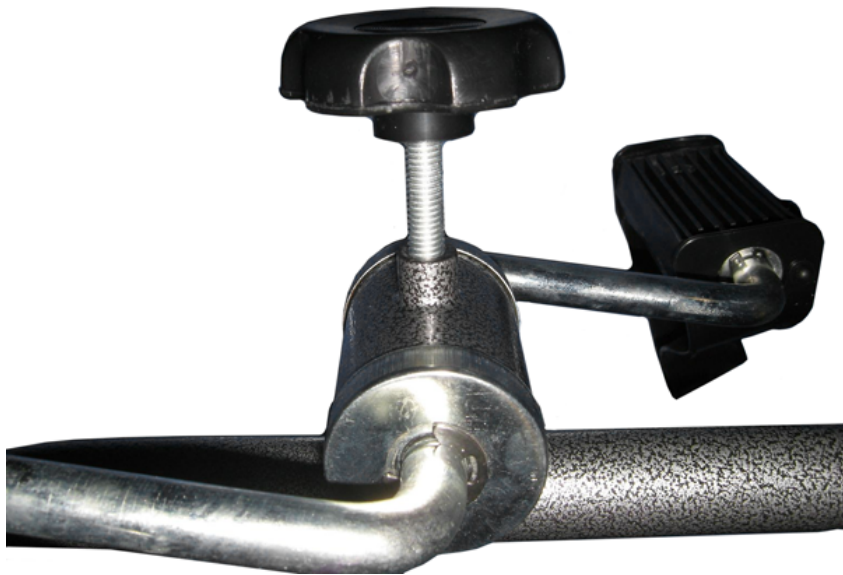

[Home](#)