

10273 BATTERY HOW TO CHANGE

1. Slide a thin flat head screw driver in to the slot on one side and pry up slightly.



2. Place the screw driver into the gap above the slot and pry up.



3. Repeat on the other side and continue to pry up until the electronics are separated from the main body of the exercise peddler. It is possible that one of the locking tabs may break. This will not have any effect on the unit.





4. Bend the battery tab on the facing the inside of the electronics.



5. Push the old battery out.



6. Slide the new battery in and bend the tab back down.





7. Align the electronics with the main body of the exercise peddler and push in.



Home