


13046 & 13046HD INSTALLATION

A. INSTALLING THE ADAPTER TO THE LIFT ARM (MAST)


1. Slide the 12mm flat washer onto the 12mm bolt.




2. Partially insert the 12mm bolt into the first hole in the lift arm.

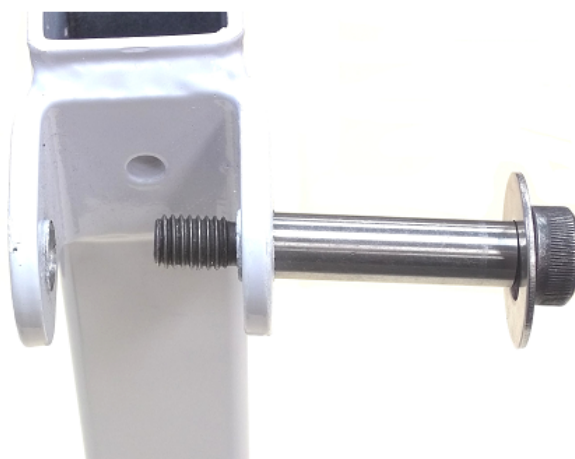

3. Then slide a 12mm spacer onto the bolt.

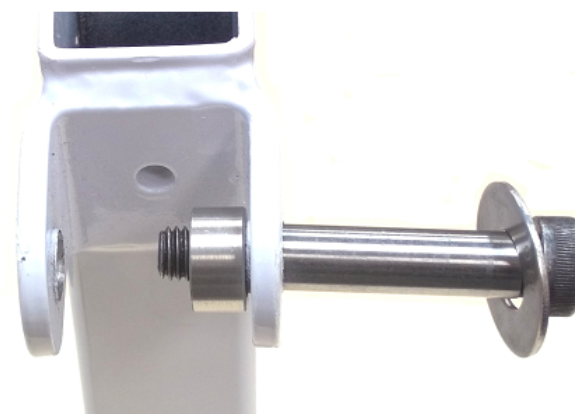


4. Pass the bolt through the single hole on the adapter.

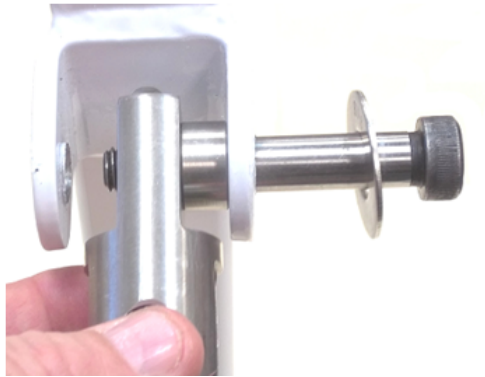


5. Next, slide another 12mm spacer onto the bolt.




6. Now, push the bolt through the second hole in the lift arm.

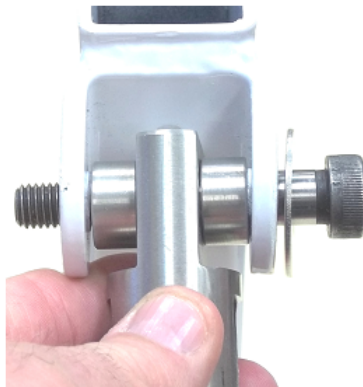

7. Add a 10mm flat washer, and then thread on and tighten a 12mm locknut.

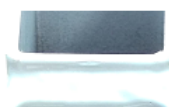

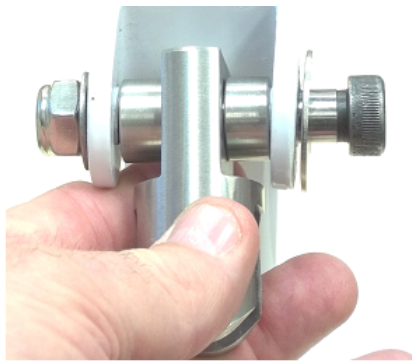


B. MOUNTING THE SCALE TO THE ADAPTER


1. Slide a 10mm washer onto the 3/8" x 1 1/4" bolt.



2. Insert the scale top mounting flange into the groove in the adapter, and slide the bolt through all three holes.

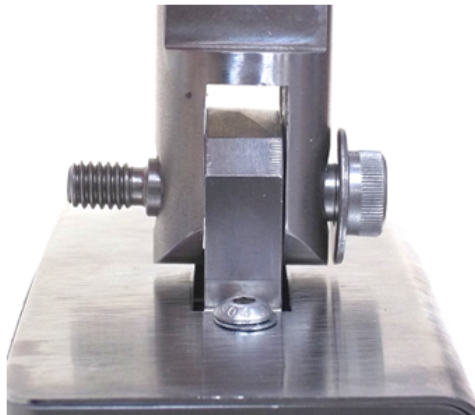


3. Slide another 10mm washer over the bolt and thread on and tighten a 5/16" locknut.

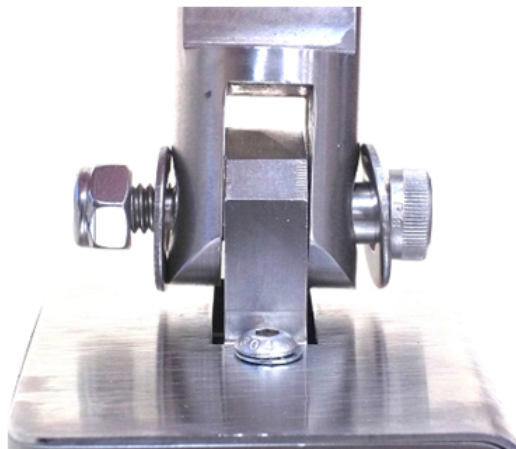

C. ATTACHING THE CRADLE

1. Slide a $\frac{1}{4}$ " spacer over a 2" long, $\frac{3}{8}$ " bolt.

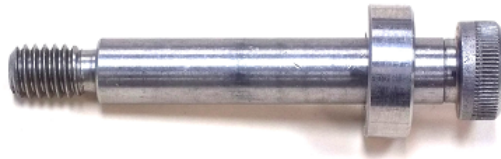

2. Pass the bolt through a link.




3. Slide another spacer onto the bolt and pass the bolt through the bottom flange of the scale.



4. Now, add another $\frac{1}{4}$ " spacer and pass it through the remaining link.



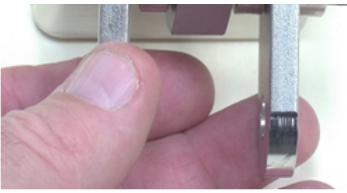

5. Install and tighten a 5/16" locknut.



6. Slide the pin through the yoke of the cradle.




7. Slide a 1/4" spacer onto a 2" bolt.




8. Pass the bolt through the first link.




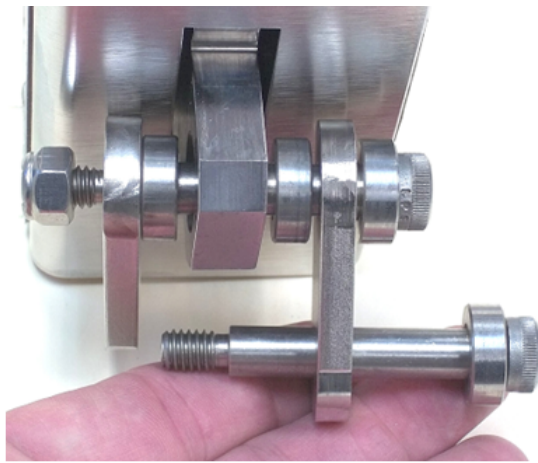


9. Pass the bolt through the pin.

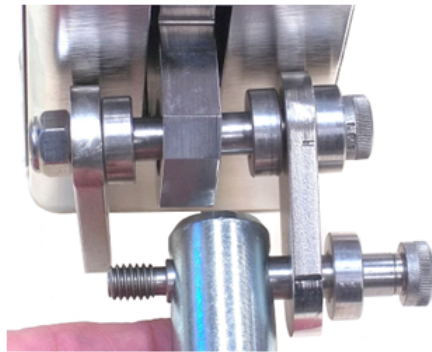


10. Pass the bolt through the second link.

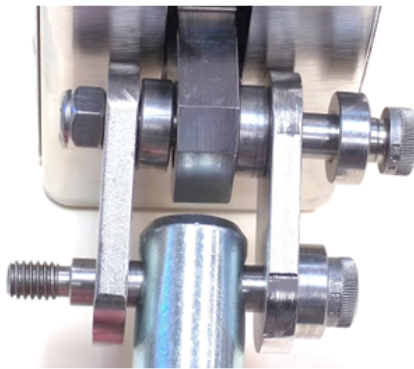

11. Slide a 1/4" spacer over the protruding bolt, and install and tighten a 5/16" locknut.

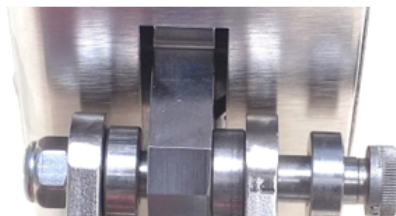

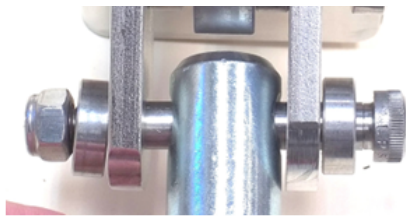

[Home](#)