


# 13240 POWER PATIENT LIFT ASSEMBLY

1. Unpack the unit. Identify and verify that all parts are present.




2. Remove the Allen head cap screw from the base of the unit with the provided Allen wrench.




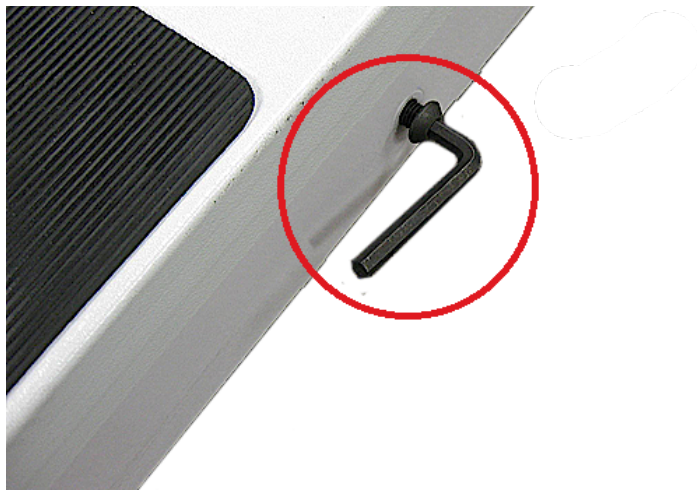


3. Remove the Allen head cap screw from the mast of the unit with the provided Allen wrench.

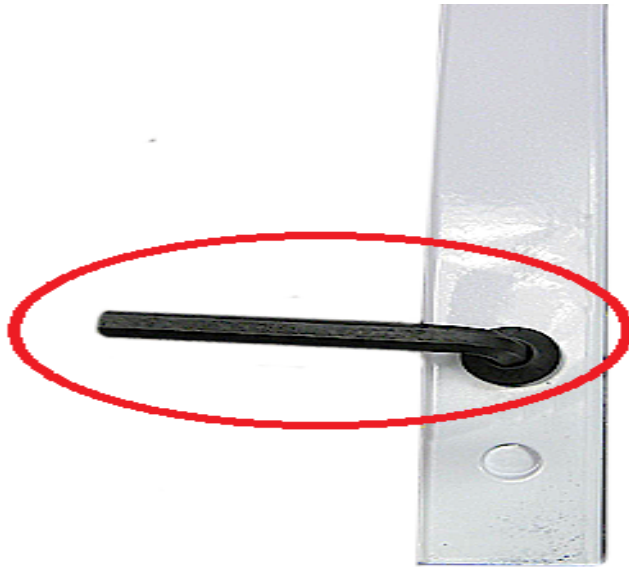


4. Install the mast into the base with the boom towards the front of the base.

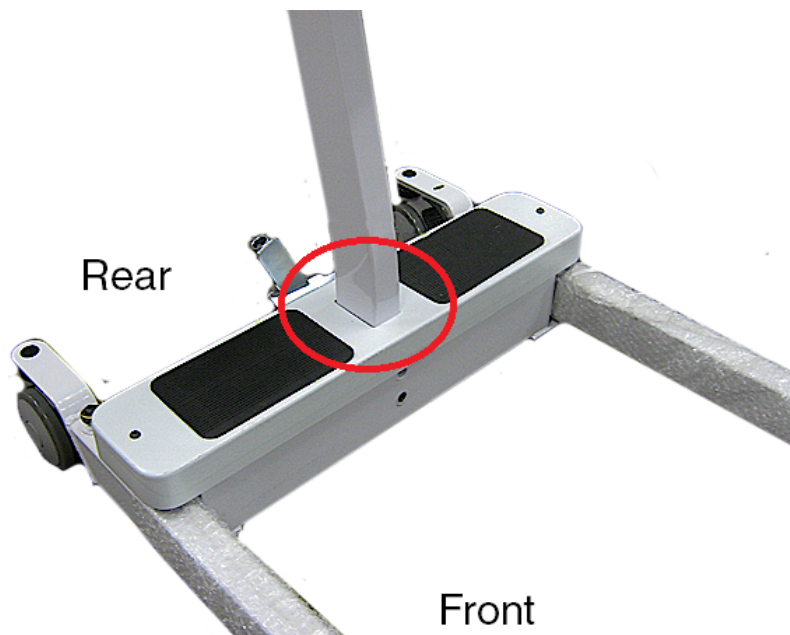


5. Install and tighten the two Allen cap screws with the provided Allen wrench. The longer cap screw goes on top and the shorter cap screw goes on the bottom.

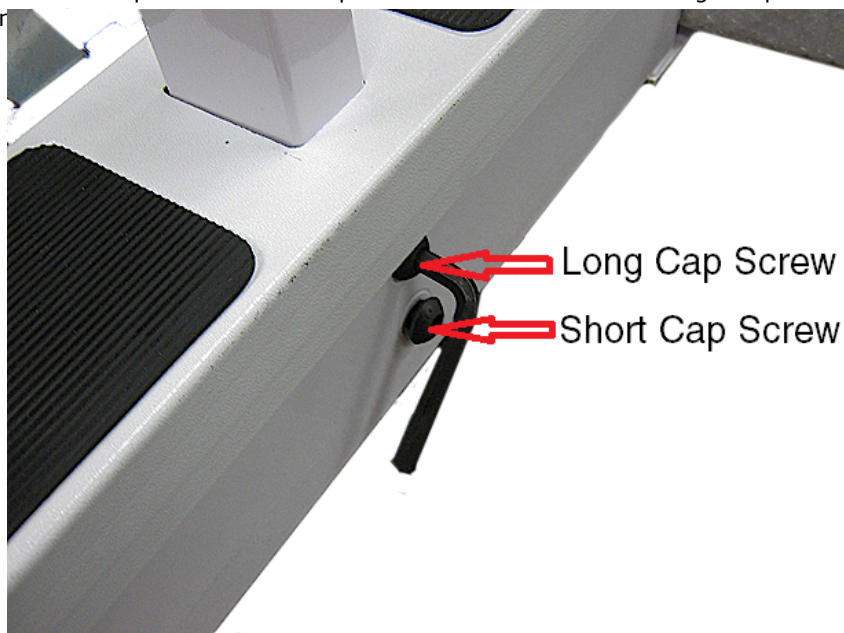

6. Remove the Allen cap screw from the upper and lower push handle posts with the Allen wrench provided.

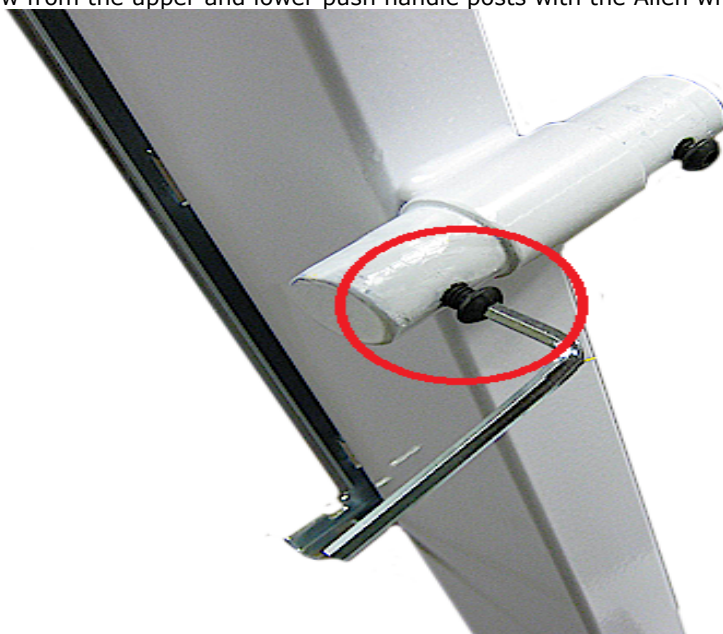

7. Slide one of the push handles over the upper and lower posts. Make sure the holes line up.

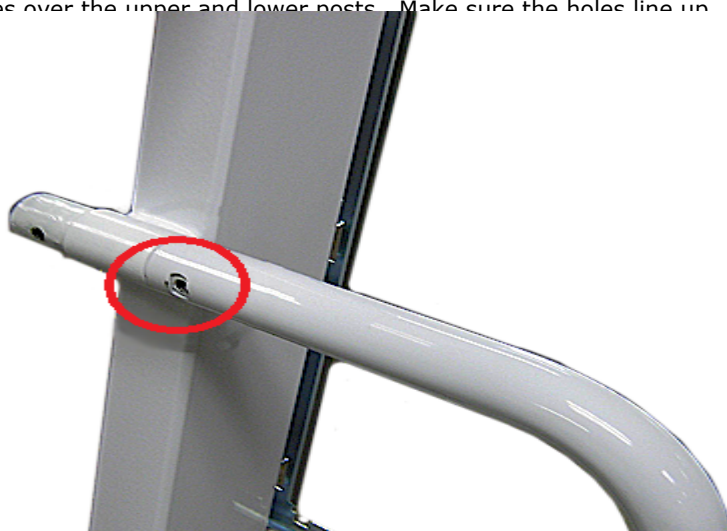




8. Install and tighten one of the Allen caps screw that were remove in the upper and lower push handle post. Repeat steps 7 and 8 for the other side.




9. Remove the small screw

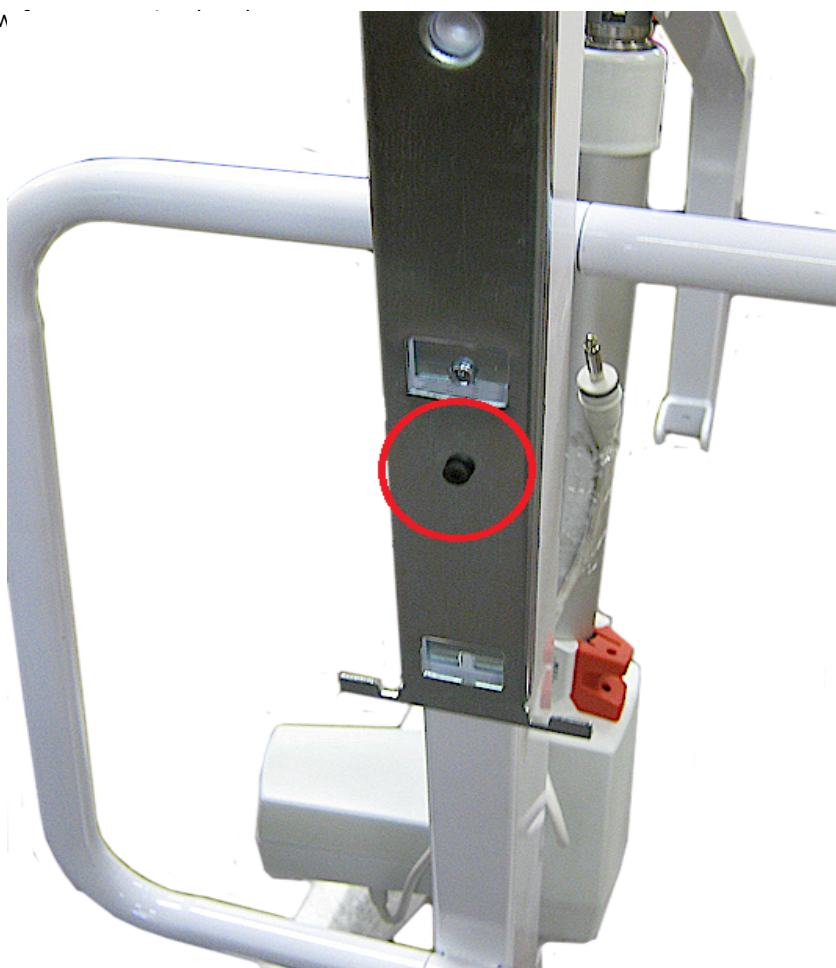


10. Slide the control box on to the lower bracket and install the small screw previously removed.



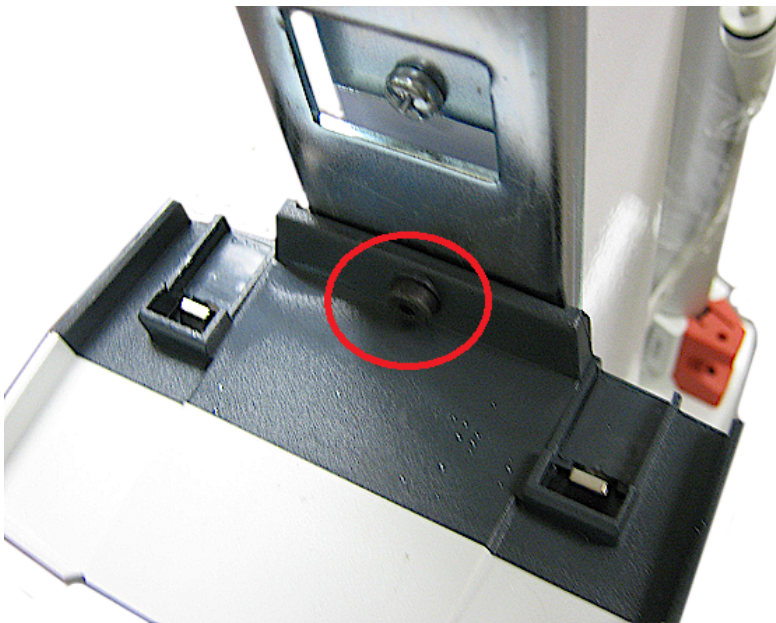


11. Line up the battery pack with the control box.

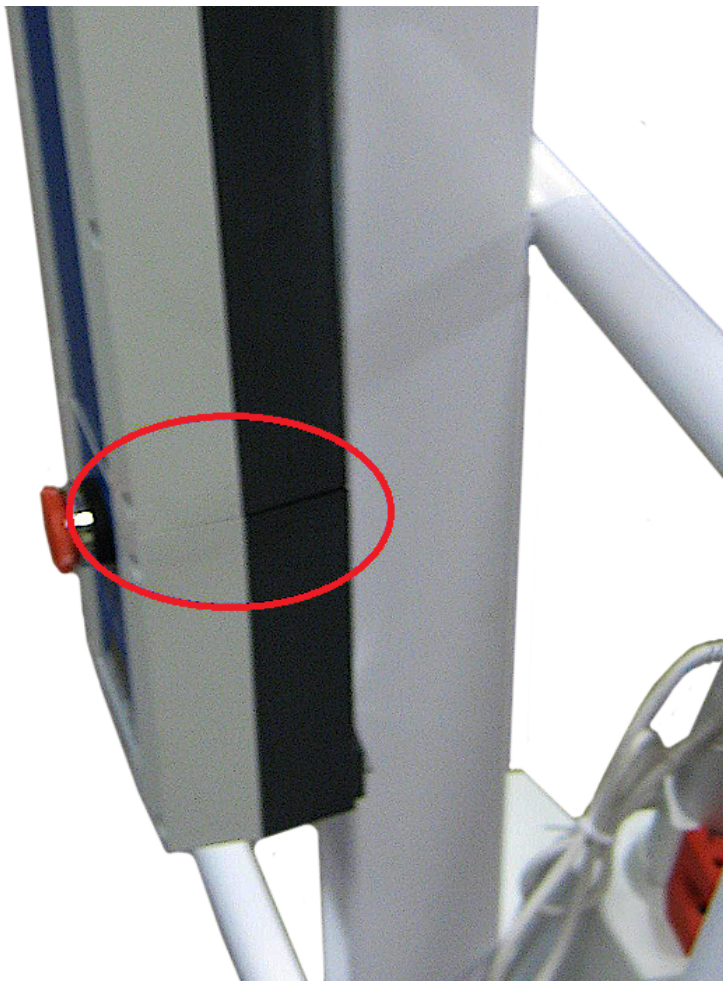

12. Pull up on the locking latch release on the battery pack.




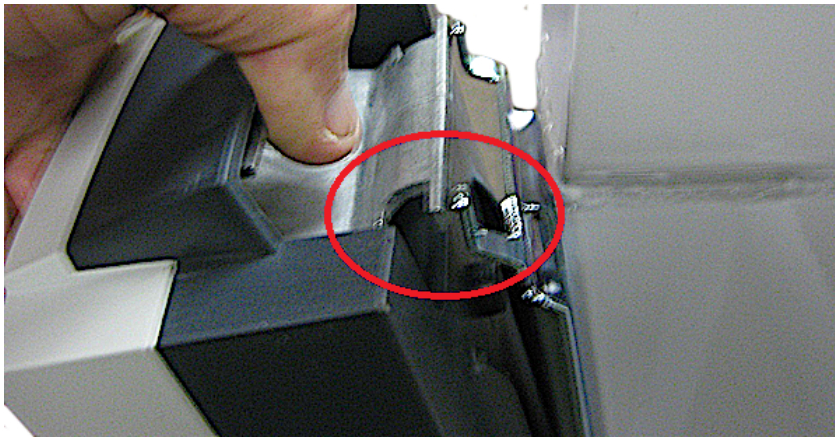


13. Push the battery pack





14. Verify that the emergency stop button is not pushed in. If the emergency stop button is pushed in, turn the button to reset it to the out position. The emergency stop button has to be in the out position for the lift to operate.

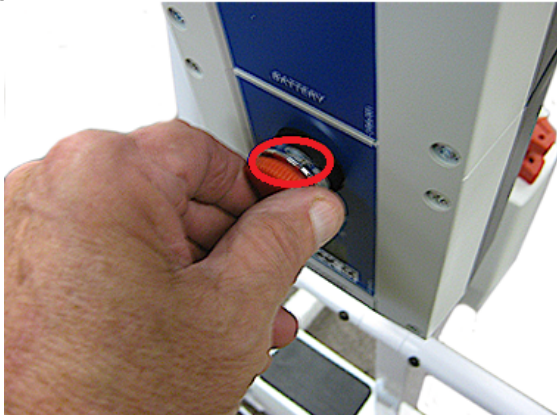

15. Insert the spreader bar into the receiver plate and depress the locking button. Continue to insert the spreader into the receiver plate until it locks into place.

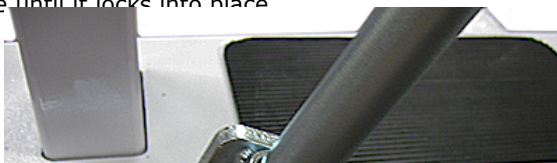




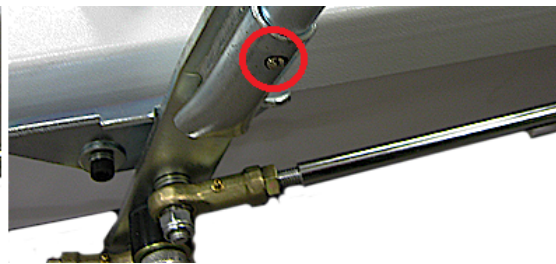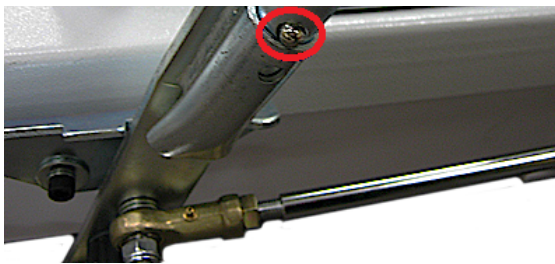

16. Insert the cradle swivel pin and nylon washer into the cradle.



17. Install the nylon cover

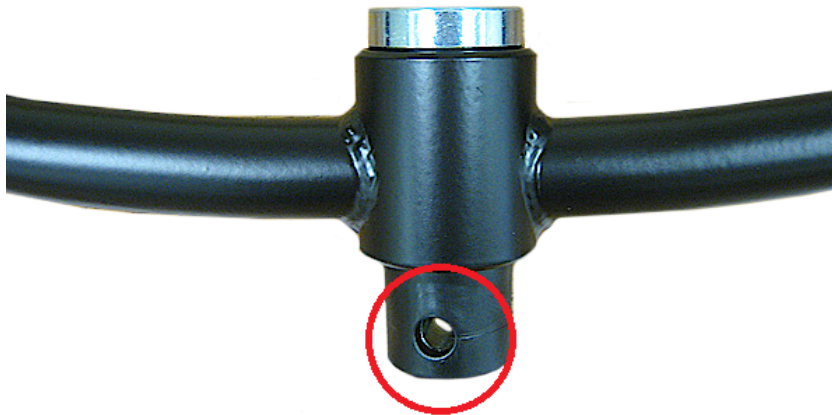

18. Align the holes of the cradle with the holes of the frame. Insert the bolt through the holes.





19. Install the nut on the b




20. Install the nut cap on t



21. Align the tab on the plug of the



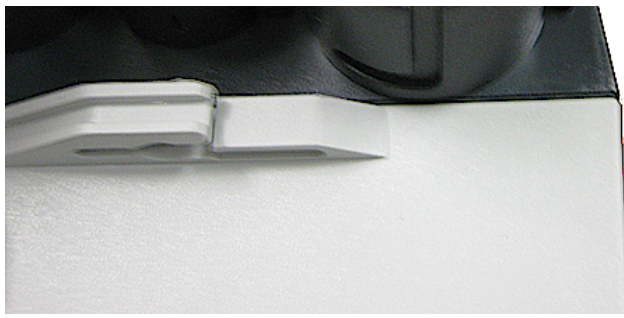

22. The plug must be push  
amount of petroleum jelly

g the plug a very small




23. Put for the connector for the motor actuator cable into the ground socket of the controller.



24. The plug must be pushed in fl  
amount of petroleum jelly on the

ting the plug a very small




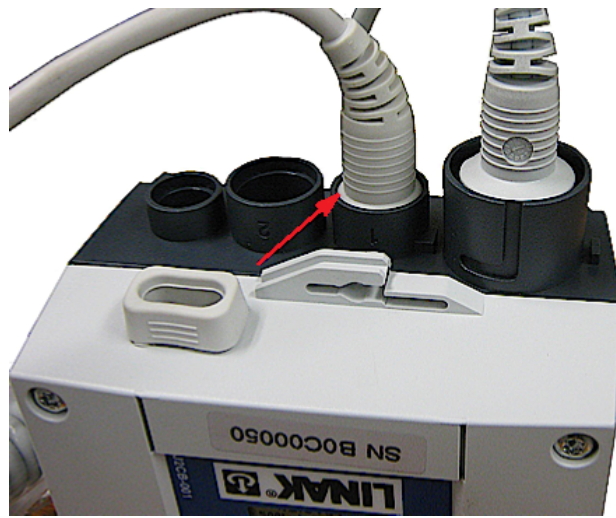

25. The lift is now fully ass




26. Charge the  
indicated by t

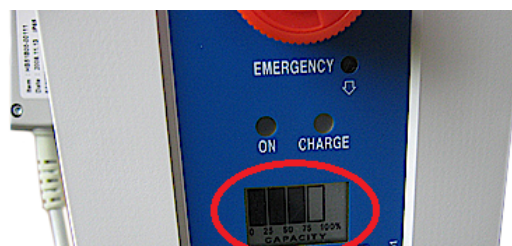


battery is  
lift to work.



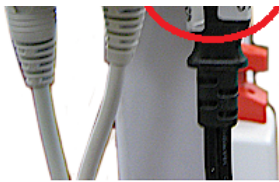

27. To spread

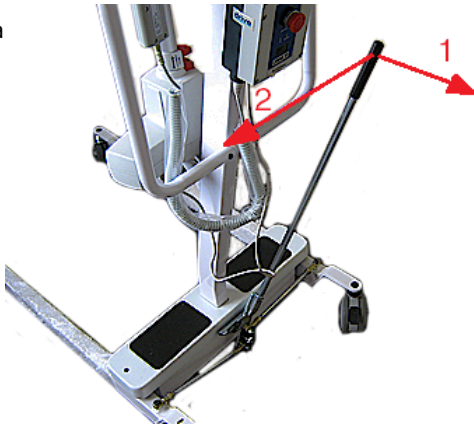

28. To operate the lift us



Home