

15500 ASSIST POLE HOW TO ASSEMBLE

1. Identify the top end of the 15500 E-Z Assist Pole.



2. Identify the top plate.



3. Insert the top plate into the top of the 15500 E-Z Assist Pole.






4. Align the holes in the top bracket with the holes in the pole.



5. Locate the clevis pin, washer and hairpin.

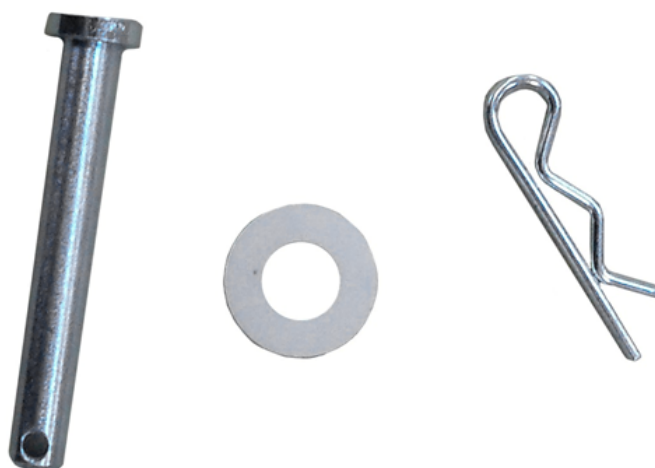

6. Slide the clevis pin through the holes in the top plate and E-Z Assist Pole.





7. Slide the nylon washer over the end of the clevis pin and install the hairpin.



8. Slide the protective cover over the bottom end of the E-Z Assist Pole with the anti-rattler towards the top of the pole.




9. Slide the protective cover up high enough so that the jam nut is exposed. Adjust the length of the E-Z Assist Pole 2 inches less than the total ceiling height.



10. Place the jam nut cup into the base.



11. The top bracket must be positioned between two ceiling joists.

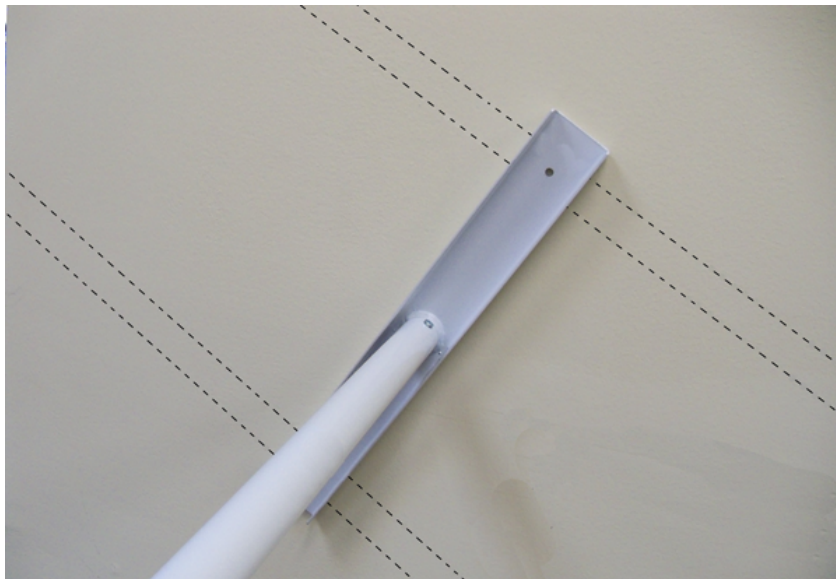


12. Insert the jam nut into the jam nut cup on the base plate.



13. Turn the jam nut until the E-Z Assist Pole is tight between the floor and the ceiling.



14. Slide the protective

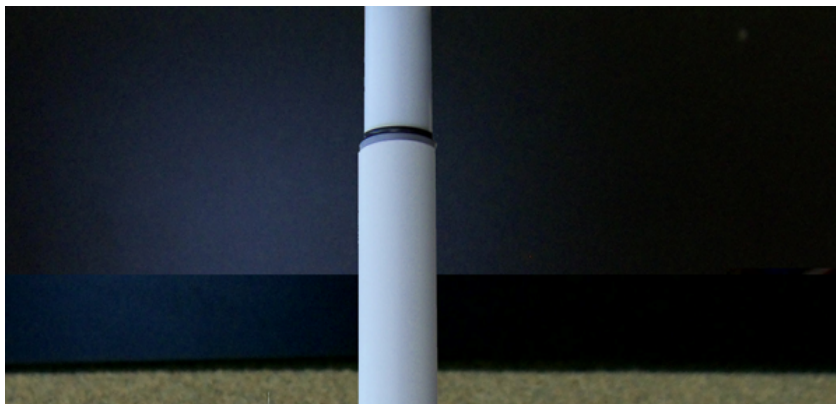



15. The top bracket sho



Home