

HOW TO INSTALL AMPUTEE ADAPTERS

1. Remove the one rear wheel by removing the hup cap and then unloosen and remove the rear axle.


2. Remove the antitipper or tipping sleeve.



3. Choose the amputee adapter that has the longer side of the axle bushing on the same side as the longer side of the axle bushing of the side of the wheelchair the adapter is being installed on.


4. Slide the tube of the amputee adapter over the tube that had the antitipper in.

5. Take the nut and bolt off the upper bracket of the amputee adapter and continue sliding the amputee adapter onto the wheelchair until it can not go

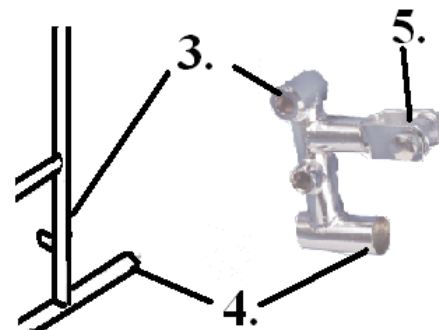

wheelchair until it can not go any further.


6. Insert the bolts through the upper amputee adapters brackets and install the nuts.

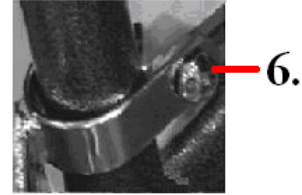

7. Install the rear wheels onto the amputee adapters.

8. Repeat steps 1 to 7 for the other wheel on the wheelchair.

9. Part number STDSBEW, brake extension wedges, must be used with amputee adapters.

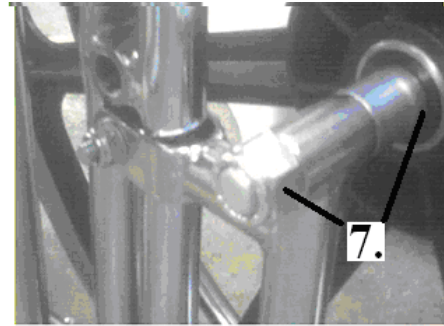

10. Install one of the STDSBEW, brake extension wedges, on each of the wheel locks on the wheelchair with the Allen wrench provided. Adjust the wheel lock as needed.

