



ZOOME FLEX AUTO HOW TO DISABLE BACK-UP ALARM




1. Locate the two screws on the rear shroud and remove

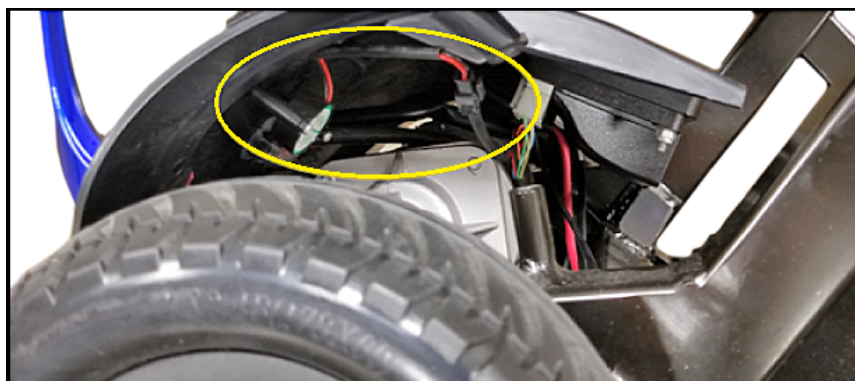

2. Turn scooter on side and locate two screws on the underside of the rear shroud and remove (or loosen enough to gain access to the back up alarm)

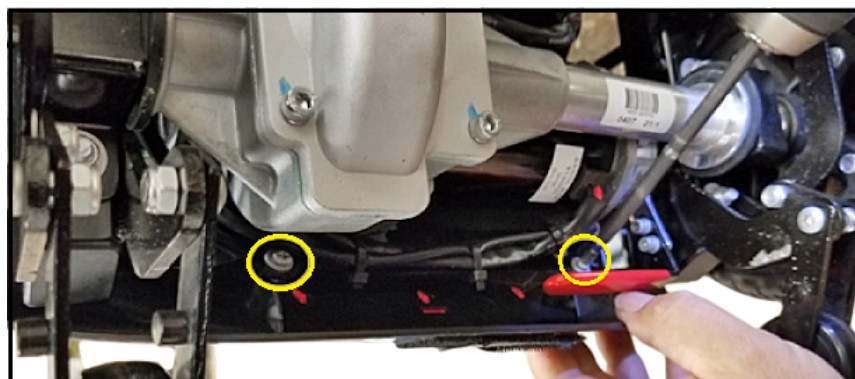


3. Disconnect harness connector from back-up alarm

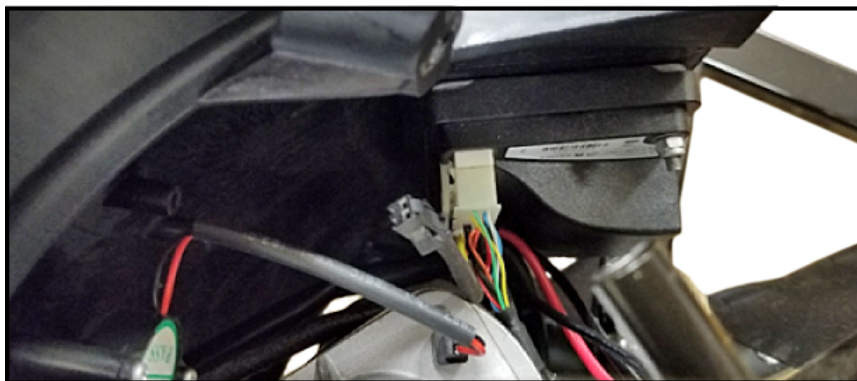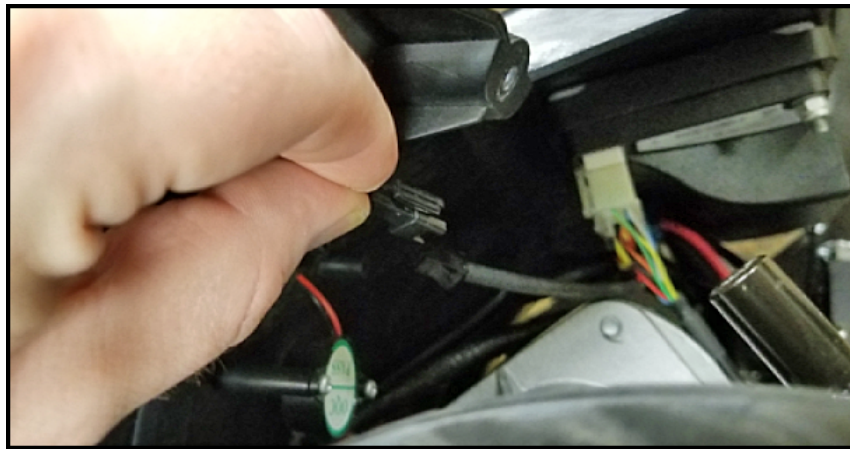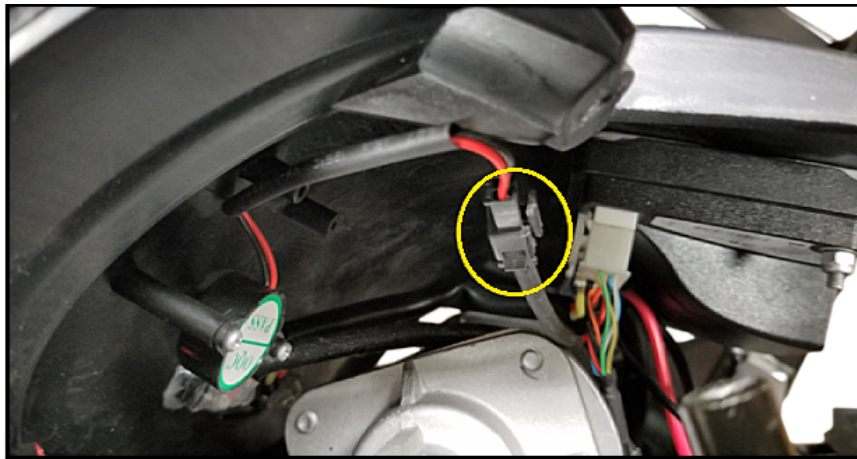

Backup alarm is now disabled

4. Replace and tighten all four shroud screws

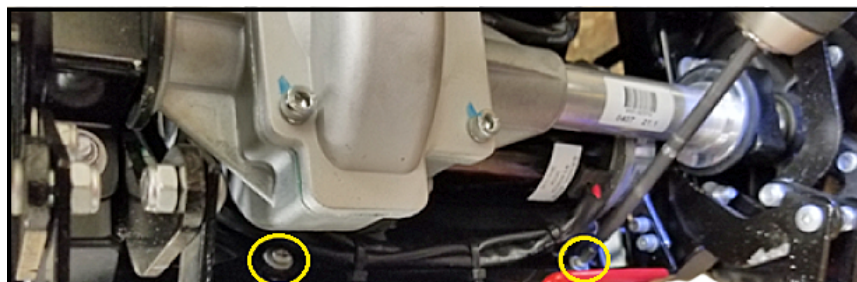



5. Test scooter to make sure the back-up alarm is disabled and the scooter functions properly



****NOTE** - Please be aware by disabling this back-up alarm you are disabling a safety feature of this device. We will not be responsible for any adverse incidents caused by the disabling of this back-up alarm**

[Home](#)