

# IMAGE EC HOW TO ASSEMBLE

1. Open top of the box using caution to avoid all metal staples



2. . Remove top packing material and seat assembly.



3. Have two people lift the scooter out of the box.






4. Remove all packing material and verify that all parts are present.



5. The batteries are shipped disconnected and must be reconnected. To access the batteries lift of the battery cover located in the rear of the base.



6. Locate the disconnected battery connector and wire harness connector.

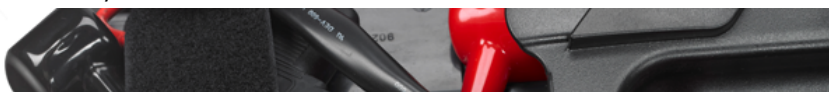



7. Reconnect the battery connector to the connector in the wire harness.



8. Locate the seat plate protection cap on the bottom of the seat assembly.





9. Remove the protection cap and discard.



10. Install the seat assembly onto the Inners FC base by sliding the seat plate into the seat tube.



11. Lift up on the seat system lever and raise the seat to the desired height. The seat can be raised to the seat height.





12. Install the left armrest



13. Set the arm to the desired height



14. The height of the arm can be adjusted by loosening the arm height adjustment knob and moving the upper arm assembly to the desired height.





15. Repeat steps 12 thro



16. Insert the joystick arm into the joystick receiver located under the right armrest. Set the joystick to the desired length and tighten the j





17. Squeeze the headrest adjustment button on the right headrest receiver. Insert the headrest into both headrest receivers.



18. Set the headrest to the desired position.



19. Plug the AC end of the







20. The power light on the



21. Verify that the chair is in the correct position and that the battery is properly connected.



22. Plug the DC end of the







23. The charge light on t



24. When the batteries a  
the wall outlet.




er from the chair and

25. The chair is now read





Be aware with all the fi

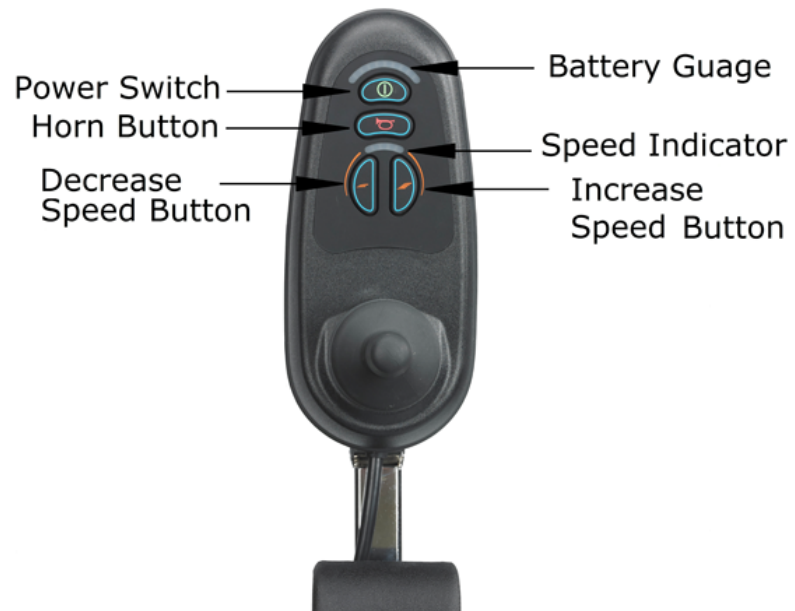

Home