

NEB KIT 500

HOW TO TROUBLE SHOOT

Troubleshooting Guide for Nebulizer Cups

If you notice that your nebulizer kit is not producing a mist, follow these steps.

1. Turn the compressor off
2. Remove the cap from the nebulizer cup by turning the cap counterclockwise
3. Remove the baffle from the nebulizer cup and rotate it ¼ turn
4. Return the baffle to the nebulizer cup
5. Place the cap back on the nebulizer cup and tighten it by turning it clockwise
6. Turn the compressor back on and confirm mist is coming from nebulizer cup, if not, turn the baffle another ¼ turn and try again.

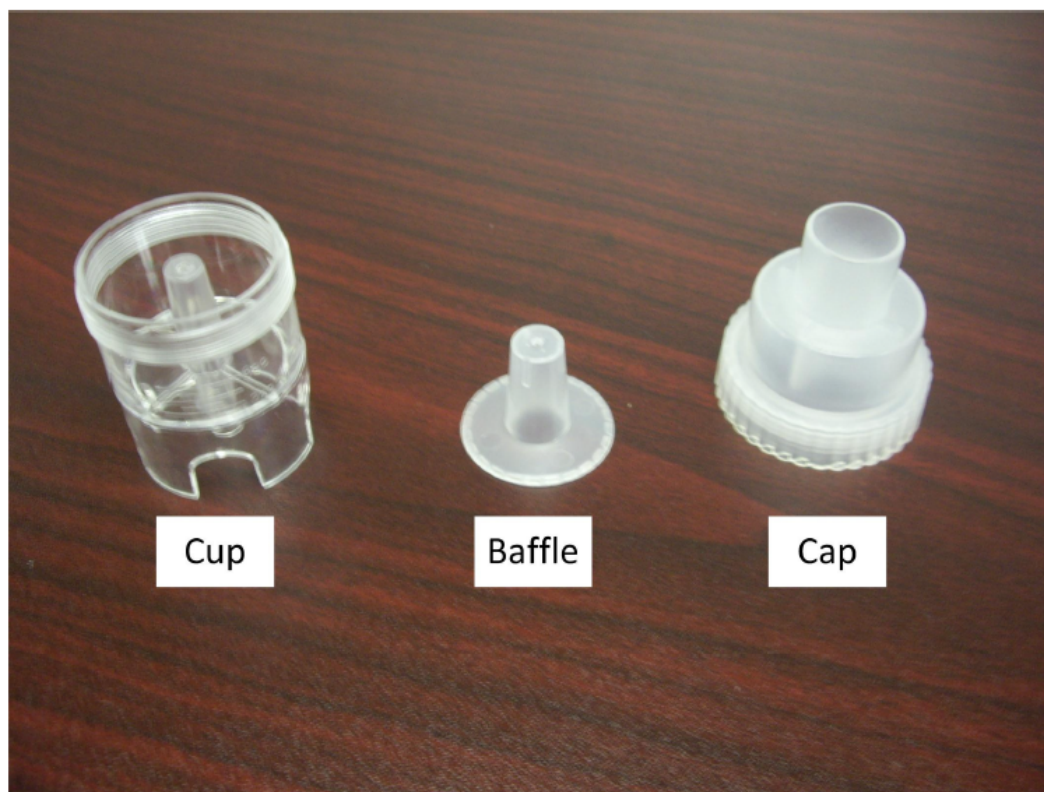




Figure2. Parts of assembled nebulizer cup

[Home](#)