


NEW DELTA MOTOR INSTALLED ON OLDER DELTA BEDS

1. Remove the old style motor mounting bracket.

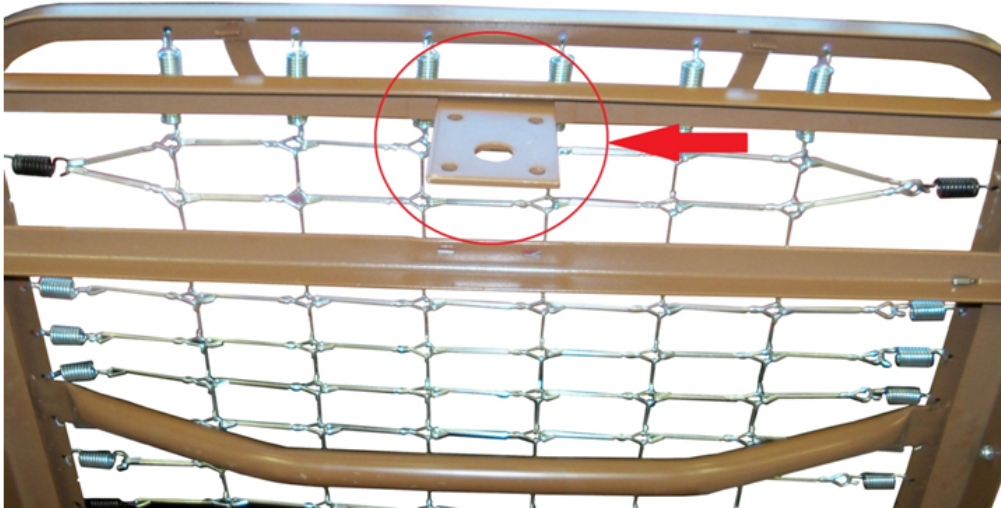


2. Verify that all the needed parts are available.




3. Install the 15544GBMB on to the frame. The 15544GBMB is mounted on the side away from the spring deck.



4. Install the 15033MBN-DELTA on the 15544GBMB. The 15033MBM-DELTA faces away from the spring deck.

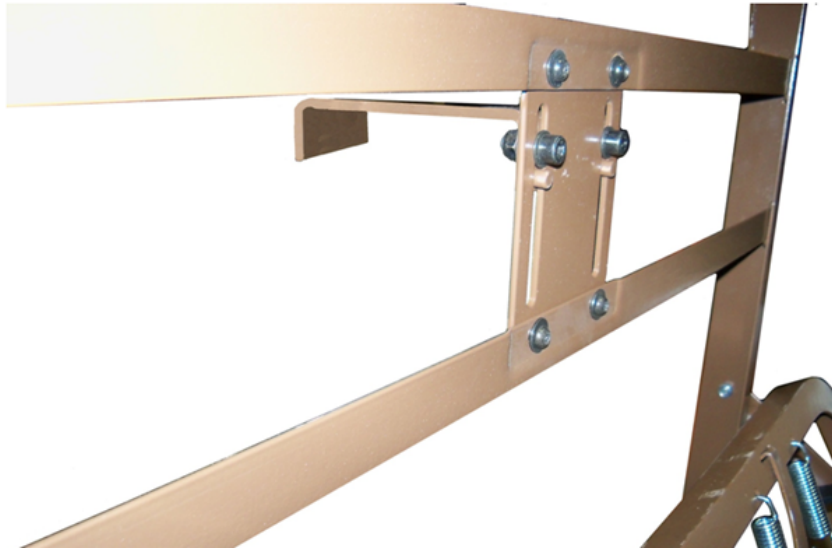

5. Mount the new style 15033MHLN-MGBA on the 15033MBM-DELTA mounting bracket.

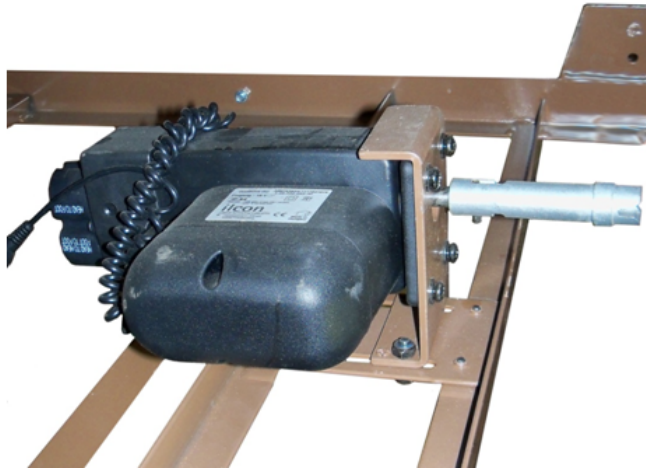

[Home](#)