


QUAD CANE ADJUSTMENT

1. Loosen the anti rattle knob on the Quad Cane.



2. Push in the locking button. Turn the handle slightly and raise the handle up to the desired height. Turn the handle back until the locking button locks into place. Tighten the anti rattle knob.

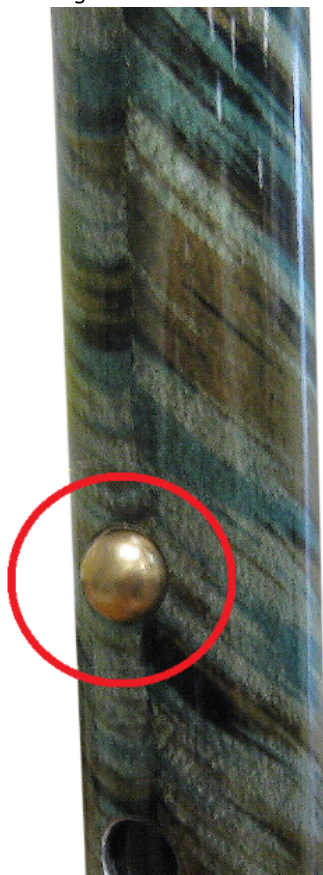



3. The short legs of



**This side
towards** 



body.

4. To change the Quad Cane from left side to a right side or a right side to a left side, loosen the anti rattle knob as in step 2. Push in the locking pin and turn the handle 180 degrees until the locking pin locks into place. Tighten the anti rattle knob.





[Home](#)