

## "L" BRACKET INSTALLATION


1. Verify that all the pieces are enclosed.

6 - Knobs


1 - Lower Cylinder Holder

1 - Upper Cylinder Holder

1 - L Bracket

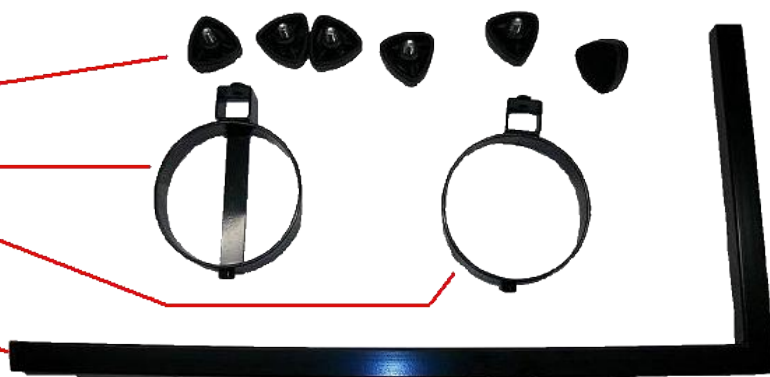


2. Remove the square plastic plug from the square tube at the rear of the seat

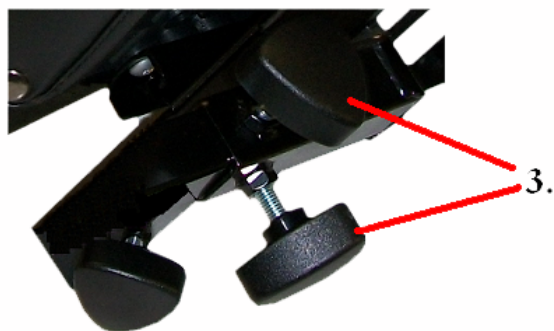

3. Install 2 of the knobs on the square tubing on the rear of the seat. Install only about 1/8".




4. Install the short end of the "L" bracket into the square tubing at

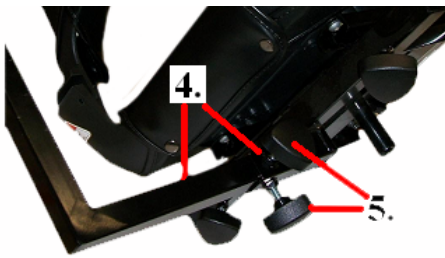

the back of the seat.

5. Tighten up both knobs.

6. Install the 4 remaining knobs on the upper and the lower cylinder holders. Install the knobs about 1/8".

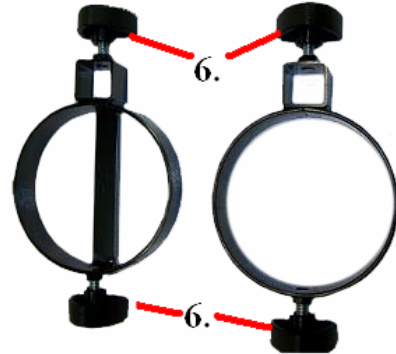

7. Install the lower cylinder holder on the "L" bracket. Slide the base all the way down. Tighten the bracket.

8. Install the upper cylinder holder on the "L" bracket. Position the bracket at a height that will accommodate the size O2 Cylinder that is being used. Tighten the bracket.



9. Place the O2 cylinder in the O2 holder and tighten the knobs on the holder.

[Home](#)