



SF8020 BASKET INSTALLATION

1. Verify all the parts are in the box.

- 1 - Basket
- 1 - L Bar
- 1 - Top Mounting Bar
- 1 - Velcro Strip
- 1 - Velcro Bracket
- 4 - Knobs

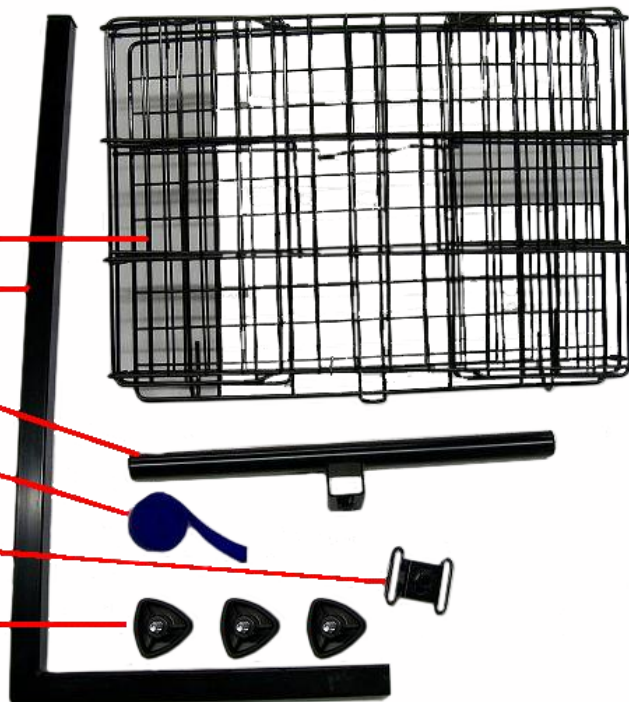


2. Remove the square plastic plug from the square tube at the rear of the seat

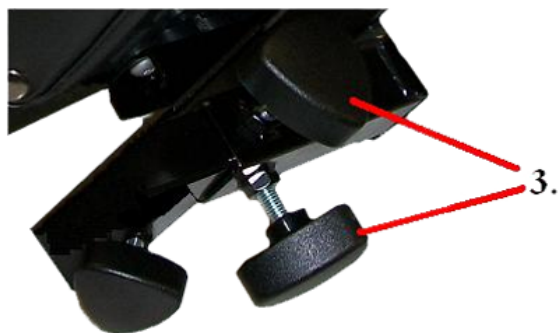

3. Install 2 of the knobs on the square tubing on the rear of the seat. Install only about 1/8".



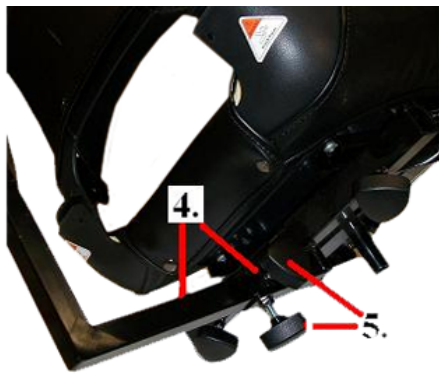


4. Install the short end of the "L" bracket into the square tubing at the back of the seat.

5. Tighten up both knobs.

6. Install one knob on the Velcro Bracket. Install about 1/8"


7. Install one Knob on the Top Mounting Bar. Install about 1/8"

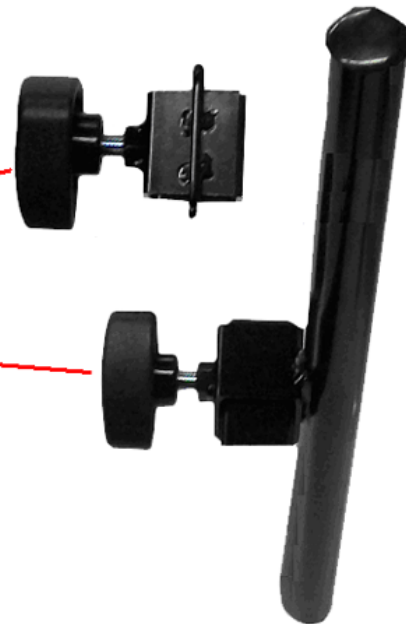


8. Slide the Velcro Bracket, with the knob towards the back of the seat, down the L Bracket.

9. Slide the Top Mounting Bar, with the knob towards the back of the seat, down the L Bracket. Slide the bracket just below the top of the L Bracket and tighten.



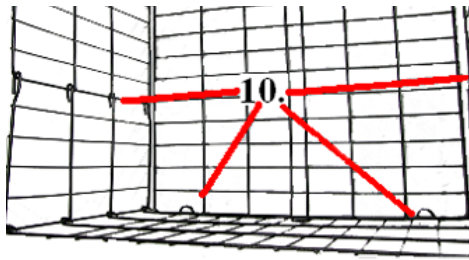

10. Unfold the basket.

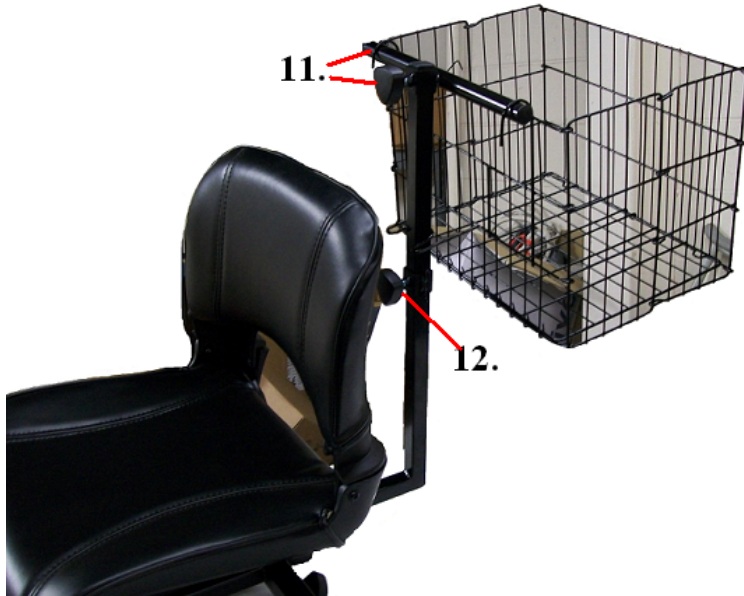

11. Put the hooks of the basket over the Top Mounting Bar.

12. Slide the Velcro Mounting Bracket so that it is just above the bottom of the basket.



13. Install the velcro strap from the Velcro Mounting Bracket to the Basket.

[Home](#)