


HOW TO INSTALL WENZELITE SPEED TABS


1. Release the anti-reverse dowel it is in the disengaged position.

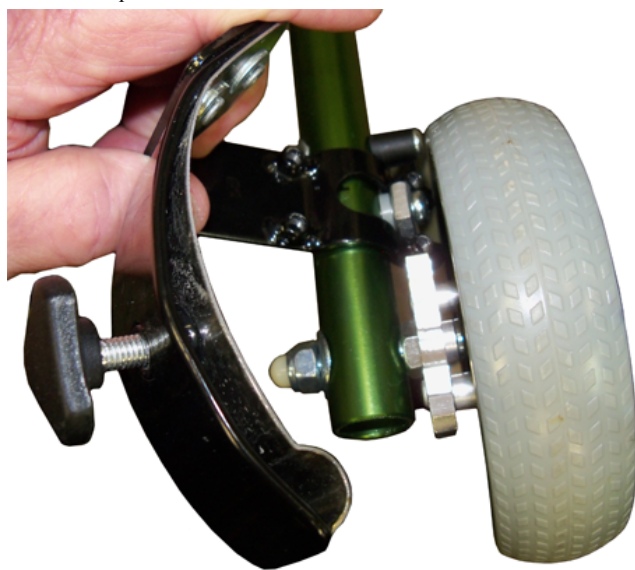

2. Slide the Dial-A-Speed Tab around the anti-reverse dowel.



3. Lock the anti-reverse dowel in the disengaged position.

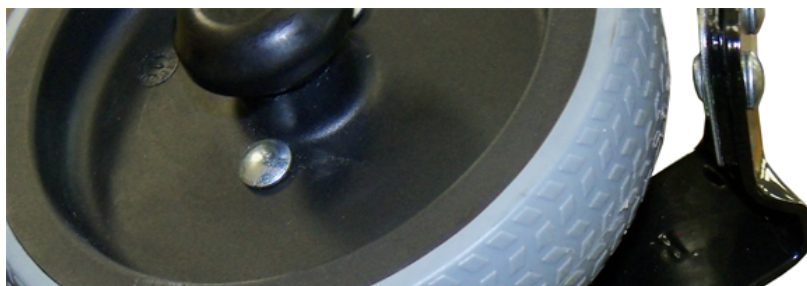



4. Place the rear bracket with the hole around the anti-reverse dowel nut.



5. Install four screws through the rear bracket to the main Dial-A-Speed Tab and hand tighten.





6. Pull the Dial-A-Speed Tab away from the wheel. Tighten all four screws equally keeping the bracket parallel to the wheel and the tab away from the wheel. Repeat steps 1 to 6 if a Dial-A-Speed tab will be used on both wheels.



7. The knob on the top of the Dial-A-Speed Tab will adjust the resistance on the rear wheel or wheels.

