


HOW TO INSTALL OLD STDS803 O2 HOLDER

1. Remove the antitipper or the tipping sleeve from the left side of the wheel chair.



2. Remove the nut and the bolt from the clamp on the O2 Holder.

3. Slide the tube of the O2 Holder over the tube of the wheelchair where the antitipper or the tipping sleeve was removed.

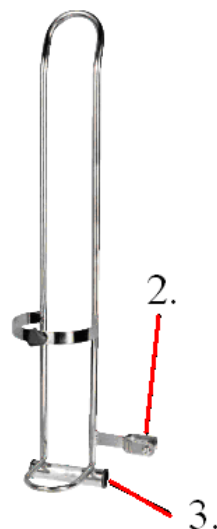

4. Install the nut and the bolt in the clamp on the O2 Holder.




5. Install the antitipper or the tipping sleeve

on the left side of the wheelchair.



6. Put the O₂ tank in the O₂ holder.

7. Tighten the retaining knob.

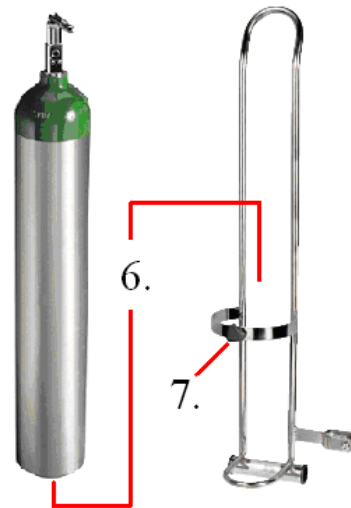

Home