

# TRID-2A47


## HOW TO INSTALL

Install a TRID-2A47 and SFELR on a Trident HD or Trident after serial # 2A1406.....


1. Remove the seat from the Trident




2. Align the TRID-2A47 bracket with the upper frame of the Trident.

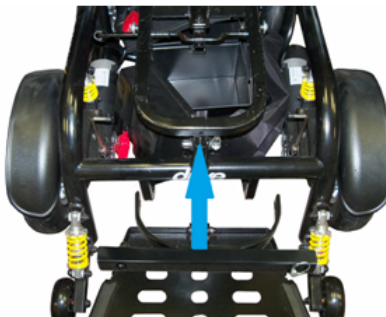

3. Install the two button head screws through the TRID-2A47 bracket and the upper frame. Thread the nut onto the button head screw and tighten.

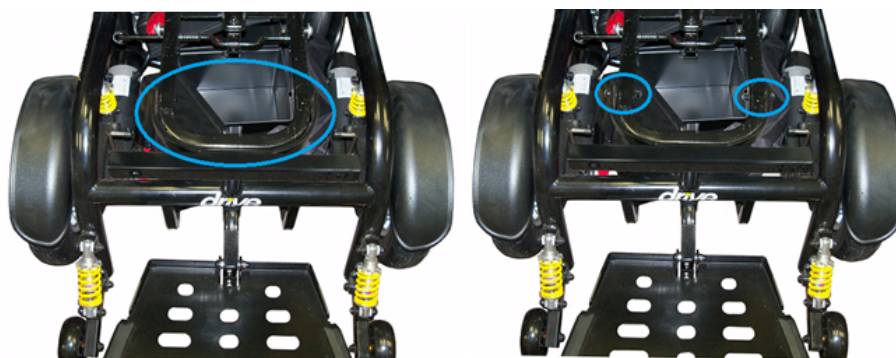

4. Slide one of the "T" brackets from the SFELR kit into the TRID-2A47 bracket with the welded nut facing up. Lock the "T" bracket in the TRID-2A47 with either the provided cap screw and nut or a grenade pin. The use of the grenade pin facilitates easy adjustments.

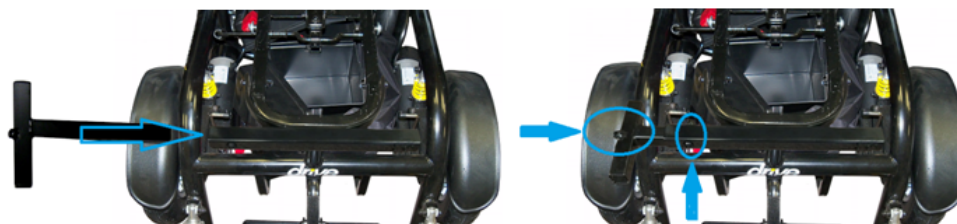





5. Repeat for the other side.



6. Slide the right ELR bracket into the right "T" bracket.

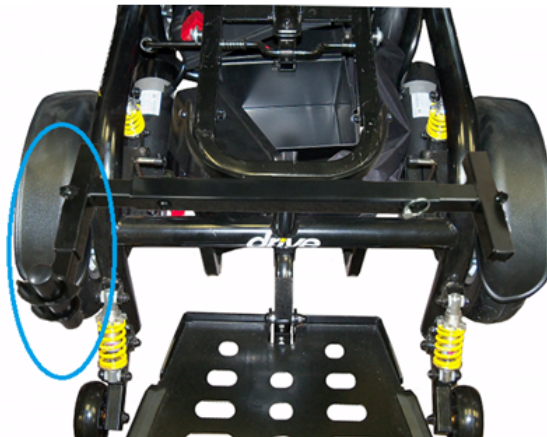

7. Slide one of the grenade pins through the "T" bracket and the ELR bracket. Install one of either the Allen head set screw or a locking knob to lock the ELR bracket in place. The use of the locking knob facilitates easy adjustments.

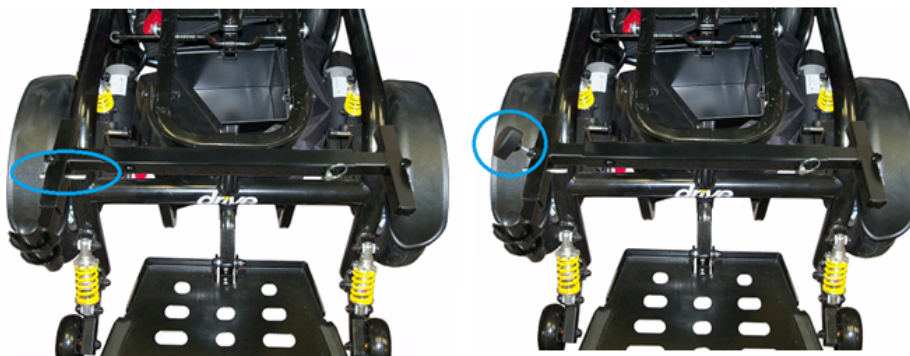

8. Repeat for the left side. Now the Trident is able to use a set of footrests or ELRs. Adjust the brackets as needed. Install the seat back onto the Trident.



---

Home