

ULTRA-LITE MOTOR INSTALLATION




NOTE: Before installing motor, plug hand pendant cable into motor as shown

For 15033 and 15235 plug in Hi/Low motor.




Connect the 9V battery (sold separately) to the battery clip on the motor. It is stored in the recess under the motor cover. With this battery installed, the motor can be operated in the event of a power failure for a limited number of times.

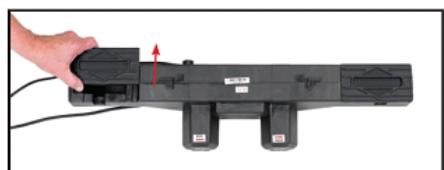

To install motor, remove locking caps on top of the motor.



Verify that the two white actuating blocks are in the fully retracted position. If they are not, plug the motor into an outlet. Using the hand control push the head and foot down buttons until the blocks are retracted.

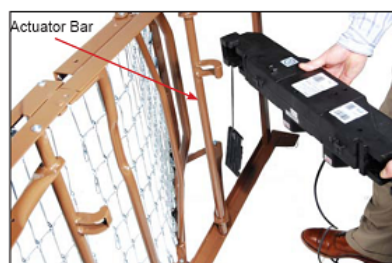

Align the opening on the head end of the motor with the flange on the head section of the bed and insert the flange into the motor compartment.



With the back of your hand, snap the motor flush to the actuator bar.




Align the opening on the foot end of the motor with the flange on the foot section of the bed and insert the flange into the motor compartment.



With the back of your hand, snap the motor flush to the actuator bar.

After the motor is installed, slide locking covers back on motor.

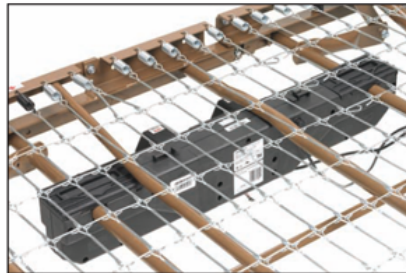

[Home](#)