


15904C - PPB1750

LONG TERM CARE BED

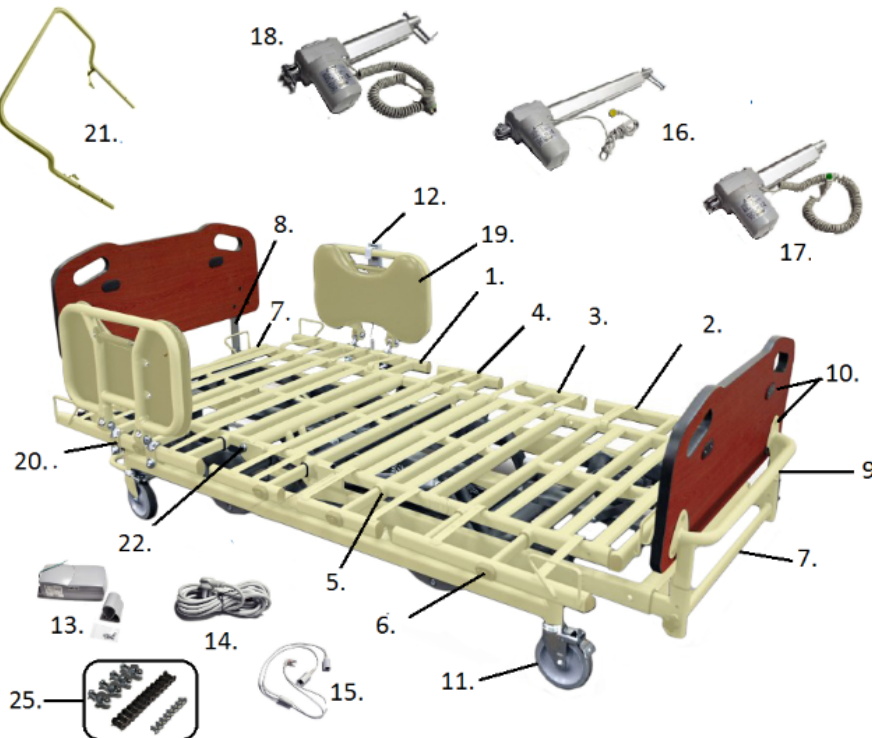

1. Head Deck- SP02-PPB1750-09D
Head Deck Width Ext Left- SP02-PPB1750-08E
Right- SP02-PPB1750-07E
2. Foot Deck- SP02-PPB1750-19D
Foot Deck Width Ext Left- SP02-PPB1750-12E
Right- SP02-PPB1750-11E
3. Thigh Deck- SP02-PPB1750-15D
Thigh Deck Width Ext- SP02-PPB2000-09E

4. Hip Deck Width Ext- SP02-PPB2000-10E
5. Width Extension Guides- SP02-PPB2000-SMB
6. Width Ext Release Assy- SP02-PPB2000-35E
7. Head/Foot Adj Mntg Brkt- SP02-PPB1750-31E
Length Ext. Brkt Bolt- SP01-KBCZ31C50
8. Head Board Mntg Brkts- SP02-PPB1750-1612E
9. Foot Board Mntg Brkt- SP02-PPB1750-1712E
10. End Board Bolts Male- SP02-UF16108051E
End Board Bolts Female- SP02-UN90108091E
Head / Foot Board Plugs- SP02-PPB2000-17-03
11. Locking Casters- SP02-UA18321520E
12. Hand Pendant - 8 Button- SP02-TH21060003
13. Control Box- SP02-TC11060005
Control Box Mounting Brackets- SP02-32905-042-2
14. Power Cords- SP02-PPB1750-3402E
15. Y-Cable- SP02-TYC1060001
Y-Cable Mntg Brkt- SP02-PPB2000-36E
16. Hi/Lo Actuator- SP02-TA71060006
17. Foot Actuator- SP02-TA71060004
18. Head Actuator- SP02-TA71060005
19. Side/Assist Rails- SP02-PPBASR *
Side/Assist Rail Pad- SP02-PPBASRP
20. Side/Assist Rail Mntg Brkt- SP02-PPBRSRB
Hand Wheel Knob- SP02-PPB1750-43E
Assist Rail Release Handle- SP02-PPB1750ARRH
Assist Rail Liner- SP02-1750-13-03E
21. Wall Guard- SP02-PPB1750-0211E
22. Head/Foot/Thigh Deck Hdw- SP02-PPB2000-DB
23. Foot Deck Pivot Bolt- SP02-PPB2000-23DB
Foot Deck Pivot Arms- SP02-PPB2000-27DB
24. Hardware Packs- 15904C-FHP

* Accessory not a part.

08/26/20