


15905C - PPB2000 - ML1000

LONG TERM CARE BED

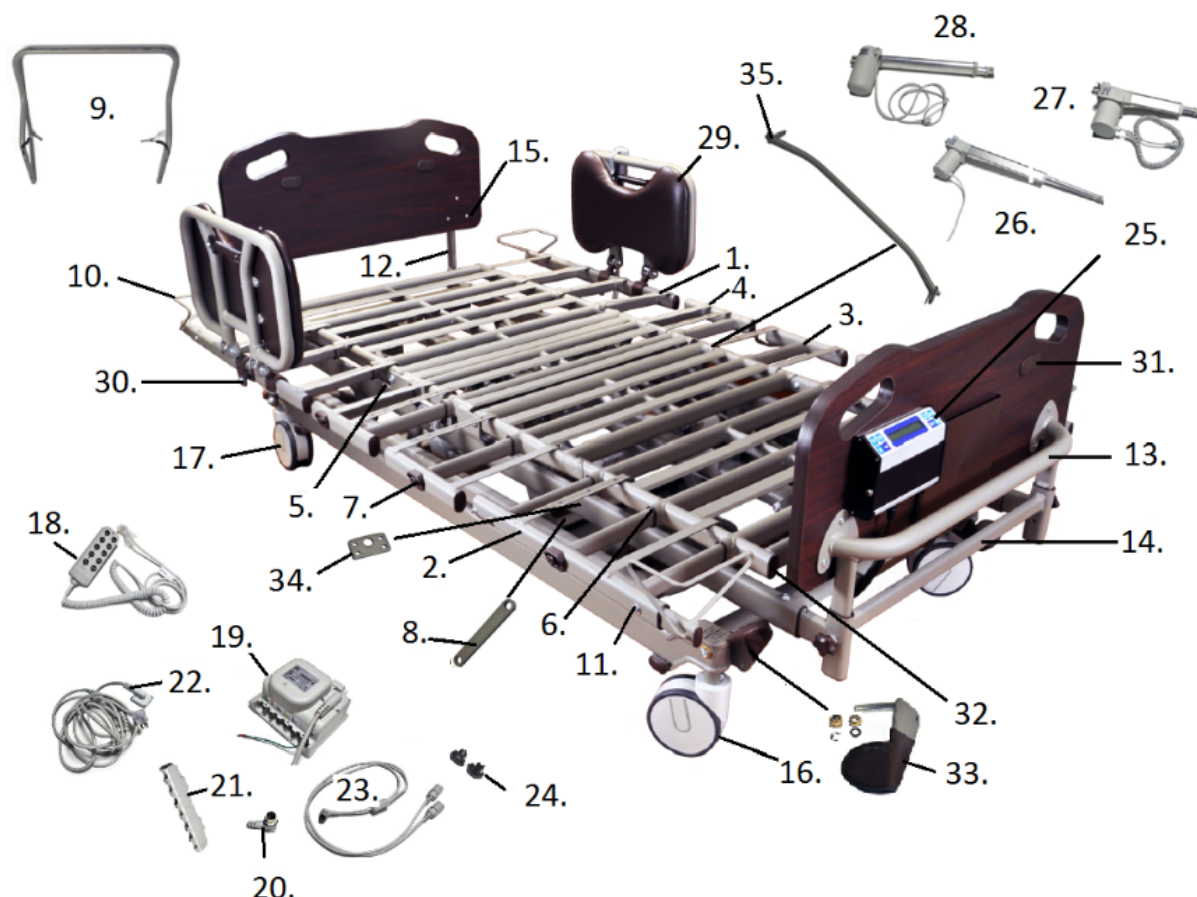

- | | |
|---|---|
| 1. Head Deck- SP02-PPB1750-09D
Head Deck Width Ext Left- SP02-PPB2000-08E
Right- SP02-PPB2000-07E | 19. Control Box- SP02-66468 |
| 2. Foot Deck- SP02-PPB1750-19D
Foot Deck Width Ext Left- SP02-PPB2000-12E
Right- SP02-PPB2000-11E | 20. Bypass Plug for Control Box- SP01-PCB900-47E |
| 3. Thigh Deck- SP02-PPB1750-15D
Thigh Deck Width Ext- SP02-PPB2000-09E | 21. Control Box Port Plug Cover- SP01-PCB900-CBC |
| 4. Hip Deck Width Ext- SP02-PPB2000-10E | 22. Power Cords- SP01-59094 |
| 5. Head/Foot/Thigh Deck Hdw- SP02-PPB2000-DB | 23. Y-Cable- SP01-CA51336 |
| 6. Width Ext Guides- SP02-PPB2000-SMB | 24. Y-Cable Mntg Brkt- SP02-PPB2000-36E |
| 7. Width Ext Release Assy- SP02-PPB2000-35E
Foot Deck Pivot Arms- SP02-PPB2000-27DB
Foot Deck Pivot Bolt- SP02-PPB2000-23DB | 25. Scale- SP02-PPBASC [1] |
| 9. Wall Guard- SP02-PPB2000-32 | 26. Hi/Lo Actuator- SP02-66261
Hi/Lo Adapter Cable- SP01-CA35825 |
| 10. Side Mattress Retainer
Head Left, Foot Right- SP02-PPB2000-45E
Head Right, Foot Left- SP02-PPB200-44E | 27. Foot Actuator- SP02-66251 [2] |
| 11. Mattress Retainer Bolts- SP01-KBCZ31C50 | 28. Head Actuator- SP02-66245 |
| 12. Head Board Mtg Brks- SP02-PPB2000-1612E | 29. Side/Assist Rails- SP02-PPBASR [1]
Side/Assist Rail Pad- SP02-PPBASRP |
| 13. Foot Board Mtg Brkt- SP02-PPB1750-1712E | 30. Side/Assist Rail Mtng Brkt- SP02-PPBRSRB
Assist Rail Release Handle- SP02-PPB1750ARRH
Assist Rail Liner- SP02-1750-13-03E
Hand Wheel Knobs- SP02-PPB1750-43E |
| | 31. Head/Foot Board Plugs- SP02-PPB2000-17-03 |
| | 32. Frame Tube End Inserts- SP02-PPB2000-42E |
| | 33. Brake Pedal Assembly Left- SP02-PPB2000-10018
Right- SP02-PPB2000-10019 |

- | | |
|--|--|
| <ul style="list-style-type: none"> 14. End Board Adj Length Mtg Brkt- SP02-PPB1750-31E
Bolt for Length Ext. Brkt- SP01-KBCZ31C50 15. Head/Foot Board Bolts Male- SP02-UF16108051E
Female- SP02-UN90108091E 16. Total Locking Caster- SP02-UA18701500E 17. Direct Locking Caster- SP02-UA18701501E 18. Hand Pendant- SP02-51495
Hand Pendant W/Magnetic Lockout- SP02-51518
Magnet for Hand Pendant- SP02-PPBA10PM | <ul style="list-style-type: none"> Plastic Brake Pedal Washer- SP02-PF43131900E Brake Pedal "E" Clip- SP02-UE31091001E Brake Pedal Sm Flange Washer- SP02-UB10122501E Brake Pedal Lg Flange Washer- SP02-UB10122500E Brake Pedal Extension Bar- SP02-ML1002-10012E 34. Cross Bar Mtg Plate- SP02-PPB2000-21B
Cross Bar Mtg Plate Bushing- SP02-PPB2000-22B 35. Cross Brake Locking Assy- SP02-PPB2000-24B
n/s Hardware Packs- SP02-ML1000-10030E
n/s Updated Software- SP01-SP55661
n/s Embedded Staff Controls- SP01-55658
Cable Embedded Controls- SP01-51335 |
|--|--|

[1] Accessory, not a part.

[2] When stock is gone the part is no longer available.

07/16/2020

[Diagrams](#)

[Home](#)