


WALLABY

WB 1200-2GFR & WB 1400-2GJB

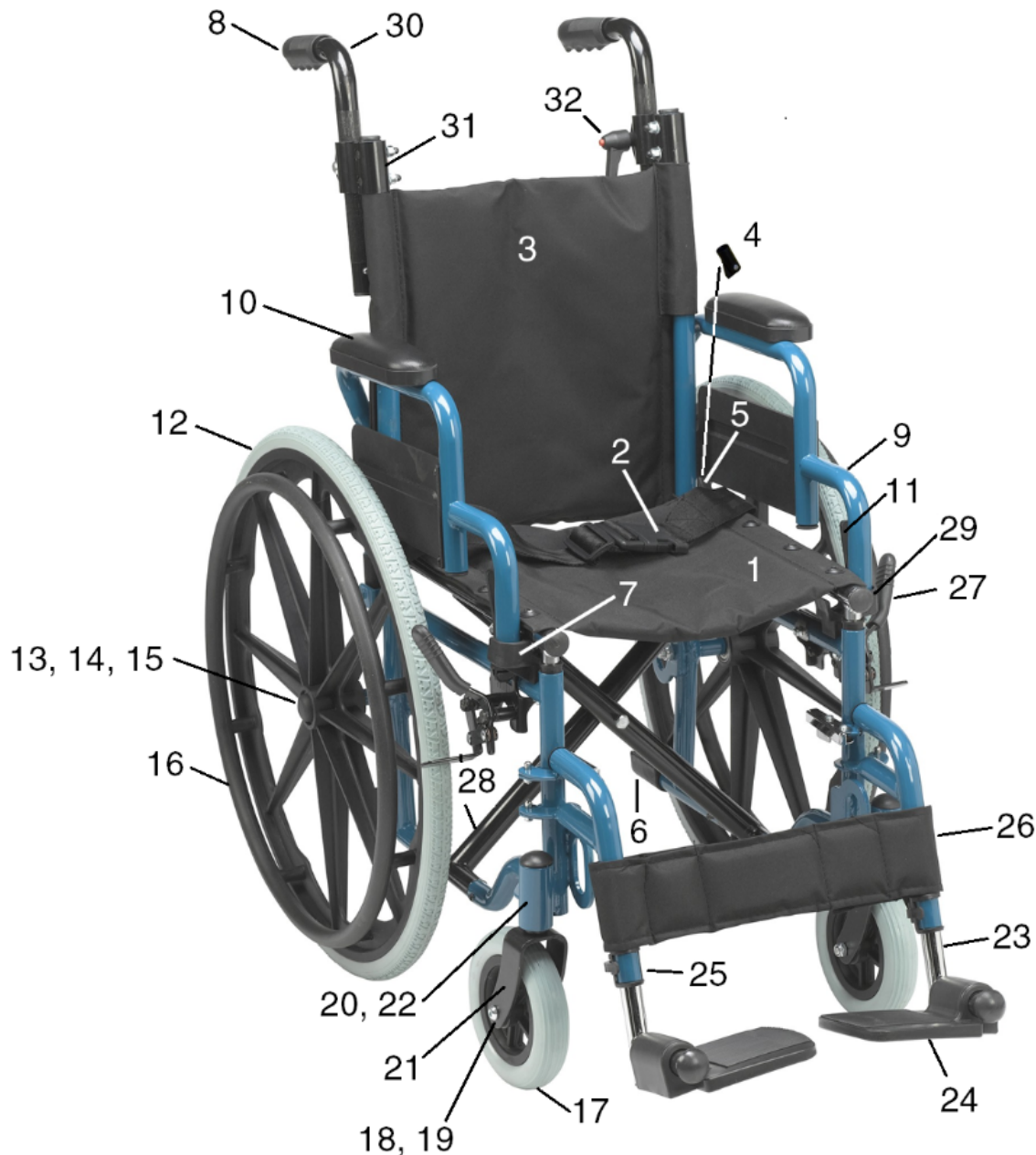

1. Seat Upholstery 12"- WB 80002G-12
Seat Upholstery 14"- WB 80002G-14
2. Seat Belt- STDS850-PB-JR
3. Back Upholstery 12"- WB 80012G-12
Back Upholstery 14"- WB 80012G-14
4. Arm Rest Pivot Bracket (pr)- **[1]**
5. Rear Seat Guide- WB 8029
6. Tipping Sleeve- STDS4020
Optional Anti-Tipper- STDS819
7. Seat Saddle/Arm Receiver Left- WB 8031
Seat Saddle/Arm Receiver Right- WB 8031-R

18. Caster Axle- WB 8015
19. Caster Bearing- WB 8016
20. Stem Bearing- STDS8M4601
21. Fork- WB 8018
22. Fork Stem Cap- WB 8019
23. Foot Rest (pr) Red- WB 8020
Foot Rest (pr) Blue- WB 8020-B
ELR Set- WB ELR
24. Foot Plate- WB 8022
25. Foot Rest Adjustment Lever- WB 8023
Substitute- STDS3J4502
Height Adj. Double Button- 700PTM

- | | |
|---|--|
| 8. Hand Grip (ea)- WB 8036 | Height Adj, Double Button- 7805111 |
| 9. Arm Rest Assy WB 1200-2GFR Rt- WB 8004 | 26. Calf Strap 12"- WB 8024 |
| Arm Rest Assy WB 1200-2GFR Lt- WB 8005 | Calf Strap 14"- WB 8024-14 |
| Arm Rest Assy WB 1400-2GJB Rt- WB 80052G-14 | 27. Brake Left- WB 8039 |
| Arm Rest Assy WB 1400-2GJB Lt- WB 80042G-14 | Brake Right- WB 8040 |
| 10. Arm Pad- WB 80082G | 28. Cross Frame 14"- WB 80352G-14 |
| 11. Arm Rest Release Lever (pr)- STDS5N2219 | 29. Slide Tube & Bushing- WB 8028 |
| 12. Rear 22" Wheel- WB 8009 | Slide Tube Retainer with Plug- WB 8027 |
| 13. Hub Cap- WB 8010 | Frame Plug (pr)- WB 8037 |
| 14. Rear Wheel Bearing- WB 8011 | 30. Push Handle & Hand Grip |
| 15. Rear Axle & Spacer- WB 8012 | WB 1200-2GFR (red)- WB 8032 |
| 16. Handrim & Hardware- WB 8013 | WB 1400-2GJB (black)- WB 8032-B |
| Handrim & Hardware- WB8041 | 31. Push Handle Attachment Hdwr- WB 8033 |
| 17. Caster- WB 8014 | 32. Push Handle Locking Lever- WB 8034 |

[1] The part is no longer available.

05/10/2021

[Diagrams](#)

[Home](#)