

# NEW VSI FLASH CODES

Flash Code	Meaning	Possible Reasons
1	Low Battery or Bad Battery Connection	Check connections to batteries. If connections are good then charge the batteries.
2	Left Motor	Check motor connections.
3	Left Motor	Motor Has a short circuit to the battery connection.
4	Right Motor	Check motor connections.
5	Right Motor	Motor Has a short circuit to the battery connection.
6	Not Used	
7	Joystick Fault	Make sure the joystick is in the center position before switching on.
8	Controller Fault	Make sure all connections are secured.
9	Brake Fault	Check brake connection and brake is not in neutral.
10	Excessive Voltage	This is usually caused by a poor battery connection.