

HOW TO USE THE ZD-08 PROGRAMMER

1. Insert the connector into the charging port on the joystick, and turn the joystick on.
The **LEFT** of the decimal point is the menu function (forward, reverse, etc.).
The **RIGHT** of the decimal point shows the value (increase or decrease a parameter).
The **UP** and **DOWN** buttons are used to scroll through the menu and to adjust the values.
The **SHIFT** button is used to toggle between the menu function and the parameter values.
The **CONFIRM** button is used to save the changes made to the parameters.
2. Scroll to the desired menu function referring to the chart below using the **UP** or **DOWN** buttons.
3. Using the **SHIFT** button, switch to the parameter side and adjust the value to the desired number.
4. To save the new value, press the **CONFIRM** key.
5. Switch the joystick off and remove the connector.
6. Test drive the chair to confirm that the parameters are saved and that the chair is safe and compatible to the client's needs.

