

## lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2006 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

foot stool  
with handrail

drive  
MEDICAL DESIGN & MANUFACTURING

[www.drivemedical.com](http://www.drivemedical.com)

# footstool




**WEIGHT CAPACITY 300 LBS.**

## parts list


CODE	DESCRIPTION	QTY
A	Screw M6 x 7/8" L	1
B	Screw M6 x 1 7/8" L	4
C	Screw M6 x 2 1/4" L	4
D	Nut M6	5
E	Washer	8

# assembly

1. Join the feet together using one piece 7/8" L screw and one nut.

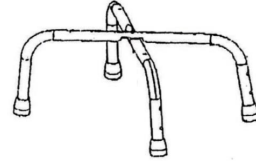

2. Then turn the seat upside down and place it flat on a table. Then turn the feet assembly upside down and place it on the seat (the holes in the feet assembly must be lined up with the holes in the seat). Use four pieces 1 7/8" L screws and four washers to join the feet assembly to the seat.

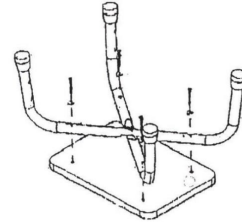

3. Join the feet together using one piece 7/8" L screw and one nut.

