



item 17100 shown

**WARNING:**

This Product has a maximum user weight limitation of 300 lbs.

Drive Medical assumes no responsibility for any damage or injury caused by improper installations, assembly, or use of this product.

If components are missing or damaged, please contact your dealer immediately. DO NOT use substitute parts!

**NOTE:**

Do not use the chair until all the assembly steps have been completed

**Warranty:**

- 2 year on the frame and 1 year on upholstery

**MAINTENANCE**

Check all hardware periodically for tightness

## two year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for two years from the date of the original consumer purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This Two Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2006 Medical Depot, Inc. All rights reserved.  
Drive is a trademark of Medical Depot, Inc.  
Port Washington N.Y. 11050 USA Made in China

hip-high chair

drive  
MEDICAL DESIGN & MANUFACTURING


[www.drivemedical.com](http://www.drivemedical.com)

## assembly / operating instructions


1. Remove the frame and seat assembly from the carton. Stand chair upright on all four legs.
2. Remove the (4) Phillips screws holding the seat to the bottom side of the frame, taking care to support the seat so it does not fall when the screws are removed.
3. Position the seat on the topside of the frame making sure the seat faces forward to the footrest support on the bottom front of the frame.
4. Reinsert the (4) Phillips screws through the bottom of the frame into the bottom of the seat. Be sure to use the long screws in the front of the seat and the short screws in the rear of the seat. Take care to start all (4) screws by hand before tightening them with the screwdriver.  
(Failure to start the screws by hand can cause permanent damage to the seat assembly)

5. Remove the (2) Phillips screws holding the armrest to the bottom side of the chair arm. Reposition the armrest to the topside of the chair arm and install the (2) screws by hand before tightening them with the screwdriver. Repeat the armrest installation on the other arm.
6. The Drive Hip-High Chair features an adjustable height footrest. To install, position the footrest with the rounded corners facing outward and the square corners facing under the chair. Position the wooden dowels on the footrest into the desired height setting until the footrest is flush with the chair. Install the (4) screws provided (2 per side) into the holes provided in the back of the chair rail. Tighten all 4 screws into the footrest.


## chair as shipped

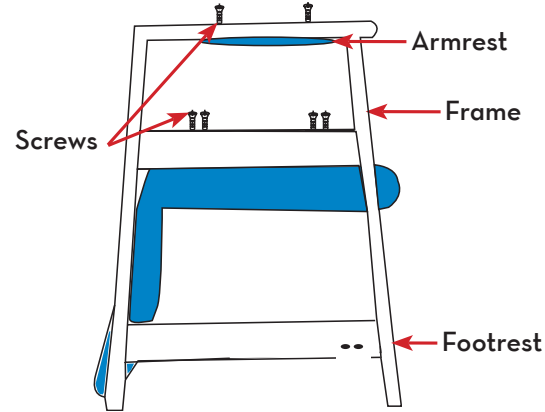

## seat & footrest assembled

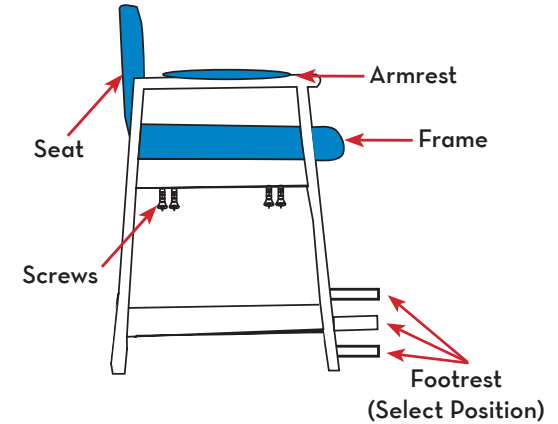

## armrest assembled

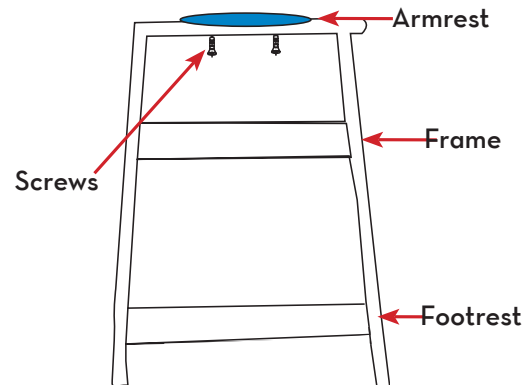

## footrest installation

