

Nebulizer Kit



Cleaning of nebulizer:

After each use, unscrew nebulizer into parts, wash in warm soapy water, and rinse well. Leave it in a clean area to air dry.

NOTE:

Do not towel dry nebulizer parts; this could cause contamination.

Cleaning accessories:

After each use rinse and dry mouthpiece. Never immerse tubing in water. Single patient use for one month.



driveTM

Kit de Nebulizador

Limpieza Del Nebulizador:

Después de cada uso, desatornille el nebulizador en partes, lave en agua tibia jabonosa y enjuague bien. Déjelo en un área limpia para secar al aire libre.

NOTA: No seque las partes del nebulizador con una toalla; esto puede causar la contaminación.

Limpiando Los Accesorios:

Después de cada uso, enjuague y seque el pedazo de boca. Nunca sumerge la tubería en agua. Uso de un mes para un paciente.

Kit de Nébuliseur

Nettoyage du Nébuliseur:

Après chaque utilisation, dévisser les parties du nébuliseur, nettoyer dans de l'eau chaude et savonneuse et bien rincer. Laisser sécher à l'air libre dans un endroit propre.

REMARQUE: Ne pas sécher les pièces du nébuliseur avec une serviette; cela pourrait causer une contamination.

Nettoyage des Accessoires:

Après chaque utilisation, rincer et sécher l'embout buccal. Ne jamais immerger le tuyau dans l'eau. À utiliser pour traitement individuel d'un mois.