driveTM

www.drivemedical.com

Rev.2.12.02.15

exercise peddler
with digital display

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item # RTL10275


EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

www.drivemedical.com

driveTM

assembly instructions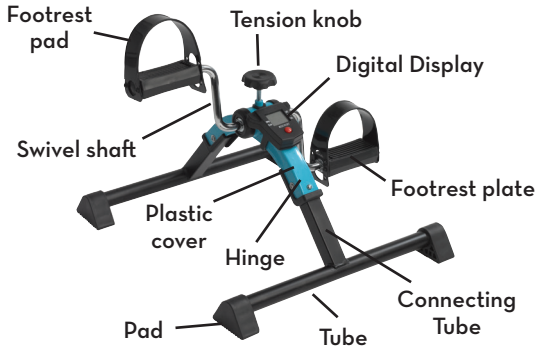

1. Remove contents from box.
2. Pull legs in opposite direction until locking button “snaps” into locking receiver (See figure A).

3. Attach digital display to the peddler by sliding the loop over the tension knob receptacle located on the top of the peddler. Push down gently on the display so the clasp on the back of the display attaches to the peddler frame. (See figure B)


4. Insert tension knob on top of peddler and turn clockwise to tighten. The tension knob will increase/decrease the amount of tension on the peddler. Tighten/ loosen to a level that feels comfortable.

5. The pedal straps have 2 sets of holes to allow you to adjust the length of the strap around your foot or shoe. (See figure C)


folding

To fold, depress push pin and fold leg inward. Repeat for second leg. (see figure D)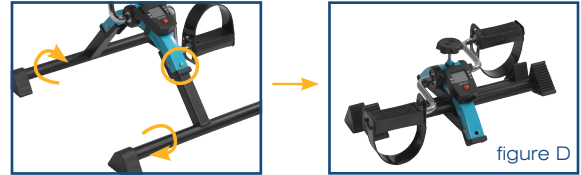

automatic operation

1. **MODE (Red Button):** This control allows you to select a particular function. To start, press and hold the Red Button for 4 seconds. The LED will display the different functions. (After a few minutes without any movement, the unit will automatically shut off).
2. To select a function, wait until the 4 second set up is complete. Push the Red Button and the arrow will move through the functions. When the desired function is selected, simply use the peddler and the unit will calculate the exercise.
3. **TIME:** Determines the amount of time the peddler has been operating.
4. **COUNTER (CNT):** Determines the amount of steps while operating the peddler.
5. **CALORIE (CAL):** Determines the amount of calories burned while using the peddler.
6. **REPS PER MINUTE (R.P.M.)** Automatically calculates the repetitions per minute.
7. **SCAN:** Will automatically review the results of all functions.

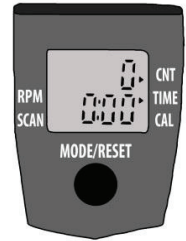battery replacement

If the display does not come on or reads incorrectly, the battery requires replacement.

To replace the battery, use a small screw driver and remove the display panel. Turn the display panel over and remove the watch-style battery. Insert a new battery and replace the display panel.

WARNING—DEVICE MAY BECOME HOT WITH EXTENDED USE.

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lifetime limited warranty

Your Drive branded product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2015 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

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A large, vertical version of the Drive logo, oriented sideways. The word "drive" is written in white, lowercase letters against a dark blue background. A small yellow and orange square is positioned above the letter 'i'. A trademark symbol (TM) is located at the top right of the word.
©2015 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2015 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.