

## lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2006 Medical Depot, Inc. All rights reserved.  
Drive is a trademark of Medical Depot, Inc.  
Port Washington N.Y. 11050 USA Made in China

clever-lite rollator

drive  
WWW.DRIVEMEDICAL.COM

[www.drivemedical.com](http://www.drivemedical.com)

## assembly instructions

Do not attempt to adjust or operate walker without carefully reading all instructions in this manual.

1. Remove all contents from carton.
2. Open walker and drop seat to down position.

## safety precautions

1. Please make sure the walker is fully opened, seat is down and secured in the down position.
2. Test walker for stability before applying full weight to hand grips.
3. Test the loop locks before operating the walker. When wheels are locked, (see illustrations for locking and releasing locks), lock bar on rear wheels should prevent wheels from turning. If the wheels turn while brakes are locked, please take unit back to dealer for adjustment. When locks are released, the wheels should turn freely.
4. Always lock loop locks before sitting and do not use seat when unit is on an incline or uneven ground.
5. Replace tires when significantly worn or damaged.
6. Do not have anyone push you while you are seated on the walker.
7. To fold walker, lift seat to upright position. Deploy one trigger release and swing brace to closed position. Repeat on the other side.

**NOTE:** This is a walking aid only. It is not to be used as a transportation device.

**WEIGHT CAPACITY: 300 LBS.**

## loop lock operation

Always engage and disengage wheel locks on both sides simultaneously.

1. To Slow Down or Come to a Temporary Stop
  - Pull up slowly on the loop locks with four fingers until unit slows down or comes to a temporary stop.
2. To Temporarily Lock Wheels in Position
  - Press down on both loop locks with palms until you hear a “click” that will tell you wheels are locked.
3. To Release Wheel Locks
  - Push up on both loop locks with palms of hand to original position.

