

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

euro style crutch

drive
MEDICAL DESIGN & MANUFACTURING

www.drivemedical.com

adjusting leg height

1. Depress the push button and loosen anti rattle collar to desired height (Figure 1).
2. Before use, adjust height of leg to your specific needs. When desired height is reached, lock leg into place by aligning button into proper hole, (Figure 2).
3. To be sure leg is locked into place by pushing down on or by pulling leg.

cleaning instructions

1. Use a non-abrasive mild detergent or a household cleaner with warm water.
2. Rinse with warm water and wipe dry.
3. **DO NOT USE AN ABRASIVE CLEANER OR CLOTH** with this product as damage may ensue.

FIGURE 1

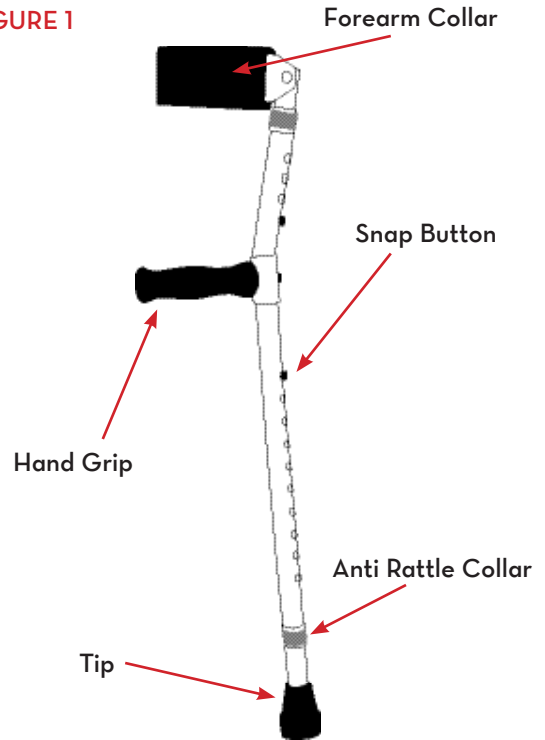


FIGURE 2

