drive™

www.drivemedical.com

Rev.5.02.05.15

toilet seat riser
with removable arms,
for standard toilets

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installation instructions

Weight Capacity: 300 lbs (136 Kg)

1. Remove product from carton and READ ALL INSTRUCTIONS CAREFULLY before attempting to install product.
2. If installation of arms is desired:
 - a. Place one arm at a time into the arm holes on the raised toilet seat (the brass buttons on the arm should be facing out, away from the toilet seat.)
 - b. Depress the brass button in the front of the arm while sliding the arm into position. You will hear a “click” when the button is positioned properly. (You will see the brass button pop through the position hole.)
 - c. Repeat process for back of arm.
 - d. Repeat process for the other arm.
 - e. To remove the arm, depress the brass button while pulling up on the arm. Repeat for the back of the arm.
3. Remove existing toilet seat and lid from toilet.
4. Position toilet seat riser correctly on rim of toilet bowl and center as necessary.
5. Attach original toilet seat to toilet bowl using the 6" bolts, with washers and butterfly nuts provided, so it is positioned over toilet seat riser.
6. Check for secure fit routinely before using product. User's weight should be centered over toilet seat, not too forward or too far to one side, as unit may tip.
7. If you have a style of toilet bowl that this product will not fit properly, it is recommended that you discontinue use.

cleaning instructions

1. Use a non-abrasive detergent or household cleaner with warm water.
2. Rinse with warm water and wipe dry.
3. DO NOT USE AN ABRASIVE CLEANER OR CLOTH with this product, as damage may ensue.

caution

1. Always be sure that the seat is correctly and securely in place before using.
2. Physically challenged users should be supervised or assisted whenever using this product.
3. This product is designed for use with the most common types of toilet bowls. It will not fit all toilet bowl types; be sure that the unit is correctly installed, fits properly and is stable before use.
4. The arms are for the purpose of providing assistance when getting on and off the seat. DO NOT try to use the arms to support full body weight during transfer, as this may result in failure of the unit.

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lifetime limited warranty

Your Drive branded product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2015 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

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www.drivemedical.com

Rev.5.02.05.15

toilet seat riser
with removable arms,
for elongated toilets

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installation instructions

Weight Capacity: 300 lbs (136 Kg)

1. Remove product from carton and READ ALL INSTRUCTIONS CAREFULLY before attempting to install product.
2. If installation of arms is desired:
 - a. Place one arm at a time into the arm holes on the raised toilet seat (the brass buttons on the arm should be facing out, away from the toilet seat.)
 - b. Depress the brass button in the front of the arm while sliding the arm into position. You will hear a “click” when the button is positioned properly. (You will see the brass button pop through the position hole.)
 - c. Repeat process for back of arm.
 - d. Repeat process for the other arm.
 - e. To remove the arm, depress the brass button while pulling up on the arm. Repeat for the back of the arm.
3. Remove existing toilet seat and lid from toilet.
4. Position toilet seat riser correctly on rim of toilet bowl and center as necessary.
5. Attach original toilet seat to toilet bowl using the 6” bolts, with washers and butterfly nuts provided, so it is positioned over toilet seat riser.
6. Check for secure fit routinely before using product. User’s weight should be centered over toilet seat, not too forward or too far to one side, as unit may tip.
7. If you have a style of toilet bowl that this product will not fit properly, it is recommended that you discontinue use.

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© 2015 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2015 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.