

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

med-aire 5" pump and
mattress overlay system



EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

operating instructions

1. Place the pump on a flat surface or suspend pump on end of bed with attached hook.
2. Place the mattress on top of bed mattress. Note foot end.
3. Connect air hoses to pump and mattress.
4. Check and ensure the air hoses are not kinked or tucked under mattress.
5. Turn pressure to firm. Plug the power cord into outlet. Switch on the pump. The green switch will light when unit is operating.
6. Adjust pressure for patient comfort.

DANGER

DO NOT use in the presence of flammable anesthetic. It may cause possible explosive hazard.

CAUTION

Risk of electric shock. DO NOT remove cover (or back). Refer to your qualified service personal for assistance.



REF 14025N Shown

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

drive
MEDICAL DESIGN & MANUFACTURING

© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China