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checker neb,  
nebulizer

drive  
MEDICAL DESIGN & MANUFACTURING

[www.drivemedical.com](http://www.drivemedical.com)

## important safeguards:

### Indications For Use:

Aerosol delivery to manage the symptoms of patients with Asthma, Chronic Lung Disease and Cystic Fibrosis.

### CAUTION:

Federal law restricts this device for sale or use on the order of a physician.

To reduce risk of electrocution:

- Always unplug unit immediately after use.
- Do not use while bathing.
- Do not place or store the unit where it can fall or be pulled into a tub or sink.
- Do not place or drop into water or other liquid.
- Do not reach for a product that has fallen into water. Unplug immediately.

### WARNING:

To reduce risk of electrocution, fire or injury:

- Electrical shock hazard. Do not remove the cabinet or open the cover.
- Disconnect the power cord from the electrical outlet before cleaning or servicing.
- Do not place this equipment near hot, sparking or burning objects.
- Do not use oil or grease on or near this device.
- Turn the unit off when it is not in use.
- Keep the cord away from heated or hot surfaces.
- Never drop or insert any object into any opening.
- Never block the air opening of the product or place it on a soft surface, such as a bed or couch where the opening may be blocked.
- Avoid the operation in wet or damp locations.
- Unplug the unit before filling the nebulizer.

## specifications:

Electrical Requirements:	115V AC/60HZ
Power Consumption:	75W
Dimensions:	4.3”(H) x 7.7”(W) x 11.8”(D).
Weight:	3.7 lbs.
Sound Pressure Level:	<53dBA
Maximum Pressure:	30 PSI
Operating Pressure:	16 PSI
Liter Flow Range:	<12 LPM
Particle Size:	Less than 5 microns

**NOTE:** Conditions may vary based on altitude, barometric pressure and temperature.

## operating instructions:

1. Place compressor on a stable, sturdy and flat surface such that the unit can be easily reached when you are seated.
2. Pull out the power cord from the base of the unit and make sure that the unit is in the “OFF” (O) position by pressing on the left side of the switch.
3. Plug the power cord into a 115 VAC/60 Hz wall outlet.
4. Connect one end of the nebulizer tubing into the air outlet connector.
5. Add the prescribed medication through the opening on cap using an eyedropper or pre-measured dose container.
6. Assemble mouthpiece and T-piece and insert into the top of the nebulizer cap. If an aerosol mask is used, connect the bottom of the mask to the top of the nebulizer cap.
7. Connect the tubing to nebulizer air inlet connector.

8. Turn the power on to start the unit.

9. Implement the treatment by placing the mouthpiece between the teeth. Inhale and exhale through the mouthpiece.

10. If aerosol mask is used, place mask over the mouth and nose.

#### 1.4 Description

- |    |                      |     |                     |
|----|----------------------|-----|---------------------|
| 1. | Air Outlet Connector | 7.  | Cap                 |
| 2. | Filter Cap           | 8.  | Baffle              |
| 3. | Power Switch         | 9.  | Air Inlet Connector |
| 4. | Power Cord           | 10. | Nebulizer Chamber   |
| 5. | Nebulizer T-Piece    | 11. | Tubing              |
| 6. | Mouthpiece           |     |                     |

## cleaning:

1. Turn the power off and unplug from the wall outlet.
2. Remove the tubing from the air inlet.
3. Disassemble mouthpiece, T-piece, nebulizer cap, chamber and baffle, and wash in hot water with dish washing soap. Rinse thoroughly.
4. Mix one part of white vinegar with three parts water in a clean container. Submerge mouthpiece, T-piece, nebulizer cap, chamber and baffle cap for one half hour.
5. Remove the components from the solution and let dry.

## maintenance:

1. The filter of air compressor should be replaced when it turns grey. Additional filters can be purchased from the dealer or pharmacy that you purchased the device from.
2. A certified technician should make repairs only.

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Item # 18040-Y

## 5 year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for five years of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This 5 year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2006 Medical Depot, Inc. All rights reserved.  
Drive is a trademark of Medical Depot, Inc.  
Port Washington N.Y. 11050 USA Made in China

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©2006 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.