

Item # CE 1000 BK
Shown



WEIGHT CAPACITY 400 LBS

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

drive
MEDICAL DESIGN & MANUFACTURING

© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

adult anterior safety roller

for use with CE 1000 B,
CE 1000 BK, CE 1000 NR,
and CE 1000 NBL



EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

Wenzel *ite* *Re/hab*

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

set up instructions

1. Remove Safety Roller from carton.
2. Unfold Safety Roller by lifting ring (Fig 2) from left upright and open frame.
3. Lift handlebar and position over left upright. Make sure spring button (A) snaps into hole on the handlebar underside (Fig 1).

height adjustments

Recommended handlebar height is at the wrist with arms fully extended. With your hand placed on the handgrip, your elbow should be flexed 20° - 30°.

1. Open both height adjustment knobs (B).
2. Push in spring button (C), one side at a time and raise or lower uprights to desired height.
3. Tighten knobs.

folding instructions

1. Press spring button on left side of handlebar and lift handlebar.
2. Drop handlebar into the center of Safety Roller.
3. Swing left upright outward and squeeze into Z position until fully folded.
4. Secure uprights by putting the ring over the left upright and under the spring button.

accessories

Adjustable forearm platforms, variable resistance tabs on rear wheels, swivel wheel locking brackets, ankle prompts, trunk support, extended uprights for additional height, IV pole and oxygen tank holder.

Call an authorized Drive dealer for detailed information about our accessories.

figure 1


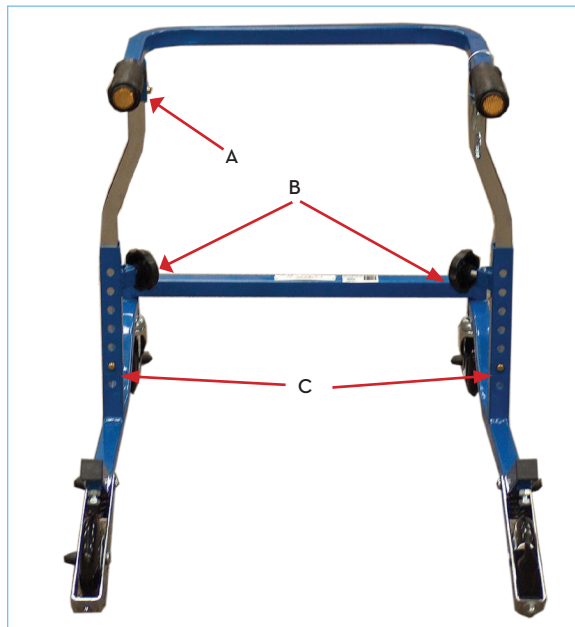


figure 2




operating safety roller


Make certain that the person for whom the walker is being fitted is standing as naturally upright on a firm, flat surface. Also, they should be wearing the same style shoes that they will be wearing when using the walker, during activities of daily living. In most cases, the height of the handles should be set to allow a 30° bend at the elbow when taking hold of the hand grips.

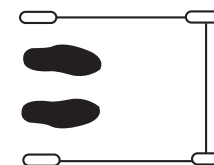


1. Stand with feet together inside lower frame. Walker will not move if weight is placed on handles.

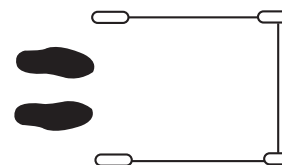

2. To move forward release enough weight on handles to allow walker to be rolled forward.

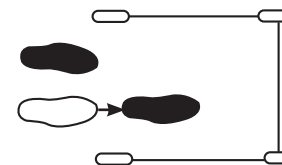

3. Reapply weight on handles to lock the wheels and step forward with one foot into lower frame.

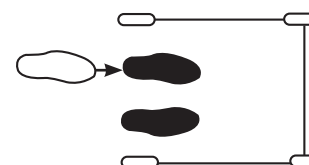

4. Place other foot into frame to complete one step. Repeat from number one to continue moving.