driveTM

www.drivemedical.com

Rev.3.07.28.14

toilet safety frame

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item # RTL12000

driveTM

www.drivemedical.com

installation instructions

WEIGHT CAPACITY: 300 LBS (136 KG)

1. Detach seat by first removing toilet seat bolts.
2. Place the rail mounting flange on the toilet and align the holes. Ensure tube on bracket is closest to the tank. (Fig. 1)
3. Place toilet seat and align holes. (Fig. 1)
4. Insert bolts through toilet seat and bracket using original nuts and secure.
5. Assemble safety frame by inserting short tube into arm tube as shown in Fig. 2.
6. Slide assist rails into bracket and secure them with built-in snap buttons. (Fig. 2)
7. Set rail to desired width and height using the built-in snap buttons. Make sure all built-in snap buttons are secured before use of product.

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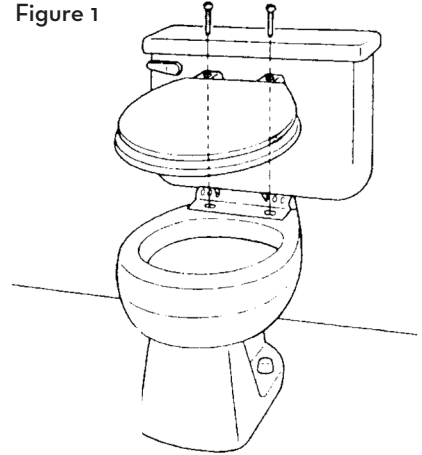
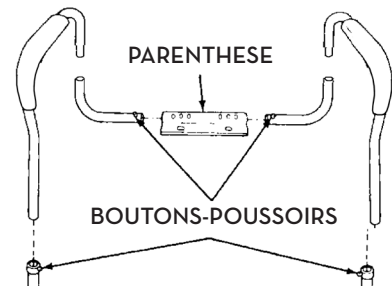
Figure 2

lifetime limited warranty

Your Drive branded product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2014 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington, N.Y. 11050 USA Made in China

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© 2014 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2014 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.