driveTM

www.drivemedical.com

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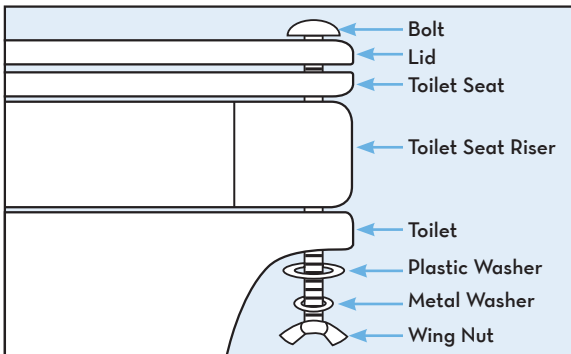
installation instructions

Weight Capacity: 250 lbs (113 Kg)

1. Remove existing toilet seat and lid from toilet.
2. Position raised toilet seat over toilet bowl with black skid strips facing downward.
3. Position original lid and cover over raised toilet seat. Secure in place by threading the the long bolt provided through the top of the seat and lid and securing in place using the plastic washer, metal washer, and wing nut (See below).

cleaning instructions

1. Use a non-abrasive detergent or household cleaner with warm water.
2. Rinse with warm water and wipe dry.
3. **DO NOT USE AN ABRASIVE CLEANER OR CLOTH** with this product, as damage may occur.

CAUTION

1. Always be sure that the seat is correctly and securely locked in place before using.
2. Be sure that the bolts are tight.
3. This product is designed for use with most common types of toilets and toilet seats. It will not fit all toilet or toilet seat types; be sure that the unit is correctly installed, fits properly and is stable before use.

lifetime limited warranty

Your Drive branded product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2014 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

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© 2014 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2014 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.