

one year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for one year from the date of the original consumer purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This One Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



*Wenzel*ite*Re/hab*

© 2006 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

headrest

item # CRS 8000

abductor

item # CRS 8025

for use with seat2go

(item # CRS 2000 & CRS 3000)

drive
MEDICAL DESIGN & MANUFACTURING

*Wenzel*ite*Re/hab*

www.drivemedical.com

CRS 8000



2. Insert headrest bracket into slit of fabric.
3. Adjust to desired height and tighten knob to secure.

NOTE: do not use black spacer when installing headrest.




installation

1. Remove knob, black spacer (A) and washer from back of Seat2Go.



CRS 8025

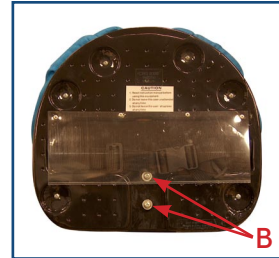

installation

1. Remove two screws (B) from underside of seat.
2. Insert abductor into groove on underside of seat.
3. Adjust abductor to desired depth, insert screws and tighten to secure.