



770-500 AQUASENSE ADJUSTABLE BATH BENCH

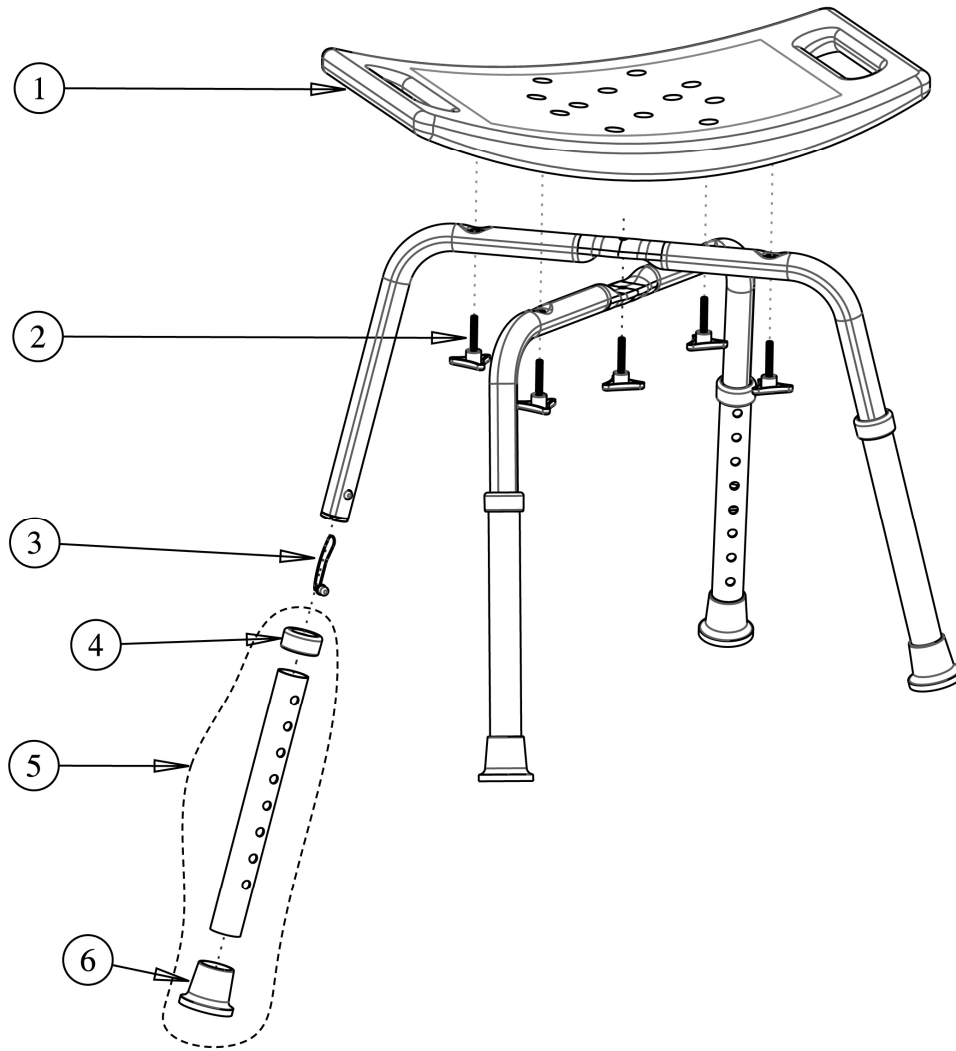


Item #	AMG Part #	Item Description	Qty Per Unit	Sold by	UOM
1.	A11-401	Seat, Blow-Molded Plastic, White (505x305x35mm)	1	1	EA
2.	B11-822	Hardware Set, Aqua-Blue (5 Triangular Knobs M6x32mm)	1	1	ST
3.	B11-811	Push Button dia:8mm & Spring - 4ea / set	1	1	ST
4.	A11-212	Anti-Rattle for dia:28.6mm, Aqua-Blue – 4 ea / set	1	1	ST
5.	B11-012	Extension Leg, Aqua-Blue (Aluminum Tube dia:28.6x250mm, Anti-Rattle & Rubber Tip) – 4 ea / set	1	1	ST
6.	A11-482	Rubber Tip for dia:28.6mm, Aqua-Blue – 4 ea / set	1	1	ST
Not Shown	B11-851	Hardware Set for Bath Bench (5 Cross-Cap Screws M6x32mm, 5 Curved Washers & 1 Locknut) -- Accessory (screw set to replace triangular knobs)	1	1	ST

770-501 ADJUSTABLE BATH BENCH

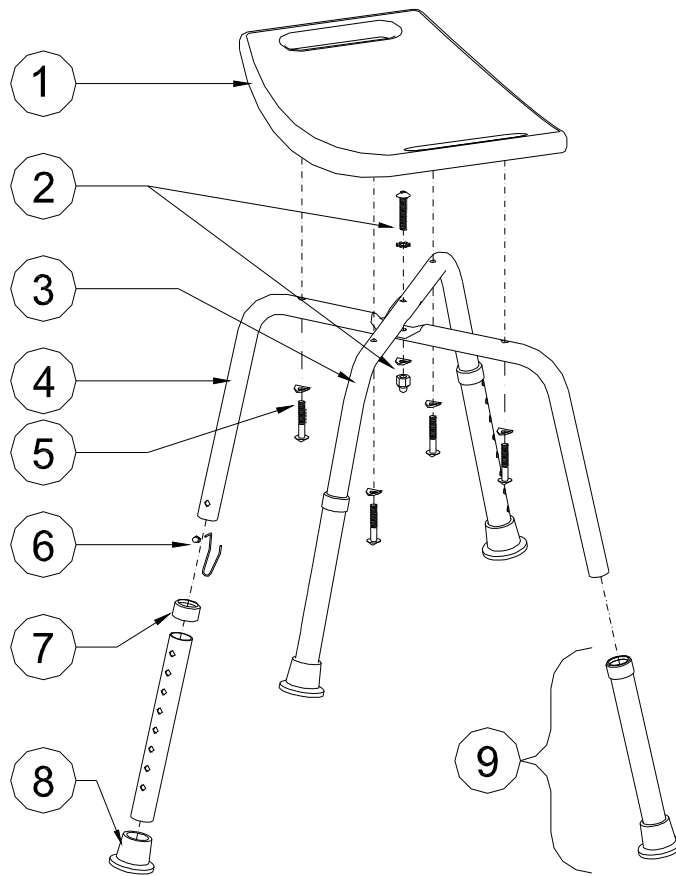


Item #	AMG Part #	Item Description	Qty Per Unit	Sold by	UOM
1.	A03-452	Seat / Backrest, Blow-Molded Plastic, White (505x305x35mm)	1	1	EA
2.	B03-815	Hardware for Legs (Bolt M6x37mm, Tooth-Lock Washer, S.Steel Curved Washer & Nut)	1	1	ST
3.	B03-011	Top Leg (Aluminum Tube ϕ 25.4mm & 2 Push-Buttons with 2 Springs)	1	1	EA
4.	B03-012	Bottom Leg (Aluminum Tube ϕ 25.4mm & 2 Push-Buttons with 2 Springs)	1	1	EA
5.	B03-816	Hardware for 770-500 Seat / 770-510 Back (Screw 1/4"x38mm & S.Steel Curved Washer)	4	1	ST
6.	B03-811	Push Button & Spring	4	1	ST
7.	A03-210	Anti-Rattle for Extension Leg, Grey Plastic	4	1	EA
8.	A03-432	Rubber Tip, Grey	4	1	EA
9.	B03-010	Leg Extension Assy (Aluminum Tube ϕ 28.6, Anti-Rattle & Rubber Tip)	4	1	EA
Not shown	B03-821	Hardware Set for 770-500 (Bolt M6x37, 4 Screws 1/4"x38mm, Tooth Lock Washer, 5 S.Steel Curved Washers & Nut)	1	1	ST

770-506 AQUASENSE ADJUSTABLE BATH BENCH - BLUE

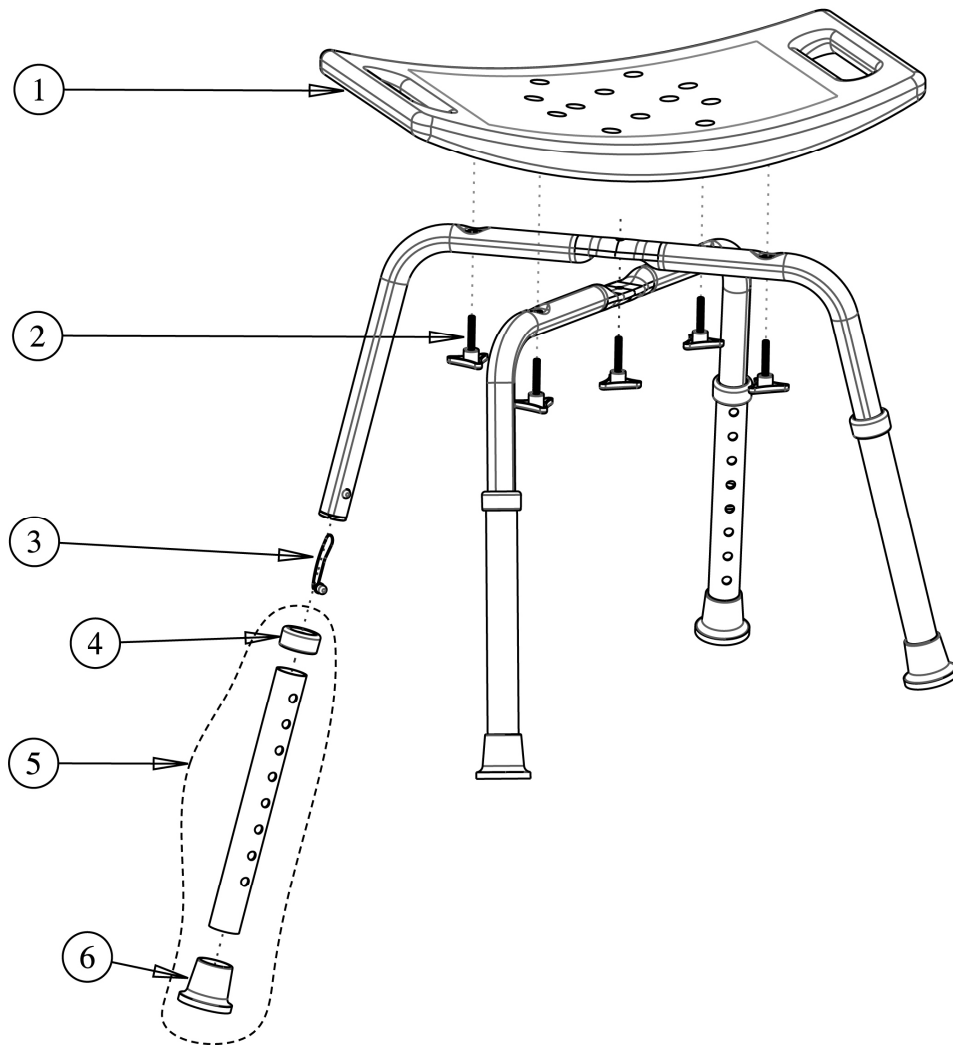


Item #	AMG Part #	Item Description	Qty Per Unit	Sold by	UOM
1.	A11-402	Seat, Blow-Molded Plastic, Aqua-Blue (505x305x35mm)	1	1	EA
2.	B11-822	Hardware Set, Aqua-Blue (5 Triangular Knobs M6x34mm)	1	1	ST
3.	B11-811	Push Button dia:8mm & Spring - 4ea / set	1	1	ST
4.	A11-212	Anti-Rattle for dia:28.6mm, Aqua-Blue - 4 ea / set	1	1	ST
5.	B11-012	Extension Leg, Aqua-Blue (Aluminum Tube dia:28.6x250mm, Anti-Rattle & Rubber Tip) - 4 ea / set	1	1	ST
6.	A11-482	Rubber Tip for dia:28.6mm, Aqua-Blue - 4 ea / set	1	1	ST
Not Shown	B11-851	Hardware Set for Bath Bench w/out Back (5 Cross-Cup Screws M6x34mm, 5 Curved Washers) - Accessory	1	1	ST

770-507 AQUASENSE ADJUSTABLE BATH BENCH - YELLOW

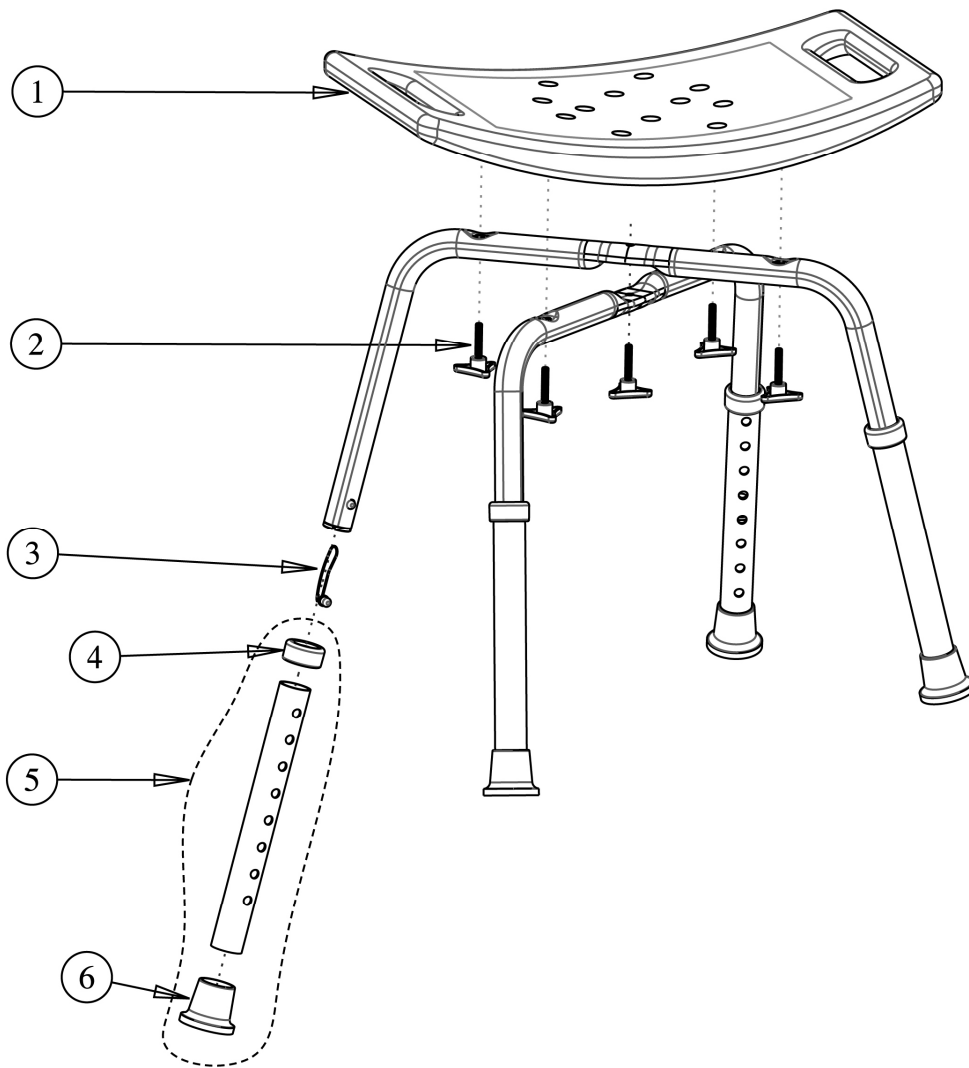


Item #	AMG Part #	Item Description	Qty Per Unit	Sold by	UOM
1.	A11-403	Seat, Blow-Molded Plastic, Yellow (505x305x35mm)	1	1	EA
2.	B11-823	Hardware Set, Yellow (5 Triangular Knobs M6x34mm)	1	1	ST
3.	B11-811	Push Button dia:8mm & Spring - 4ea / set	1	1	ST
4.	A11-213	Anti-Rattle for dia:28.6mm, Yellow - 4 ea / set	1	1	ST
5.	B11-013	Extension Leg, Yellow (Aluminum Tube dia:28.6x250mm, Anti-Rattle & Rubber Tip) - 4 ea / set	1	1	ST
6.	A11-483	Rubber Tip for dia:28.6mm, Yellow - 4 ea / set	1	1	ST
Not Shown	B11-851	Hardware Set for Bath Bench w/out Back (5 Cross-Cup Screws M6x34mm, 5 Curved Washers) - Accessory	1	1	ST

770-508 AQUASENSE ADJUSTABLE BATH BENCH - TAUPE

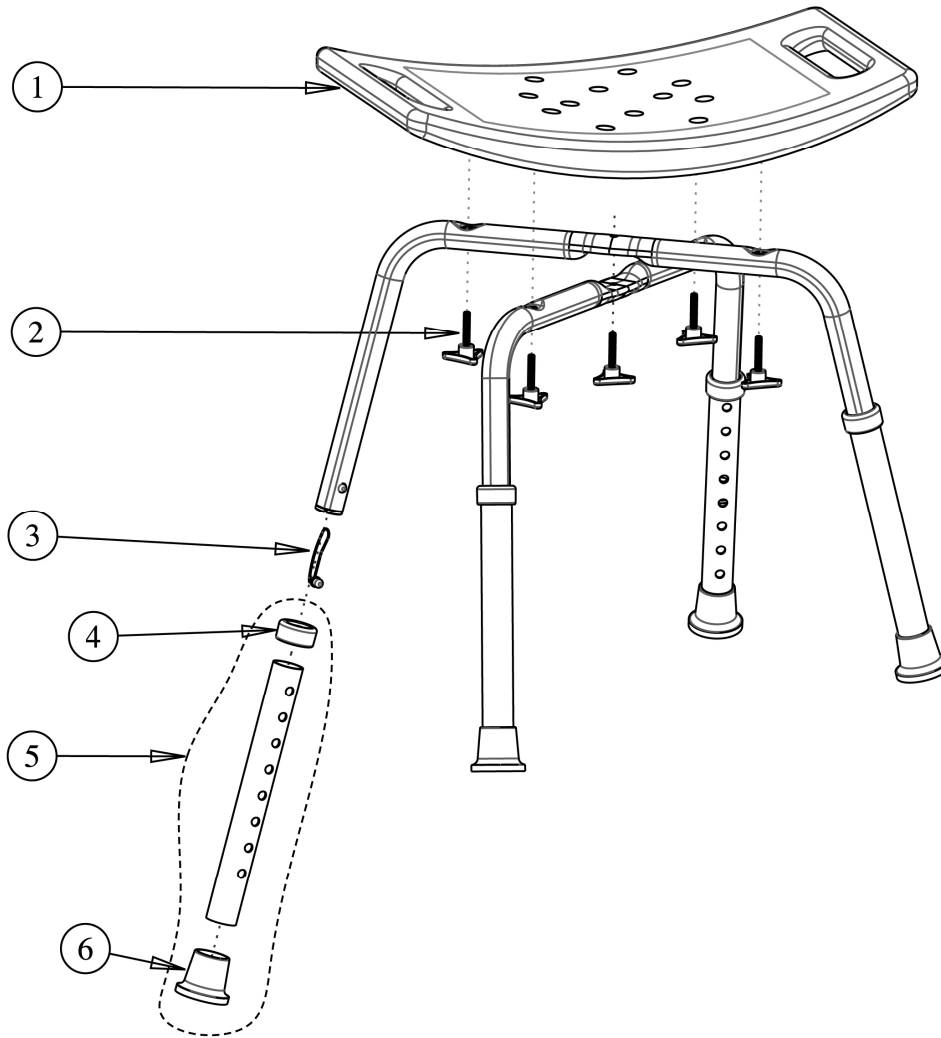

Item #	AMG Part #	Item Description	Qty Per Unit	Sold by	UOM
1.	A11-404	Seat, Blow-Molded Plastic, Taupe (505x305x35mm)	1	1	EA
2.	B11-824	Hardware Set, Taupe (5 Triangular Knobs M6x34mm)	1	1	ST
3.	B11-811	Push Button dia:8mm & Spring - 4ea / set	1	1	ST
4.	A11-214	Anti-Rattle for dia:28.6mm, Taupe - 4 ea / set	1	1	ST
5.	B11-014	Extension Leg, Taupe (Aluminum Tube dia:28.6x250mm, Anti-Rattle & Rubber Tip) - 4 ea / set	1	1	ST
6.	A11-484	Rubber Tip for dia:28.6mm, Taupe - 4 ea / set	1	1	ST
Not Shown	B11-851	Hardware Set for Bath Bench w/out Back (5 Cross-Cup Screws M6x34mm, 5 Curved Washers) - Accessory	1	1	ST

770-509 AQUASENSE ADJUSTABLE BATH BENCH

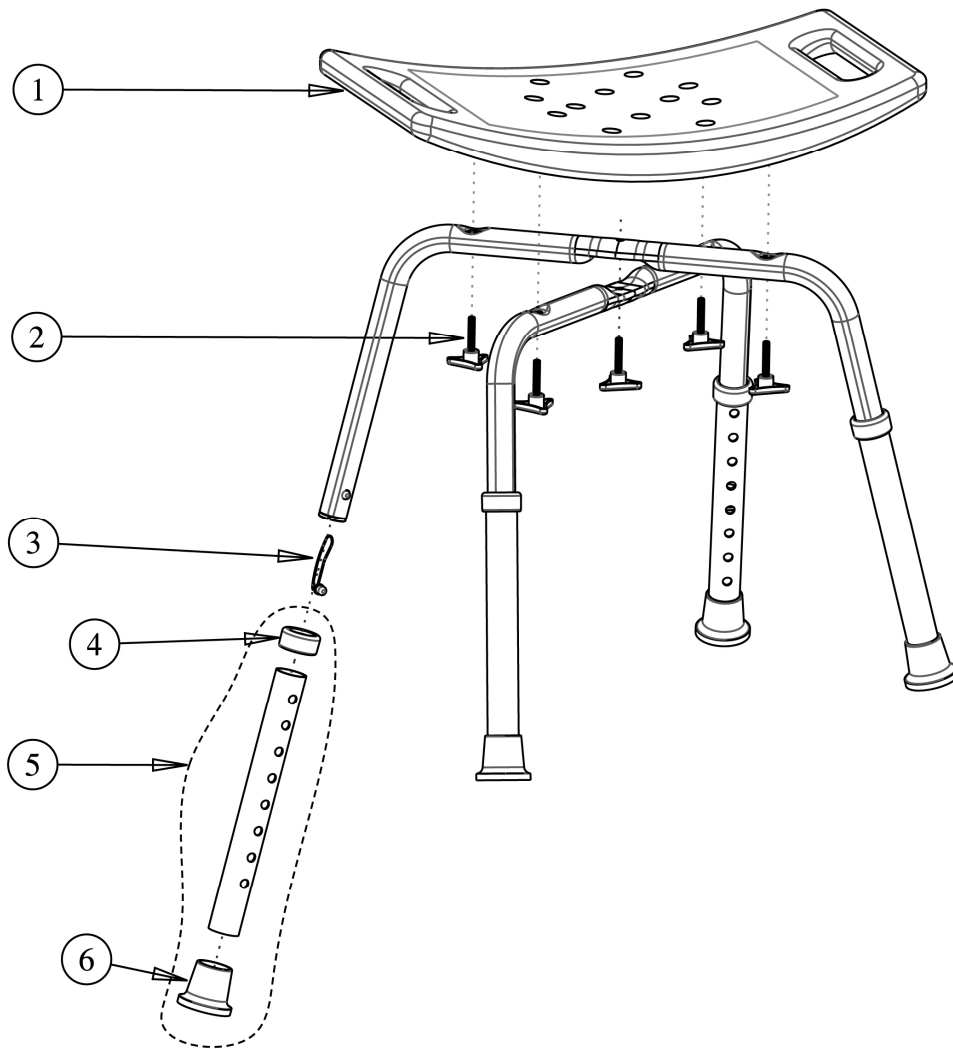

Item #	AMG Part #	Item Description	Qty Per Unit	Sold by	UOM
1.	A11-401	Seat, Blow-Molded Plastic, White (505x305x35mm)	1	1	EA
2.	B11-822	Hardware Set, Aqua-Blue (5 Triangular Knobs M6x32mm)	1	1	ST
3.	B11-811	Push Button dia:8mm & Spring - 4ea / set	1	1	ST
4.	A11-212	Anti-Rattle for dia:28.6mm, Aqua-Blue – 4 ea / set	1	1	ST
5.	B11-012	Extension Leg, Aqua-Blue (Aluminum Tube dia:28.6x250mm, Anti-Rattle & Rubber Tip) – 4 ea / set	1	1	ST
6.	A11-482	Rubber Tip for dia:28.6mm, Aqua-Blue – 4 ea / set	1	1	ST
Not Shown	B11-851	Hardware Set for Bath Bench (5 Cross-Cap Screws M6x32mm, 5 Curved Washers & 1 Locknut) -- Accessory (screw set to replace triangular knobs)	1	1	ST