

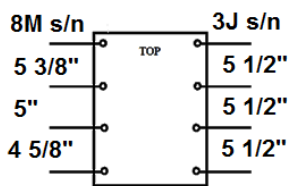


## CHROME SPORT

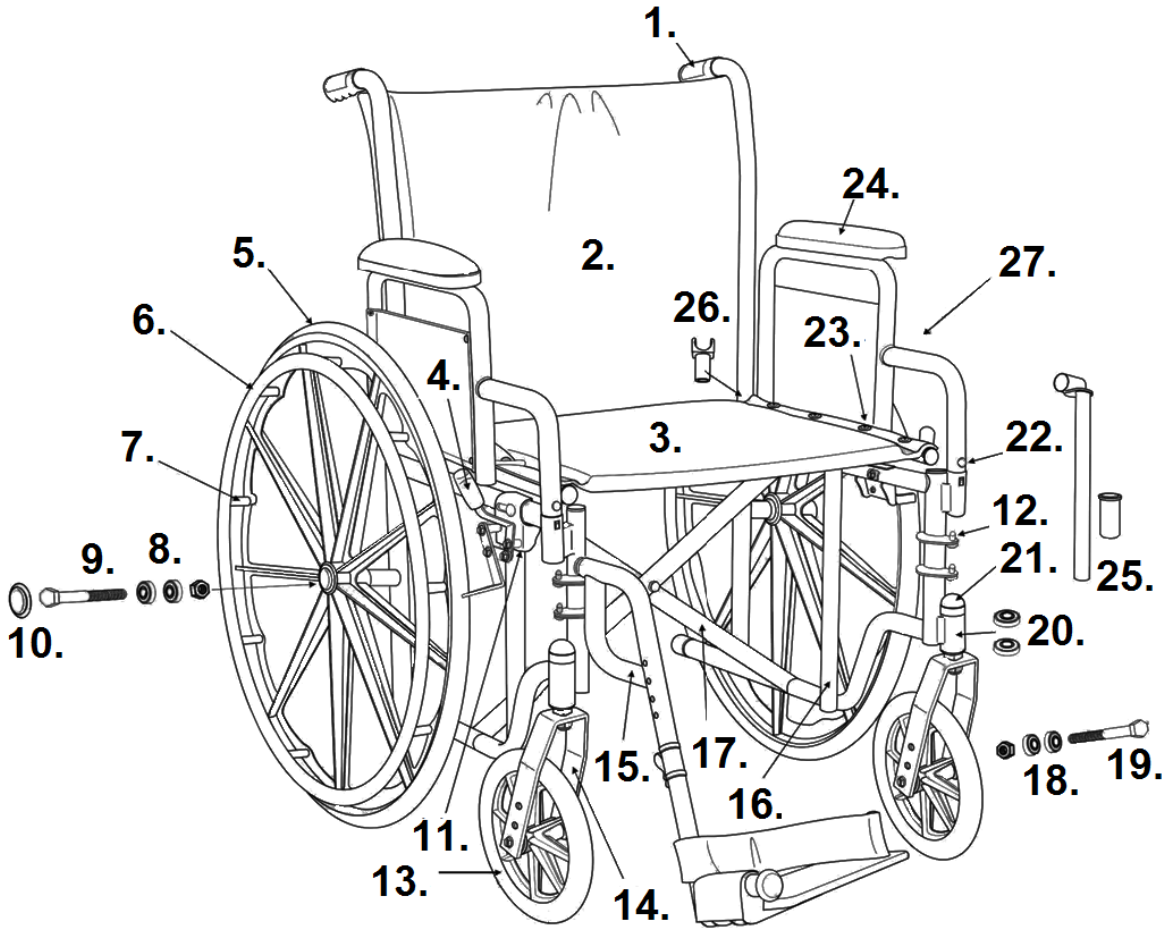

Back Hole Spacing

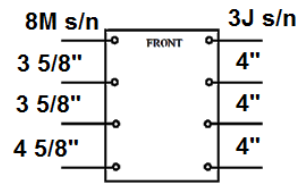

Seat Hole Spacing

\* 8M s/n 10/2009 & before.  
\*\* 3J s/n 11/2009 & after.

- |   |   |
|---|---|
| <ul style="list-style-type: none"> <li>1. Hand Grip- WB 8036</li> <li>2. Back Upholstery 8M s/n 16" *- STDS8M4606<br/>Back Upholstery 8M s/n 18" *- STDS8M4608<br/>Back Upholstery 8M s/n 20" *- STDS8M4602<br/>Back Upholstery 3J s/n 16" **- STDS3J4606<br/>Back Upholstery 3J s/n 18" **- STDS3J4608<br/>Back Upholstery 3J s/n 20" **- STDS3J4602</li> <li>3. Seat Upholstery 8M s/n 16"- STDS8M4616<br/>Seat Upholstery 8M s/n 18"- STDS8M4618<br/>Seat Upholstery 8M s/n 20"- STDS8M4620<br/>Seat Upholstery 3J s/n 16"- STDS3J4616<br/>Seat Upholstery 3J s/n 18"- STDS3J4618<br/>Seat Upholstery 3J s/n 20"- STDS3J4620</li> <li>4. Wheel Lock Tip- STDS8M4603</li> <li>5. Rear Wheel- STDS3J2424</li> <li>6. Hand Rim- STDS3J2400</li> <li>7. Hand Rim Hardware (9 ea)- STDS004609-9</li> <li>8. Rear Wheel Bearing- STDS3J2413</li> <li>9. Rear Axle Detachable Arms- STDS3J2425<br/>Rear Axle Fixed Arm- STDS3J4625</li> <li>10. Hub Cap- STDS3J2421</li> <li>11. Wheel Lock Det. Arms Left- STDS8M482L<br/>Wheel Lock Det. Arms Right- STDS8M482R<br/>Wheel Lock Fixed Arms Left- STDS8M481L<br/>Wheel Lock Fixed Arms Right- STDS8M481R</li> <li>12. Hanger Pins (4 ea)- STDS4S1069</li> <li>13. Caster- STDS4S2432</li> <li>14. Fork- STDS8M4633</li> </ul> | <ul style="list-style-type: none"> <li>15. Footrest (pr)- STDSF-TF<br/>Elevating Leg Rest (pr)- LK3JELR</li> <li>16. Side Frame Det. Arms Left- STDS8M48L1<br/>Side Frame Det. Arms Right- STDS8M48R1<br/>Side Frame Fixed Arms Left- STDS8M48L2<br/>Side Frame Fixed Arms Right- STDS8M48R2</li> <li>17. Cross Brace 3J **- STDS3J46X6<br/>Cross Brace 3J **- STDS3J46X8<br/>Cross Brace 3J **- STDS3J46X2</li> <li>18. Caster Bearing- STDS1011HD</li> <li>19. Caster Axle- STDS8M4631</li> <li>20. Fork Stem Bearing- STDS8M4601</li> <li>21. Dust Cap- STDS8M4651</li> <li>22. Arm Release Button- STDS3J4503</li> <li>23. Back Uphol. Hardware (8 ea)- STDS2015<br/>Seat Uphol. Hardware (8 ea)- STDS3J4510</li> <li>24. Desk Arm Pad- STDS8M48DL<br/>Full Arm Pad- STDS8M48FL</li> <li>25. Slide Tube &amp; Bushing- STDS3J4601</li> <li>26. Seat Rail Guide Det. Arms- STDS1070<br/>Seat Rail Guide Fixed Arms Front- STDS004602<br/>Seat Rail Guide Fixed Arms Rear- STDS1070</li> <li>27. Desk Arm Assembly Left- STDS8M48DL<br/>Desk Arm Assembly Right- STDS8M48RL<br/>Full Arm Assembly Left- STDS8M48FL<br/>Full Arm Assembly Right- STDS8M48FR<br/>Adjustable Height Arms (pr)- CSADFA</li> </ul> |
|---|---|