


SILVER SPORT 1 (FOLD DOWN BACK - SERIAL 4S1605312000 & LATER)

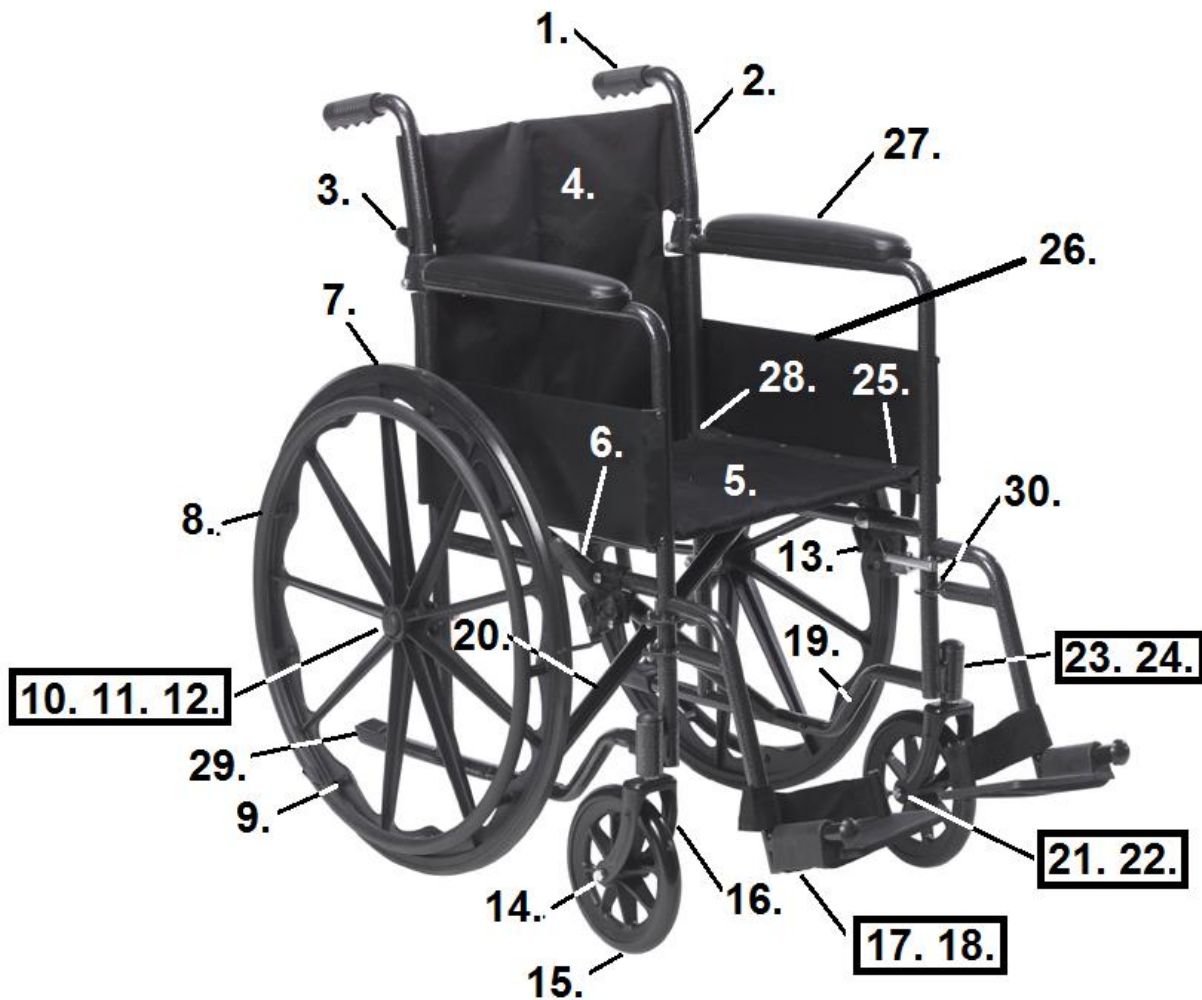

- | | |
|---|---|
| 1. Hand Grip- WB 8036 | 17. Footrest- STDS3J24SF |
| 2. Push Handle- STDS4S4501 | Elevating Leg Rest- LK3JELR |
| 3. Folding Hinge- STDS4S1503 | 18. Footrest Adjustment Lever- STDS3J4502 |
| 4. Back Upholstery- STDS4S4504 | 19. Side Frame Left- STDS4S4505L |
| 5. Seat Upholstery- STDS3J2418 | Side Frame Right- STDS4S4505R |
| 6. Wheel Lock Tip- STDS8M4603 | 20. Cross Frame- STDS3J45X8 |
| 7. Rear Wheel- STDS3J2424 | 21. Caster Bearing (8 mm)- STDS1011 |
| 8. Hand Rim- STDS3J2400 | 22. Caster Axle- STDS3J2431 |
| 9. Hand Rim Hardware (9 ea)- STDS004609-9 | 23. Fork Stem Bearing (12.5mm)- STDS8M4601 |
| 10. Rear Wheel Bearing (11mm)- STDS3J2413 | 24. Dust Cap- STDS8M4651 |
| 11. Rear Axle- STDS3J4625 | 25. Uphol. Hardware Back/ Seat (8 ea)- STDS2015 |
| 12. Hub Cap- STDS3J2421 | 26. Side Guard- STDS4S4503 |
| 13. Wheel Lock Left- STDS4S4502L | 27. Full Arm Pad- STDSFLNB |
| Wheel Lock Right- STDS4S4502R | 28. Seat Rail Guide Front- STDS004602 |
| 14. Axle Cap- STDS001000 | Seat Rail Guide Rear- STDS1070 |
| 15. Caster 8 Spoke- STDS3J2432 | 29. Tipping Sleeve- STDS3J2422 |
| 16. Fork- STDS3J4533 | 30. Hanger Pins (4 ea)- STDS4S1069 |