TM
drive

grab bar,
white powder coated
barra de apoyo,
cubierta de polvo blanco
barre d'appui, peinture
poudre blanche

item #s:

RTL12012

12016-3

RTL12018

12024-3

12032-3

EU Authorized Representative
CMC Medical Devices & Drugs S.L.
C/Horacio Lengo 18 CP 29006
Málaga, Spain

www.drivemedical.com

TM
drive

mounting instructions

NOTE: Mounting flanges of grab bar must be aligned to mount to wall studs. Wood screws must go through the wall board or tile and attach to wood wall studs for solid support. For plastic tub and shower enclosures, drill through wall and mount grab bar to wall studs.

1. Place grab bar on wall in line with wall studs and mark location of screw holes. (Horizontal Attachment See Figure 1, Vertical Attachment See Figure 2.) After deciding on type of attachment, drill small hole to confirm the location of the stud.

CAUTION: Wood screws should only be used in wooded wall studs.

2. Determine for each flange whether to use wood screws or toggle bolts (not included).

WARNING: At least two wood screws in each flange must be attached to wall studs or grab bar may pull out and serious injury could result

3. Use masonry bit to drill through tile or grout. For wood screws, drill small pilot hole into wood studs.
4. After holes are drilled, place grab bar on wall and attach with wood screw. If using toggle bolts (not included), make sure bolt is attached to grab bar flange before being forced into hole.

NOTE: Use caulking around grab bar flanges to prevent moisture from seeping in.

NOTE: Clean with mild soap and rinse with warm water.

5. Tighten all screws and bolts with screwdriver. Check periodically for tightness. Never use grab bar if any looseness is detected as it could result in personal injury.

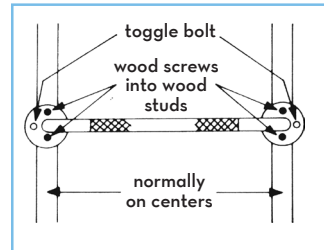
figure 1

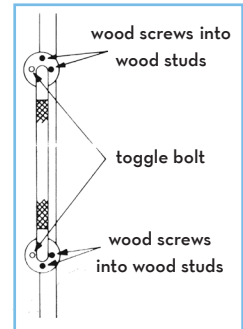
figure 2

This grab bar meets ADA & CSA Standards for Accessible Design.

The ADA & CSA provide guidelines for public accommodations and commercial facilities. Under these guidelines grab bars must meet following requirements.

- Installed horizontally
- Provide a sizeable gripping surface
- Diameters of 1.25" - 2"
- Space between the grab bar and the wall of at least 1.5", so that someone can easily grab onto the bar
- Cannot rotate in their fittings, as that makes it more difficult to use as support
- Support up to 250 lbs.

Your Drive grab bar meets each of these standards.

Below are some additional tips to consider when installing a grab bar in your own personal residence:

- Make sure surroundings have rounded edges and are free of sharp edges
- Objects below and at the ends of the bar should be at least 1.5" away
- Objects above should be at least 12" away

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maintenance

Drive recommends periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product. If you experience problems, please contact the provider that sold you the product or visit www.drivemedical.com/help for more info.

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2020 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

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© 2020 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2020 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.