


PANTHER

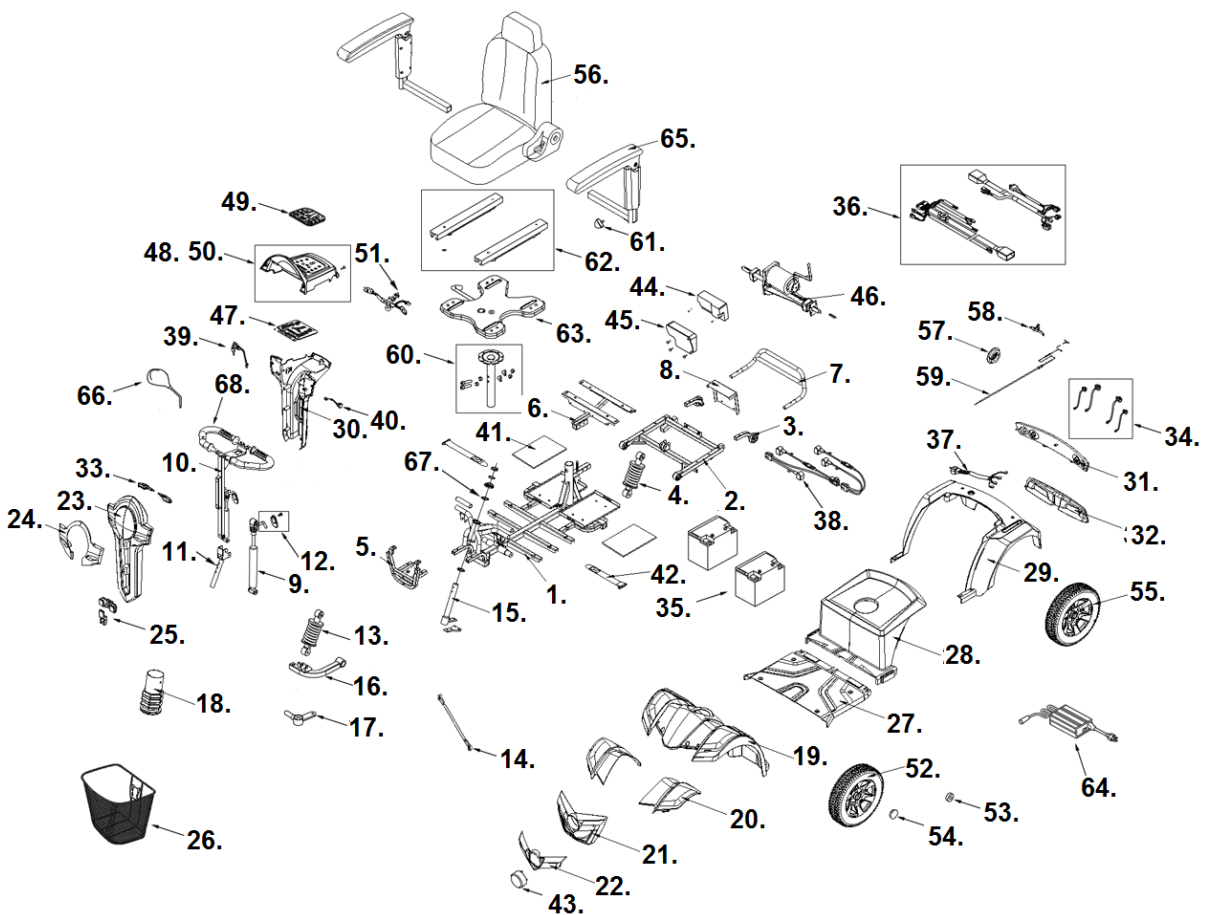

- | | |
|--|--|
| 1. Front Frame- C01-077-00101 | 39. Key Switch- C18-001-01501
Key- C37-001-02300 |
| 2. Rear Frame- C01-077-00200 | 40. Charger Socket- C12-051-00200 |
| 3. Anti-Tipper Wheel- C28-032-00100 | 41. Battery Pad- C25-005-00100 |
| 4. Rear Shock- C04-040-00100 | 42. Battery Strap- C31-029-00101 |
| 5. Front Bumper- C01-077-00400 | 43. Head Light- C13-077-00100 |
| 6. Rear Shroud Bracket- C01-077-00500 | 44. Controller- C10-077-00200 |
| 7. Rear Bumper- C01-077-00600 | 45. Controller Cover- C26-051-00501 |
| 8. Controller Bracket- C10-077-00100 | 46. Motor, Brake, Transaxle Assy.- C09-077-00100
Transaxle Only- C09-076-01200
Motor Key Way- 29002W7011
Motor Only- C09-077-00300
Motor Brush Set (4 ea)- C09-029-07900
Motor Brush Covers (4 ea)- C09-029-04500
Electro Magnetic Brake- C09-076-00700
Free Wheel Lever- C09-077-00401 |
| 9. Tilt Piston- C02-029-00200 | 47. Control Panel PC Board- C10-051-00507 |
| 10. Upper Tiller- C02-077-00200 | 48. Upper Tiller Cover- C08-077-00865 |
| 11. Lower Tiller- C02-036-00400 | 49. Overlay- C10-051-00601 |
| 12. Tiller Release Handle- PANTHER-01 | 50. Battery Indicator- C27-077-00100 |
| 13. Front Shock- C03-002-00300 | 51. Throttle- S12-051-00100 |
| 14. Tie Rod- PANTHER-02 | 52. Front Wheel Assembly- S05-051-00400
Tire- C06-051-00100
Inner Tube- C06-051-00201
Front Axle Bolt- S350112AN |
| 15. Steering Fork- PANTHER-03 | 53. Front Wheel Bearing- STDS2A5303 |
| 16. Control Arm Left- C03-077-00201
Control Arm Right- C03-077-00301 | 54. Hub Cap- No Longer Available. |
| 17. Steering Knuckle- C03-001-00107 | 55. Rear Wheel Assembly Left- S06-051-00600
Rear Wheel Assembly Right- S05-051-00500
Tire- C06-051-00100
Inner Tube- C06-051-00201
Rear Axle Bolt- S350112AN |
| 18. Tiller Boot- C25-045-00100 | 56. 20" Seat Assy. Arms & Seat Belt- S07-062-00100
22" Seat Assy. Arms & Seat Belt- S07-062-00200
Headrest Only- C07-062-00400 |
| 19. Front Shroud- C08-077-00100 | 57. Brake- PANTHER-08 |
| 20. Front Shroud Decoration Red- PANTHER-04
Front Shroud Decoration Blue- PANTHER-05 | 58. Brake Lever- PANTHER-09
Conversion Kit Lt To Rt Hand- S35-077-00100 |
| 21. Front Bumper- C08-077-00400 | 59. Brake Cable Assembly- PANTHER-10 |
| 22. Front Bumper Decoration Red- C08-077-00518
Front Bumper Decoration Blue- C08-077-00551 | 60. Seat Post Assembly- PANTHER-11 |
| 23. Front Tiller Cover Red- C08-077-00618
Front Tiller Cover Blue- C08-077-00651 | 61. Adjustment Knob- S31052 |
| 24. Front Tiller Decoration- C08-077-00700 | 62. Seat Slider Set- C07-016-00600 |
| 25. Basket Mounting Bracket- BC35051 | 63. Seat Plate- C07-062-00202 |
| 29. Basket- C23-051-00200
Substitute- S35003-PL | 64. Charger- LRA402210 |
| 27. Floor Shroud- C08-077-01000 | 65. Arm Pad Only- C07-061-01900
Arm Rest Assembly Left- C07-061-01700
Arm Rest Assembly Right- C07-061-01800 |
| 28. Battery Shroud- C08-077-01100 | 66. Right Mirror- VENT-53
Left Mirror- VENT-54 |
| 29. Rear Shroud Red- C08-077-01218
Rear Shroud Blue- C08-077-01251 | 67. Steering Bearing Assembly- S02-001-00601 |
| 30. Rear Tiller Shroud- C08-077-00900 | 68. Handlebar (left/right)- C02-062-00101 |
| 31. Light Shroud Decoration- C08-077-01400 | |
| 32. Light Shroud- C08-077-01300 | |
| 33. Front Right Turn Signal Light- C15-062-00401
Front Left Turn Signal Light- C15-062-00801 | |
| 34. Rear Right Turn Signal Light- C14-062-00600
Rear Left Turn Signal Light- C14-062-01000
Rear Right Stop Signal Light- C14-077-00100
Rear Left Stop Signal Light- C14-077-00200 | |
| 35. Battery 50 Ah (2 needed)- LRB402308
Battery 75 Ah (2 needed)- LRC402407 | |
| 36. Wire Harness Assembly- C18-077-00100 | |
| 37. Rear Light Harness- C18-062-00400 | |
| 38. Negative Battery Harness- C17-029-00403
Positive Battery Harness- C17-029-00303
Battery Harness 125A Fuse- 6AYK5 | |