


TITAN LTE

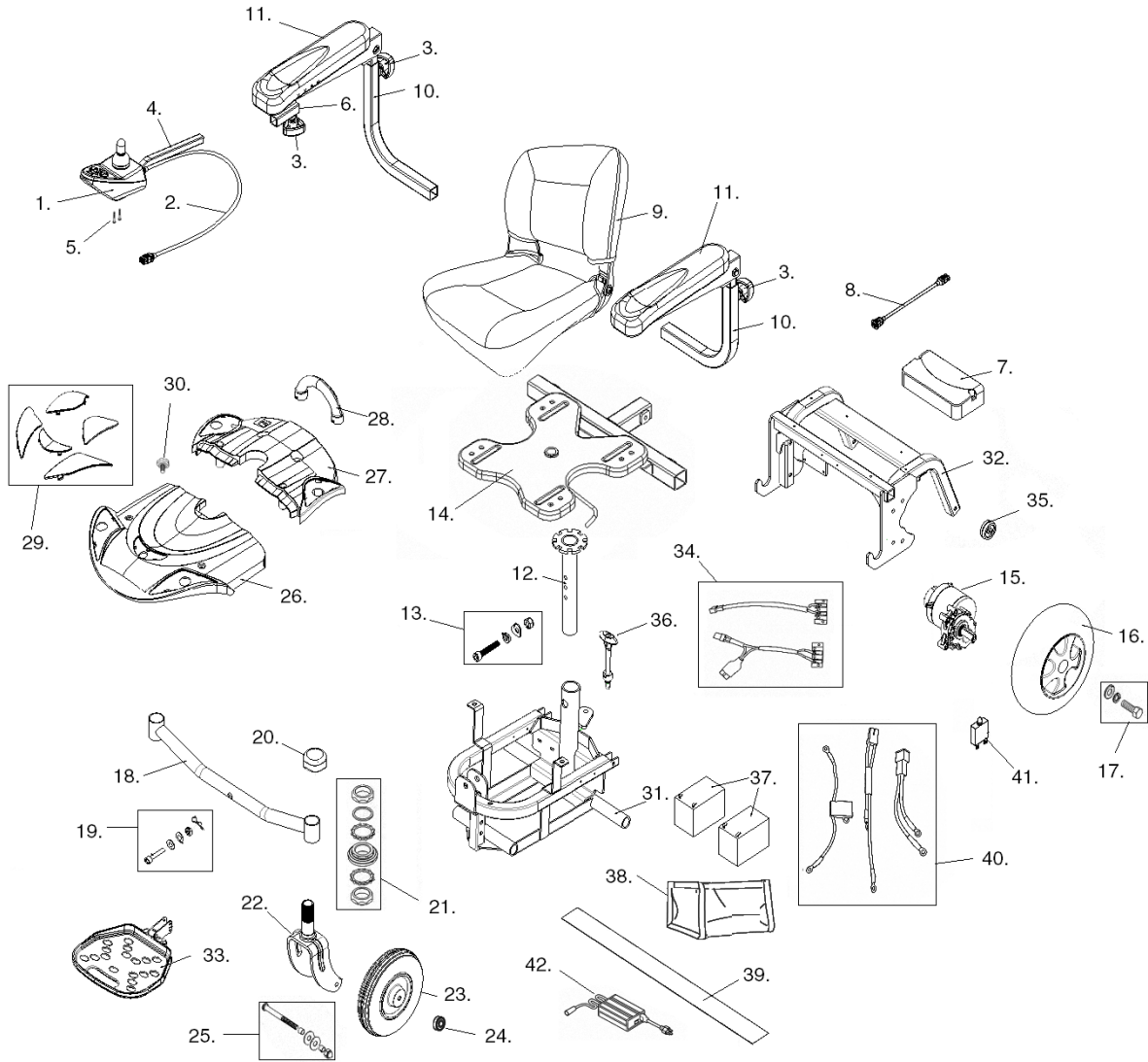

- | | |
|---|--|
| 1. Joystick- LINX-JOYSTICK
Joystick Tip Linx- DC GPL55470 | 19. Articulating Bar Mtg Hdwr- TITANLTE-13 |
| 2. Joystick Cable- GLM-BUS100-A | 20. Dust Cover- C26-084-00100 |
| 3. Locking Knob- LDM400001-07 | 21. Fork Stem Bearing Assembly- DST-010 |
| 4. Joystick Arm- LDM400001-04 | 22. Fork - C03-017-00800 |
| 5. Joystick Mtg Screw & Washer- LDM400001-01 | 23. Caster- C05-085-00100 |
| 6. Joystick Arm Mtg Bracket- LDM400001-05 | 24. Caster Bearing- SEC9031-B |
| 7. Controller- DLX-PM40-A-LTE | 25. Caster Axle- TITANLTE-03 |
| 8. Controller to Shroud Cable- GLM-EXT064-A
Controller Cable Clamp- C10-085-00500 | 26. Front Shroud- C08-085-00100 |
| 9. Seat Assembly w/o Arms- DST-013 | 27. Rear Shroud- C08-085-00500 |
| 10. Armrest Assembly Left- LDR410011L
Armrest Assembly Right- LDR410011R | 28. Handle- TITANLTE-16 |
| 11. Arm Pad- LDR410110 | 29. Red Decorations Set- TITANLTE-07
Blue Decorations Set- TITANLTE-08 |
| 12. Seat Post *- C07-069-00201 | 30. Shroud Knob- VENT-86 |
| 13. Seat Post Bolt- TITANLTE-11 | 31. Front Frame- C01-085-00100 |
| 14. Seat Plate- C07-072-01001 | 32. Rear Frame- C01-085-00200 |
| 15. Motor, Gearbox, Brake Left- C09-085-00200
Motor, Gearbox, Brake Right- C09-085-00100
Brake Only Left- C09-085-00700
Brake Only Right- C09-085-00800
Motor Key Way- C88-085-00100 | 33. Footrest & Mount- TITANLTE-09 |
| 16. Rear Wheel- S31132B-HD | 34. Contact Block Assy & Hrwr- TITANLTE-14 |
| 17. Rear Wheel Mtg Bolt & Washers- TITANLTE-04 | 35. Anti-Tipper Assembly- TITANLTE-06 |
| 18. Articulating Bar- TITANLTE-01 | 36. Frame Lock Bolt Assembly- TITANLTE-05 |
| | 37. Battery 12V 20Ah (2 needed)- LRM412123 |
| | 38. Battery Bag- C17-085-00100 |
| | 39. Battery Strap- C31-005-00400 |
| | 40. Wire Harness- TITANLTE-12 |
| | 41. Circuit Breaker 20 Amp- LRM302107 |
| | 42. Charger (2 amp)- LRM302111 |

*** When current stock is gone the part will no longer be chrome plated. It will be black.**