


ZOOME AUTO-FLEX

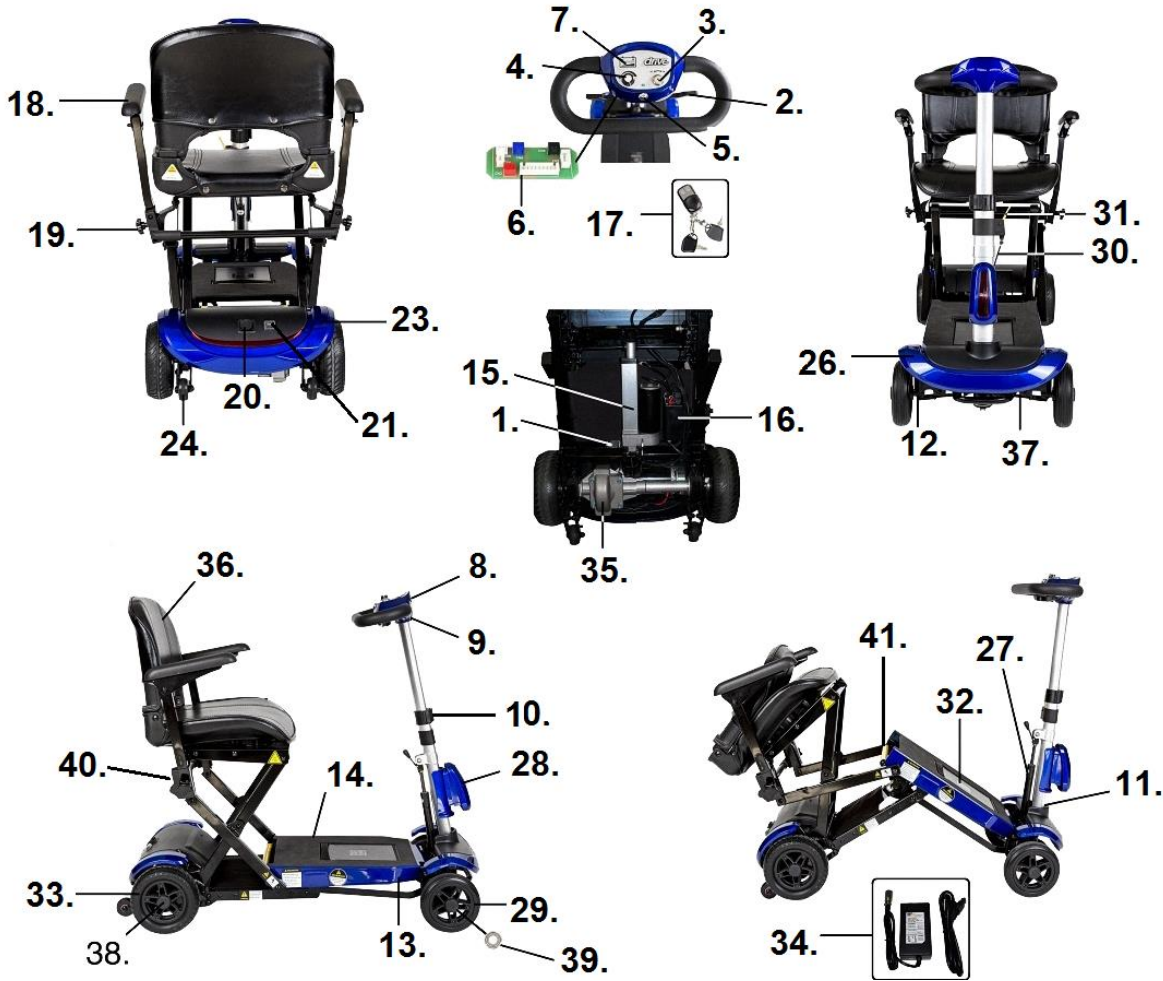

- 1. Controller- FLEX-29
 - 2. Throttle Pot- FLEX-AUTO-01
Throttle Pot Assembly- FLEX-AUTO-02
Throttle Lever- FLEX-AUTO-03
 - 3. On-Off Switch- FLEX-AUTO-05
 - 4. Speed Pot- FLEX-AUTO-06
 - 5. Horn Button- FLEX-AUTO-07
Horn- FLEX-AUTO-04
 - 6. Junction Board- FLEX-AUTO-08
 - 7. Battery Gauge- FLEX-AUTO-09
 - 8. Upper Console Blue- FLEX-AUTO-10
Upper Console Red- FLEX-AUTO-10R
 - 9. Lower Console Blue- FLEX-AUTO-11
Lower Console - FLEX-AUTO-11R
 - 10. Quick Release Clamp- FLEX-AUTO-12
 - 11. Handle Connector- FLEX-AUTO-13
 - 12. Steering Knuckle, Left- FLEX-AUTO-14
Steering Knuckle, Right- FLEX-AUTO-15
Steering Knuckle Bearing - FLEX-11
 - 13. Footboard Blue- FLEX-AUTO-16
Floorboard Red- FLEX-AUTO-16R
 - 14. Floor Mat- FLEX-AUTO-17
 - 15. Folding Actuator Assembly- FLEX-AUTO-18
Folding Actuator Assembly- FLEX-AUTO-51 *
 - 16. Wireless Receiver- FLEX-AUTO-19
Wireless Receiver- FLEX-AUTO-50 *
Wireless Receiver- FLEX-AUTO-50N **
 - 17. Key- FLEX-AUTO-20
Wireless Remote Key- FLEX-AUTO-21
 - 18. Armrest Assembly, Left- FLEX-AUTO-23
Armrest Assembly, Right- FLEX-AUTO-24
Arm Pad- FLEX-37
 - 19. Armrest Knob- FLEX-AUTO-25
 - 20. Charger Port- FLEX-AUTO-26
 - 21. Actuator Switch- FLEX-AUTO-27
 - 22. Reverse Horn- FLEX-AUTO-28
 - 23. Rear Shroud Blue- FLEX-AUTO-29
Rear Shroud Red- FLEX-AUTO-29R
Rear Shroud Decoration Blue- FLEX-AUTO-30
 - 24. Anti-Tipper Wheel- FLEX-AUTO-31
 - 25. Main Harness- FLEX-AUTO-45
 - 26. Front Cover- FLEX-12
 - 27. Gas Spring Assembly- FLEX-AUTO-34
 - 28. Front Reflector Assembly Blue- FLEX-AUTO-39
Front Reflector Assembly Red- FLEX-AUTO-39R
 - 29. Front Wheel- FLEX-07
 - 30. Release Cable- FLEX-AUTO-46
 - 31. Release Lever- FLEX-21
 - 32. Battery Pack- FLEX-22 ***
Battery Case Only- FLEX-16 ***
Battery Pack- FLEX-AUTO-47 ****
Battery Case Only- FLEX-AUTO-48 ****
Battery Contacts (4 ea)- FLEX-31
 - 33. Rear Wheel- FLEX-25
 - 34. Battery Charger- FLEX-27
 - 35. Motor/Brake/Transaxle Assembly- FLEX-AUTO-44
Transaxle- FLEX-51
Motor- FLEX-52
Brake- FLEX-AUTO-43
 - 36. Seat Complete- FLEX-AUTO-37
Back Upholstery- FLEX-AUTO-36
Seat Upholstery- FLEX-AUTO-35
 - 37. Tie Rod Complete- FLEX-13
 - 38. Hub Cap- FLEX-10
 - 39. Wheel/Stem Bearing- FLEX-11
 - 40. Armrest Receiver, Left- FLEX-AUTO-32
Armrest Receiver, Right- FLEX-AUTO-33
 - 41. Folding Latch Assembly Yellow- FLEX-AUTO-38
Folding Latch Assembly Red- FLEX-AUTO-49 *
- Not Shown-
External Charging Docking Station- FLEX-50
Basket Only- FLEX-AUTO-40
Basket mount- FLEX-AUTO-41
Cup Holder- FLEX-AUTO-42

* Between Serial # (21)24S1803000001 & Serial # (21)24S1909000001
** Serial # (21) 24S1909000001 and higher
*** Serial # lower than (21)24S1712000001
**** Serial # (21)24S1712000001 & higher